

Cato[®] edition Anaesthetic Workstation

Instructions for Use

Software versions:

Ventilator: 7.n

Monitor: 2.n



Working with these Instructions for Use

In the header line...

the subject of the main section

The sub-section title is given underneath the main title to help you find your way rapidly through the manual.

Page body

Instructions for Use

in text-graphic combination. The information is translated directly into physical actions that teach the user in practical steps how to use the apparatus.

Left-hand column...

the text

contains explanations and guides the user in the operation of the product with concise, clear and unmistakable instructions in ergonomic sequence.

The bullets (dots) indicate separate steps, and, when several steps are described, the numbers indicate details in the illustrations and also specify the order of action.

Right-hand column...

the illustrations

provide a visual reference for the text and make it easier to locate the various parts of the equipment. Details mentioned in the text are highlighted. Irrelevant details are omitted.

The user is also prompted by screen displays which confirm the operating sequence.

Operation
Manual / Spontaneous
Select Manual / Spontaneous mode
Spontaneous breathing


Manual / Spontaneous

Before connecting a patient


- Check workstation with checklist (see page 21).
- Check that breathing system is complete (see page 20) and
- perform leakage test (see page 31).

Select Manual / Spontaneous mode

- 1 Press »MAN/SPONT« on ventilator for at least one second –
- 2 Display in dialogue field:
MAN/SPONT

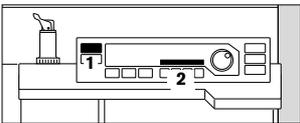

The standard screen appears with the alarm limits for MAN/SPONT mode.

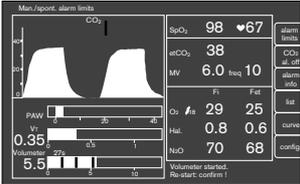

Man./spont. alarm limits		alarm limits	
CO ₂	98	SpO ₂	67
etCO ₂	38	etCO ₂	38
MV	6.0	MV	10
Fi	29	Fi	25
Di	0.8	Di	0.6
NiO	70	NiO	68

Volume: 0.35
Volume: 5.5

Volume: 22
Volume: 0.5
Volume: 1

Volume: 2
Volume: 4
Volume: 6

Volume: 8
Volume: 10

Volume: 12
Volume: 14
Volume: 16

Volume: 18
Volume: 20
Volume: 22

Volume: 24
Volume: 26
Volume: 28

Volume: 30
Volume: 32
Volume: 34

Volume: 36
Volume: 38
Volume: 40

Volume: 42
Volume: 44
Volume: 46

Volume: 48
Volume: 50
Volume: 52

Volume: 54
Volume: 56
Volume: 58

Volume: 60
Volume: 62
Volume: 64

Volume: 66
Volume: 68
Volume: 70

Volume: 72
Volume: 74
Volume: 76

Volume: 78
Volume: 80
Volume: 82

Volume: 84
Volume: 86
Volume: 88

Volume: 90
Volume: 92
Volume: 94

Volume: 96
Volume: 98
Volume: 100

Volume: 102
Volume: 104
Volume: 106

Volume: 108
Volume: 110
Volume: 112

Volume: 114
Volume: 116
Volume: 118

Volume: 120
Volume: 122
Volume: 124

Volume: 126
Volume: 128
Volume: 130

Volume: 132
Volume: 134
Volume: 136

Volume: 138
Volume: 140
Volume: 142

Volume: 144
Volume: 146
Volume: 148

Volume: 150
Volume: 152
Volume: 154

Volume: 156
Volume: 158
Volume: 160

Volume: 162
Volume: 164
Volume: 166

Volume: 168
Volume: 170
Volume: 172

Volume: 174
Volume: 176
Volume: 178

Volume: 180
Volume: 182
Volume: 184

Volume: 186
Volume: 188
Volume: 190

Volume: 192
Volume: 194
Volume: 196

Volume: 198
Volume: 200
Volume: 202

Volume: 204
Volume: 206
Volume: 208

Volume: 210
Volume: 212
Volume: 214

Volume: 216
Volume: 218
Volume: 220

Volume: 222
Volume: 224
Volume: 226

Volume: 228
Volume: 230
Volume: 232

Volume: 234
Volume: 236
Volume: 238

Volume: 240
Volume: 242
Volume: 244

Volume: 246
Volume: 248
Volume: 250

Volume: 252
Volume: 254
Volume: 256

Volume: 258
Volume: 260
Volume: 262

Volume: 264
Volume: 266
Volume: 268

Volume: 270
Volume: 272
Volume: 274

Volume: 276
Volume: 278
Volume: 280

Volume: 282
Volume: 284
Volume: 286

Volume: 288
Volume: 290
Volume: 292

Volume: 294
Volume: 296
Volume: 298

Volume: 300
Volume: 302
Volume: 304

Volume: 306
Volume: 308
Volume: 310

Volume: 312
Volume: 314
Volume: 316

Volume: 318
Volume: 320
Volume: 322

Volume: 324
Volume: 326
Volume: 328

Volume: 330
Volume: 332
Volume: 334

Volume: 336
Volume: 338
Volume: 340

Volume: 342
Volume: 344
Volume: 346

Volume: 348
Volume: 350
Volume: 352

Volume: 354
Volume: 356
Volume: 358

Volume: 360
Volume: 362
Volume: 364

Volume: 366
Volume: 368
Volume: 370

Volume: 372
Volume: 374
Volume: 376

Volume: 378
Volume: 380
Volume: 382

Volume: 384
Volume: 386
Volume: 388

Volume: 390
Volume: 392
Volume: 394

Volume: 396
Volume: 398
Volume: 400

Volume: 402
Volume: 404
Volume: 406

Volume: 408
Volume: 410
Volume: 412

Volume: 414
Volume: 416
Volume: 418

Volume: 420
Volume: 422
Volume: 424

Volume: 426
Volume: 428
Volume: 430

Volume: 432
Volume: 434
Volume: 436

Volume: 438
Volume: 440
Volume: 442

Volume: 444
Volume: 446
Volume: 448

Volume: 450
Volume: 452
Volume: 454

Volume: 456
Volume: 458
Volume: 460

Volume: 462
Volume: 464
Volume: 466

Volume: 468
Volume: 470
Volume: 472

Volume: 474
Volume: 476
Volume: 478

Volume: 480
Volume: 482
Volume: 484

Volume: 486
Volume: 488
Volume: 490

Volume: 492
Volume: 494
Volume: 496

Volume: 498
Volume: 500
Volume: 502

Volume: 504
Volume: 506
Volume: 508

Volume: 510
Volume: 512
Volume: 514

Volume: 516
Volume: 518
Volume: 520

Volume: 522
Volume: 524
Volume: 526

Volume: 528
Volume: 530
Volume: 532

Volume: 534
Volume: 536
Volume: 538

Volume: 540
Volume: 542
Volume: 544

Volume: 546
Volume: 548
Volume: 550

Volume: 552
Volume: 554
Volume: 556

Volume: 558
Volume: 560
Volume: 562

Volume: 564
Volume: 566
Volume: 568

Volume: 570
Volume: 572
Volume: 574

Volume: 576
Volume: 578
Volume: 580

Volume: 582
Volume: 584
Volume: 586

Volume: 588
Volume: 590
Volume: 592

Volume: 594
Volume: 596
Volume: 598

Volume: 600
Volume: 602
Volume: 604

Volume: 606
Volume: 608
Volume: 610

Volume: 612
Volume: 614
Volume: 616

Volume: 618
Volume: 620
Volume: 622

Volume: 624
Volume: 626
Volume: 628

Volume: 630
Volume: 632
Volume: 634

Volume: 636
Volume: 638
Volume: 640

Volume: 642
Volume: 644
Volume: 646

Volume: 648
Volume: 650
Volume: 652

Volume: 654
Volume: 656
Volume: 658

Volume: 660
Volume: 662
Volume: 664

Volume: 666
Volume: 668
Volume: 670

Volume: 672
Volume: 674
Volume: 676

Volume: 678
Volume: 680
Volume: 682

Volume: 684
Volume: 686
Volume: 688

Volume: 690
Volume: 692
Volume: 694

Volume: 696
Volume: 698
Volume: 700

Volume: 702
Volume: 704
Volume: 706

Volume: 708
Volume: 710
Volume: 712

Volume: 714
Volume: 716
Volume: 718

Volume: 720
Volume: 722
Volume: 724

Volume: 726
Volume: 728
Volume: 730

Volume: 732
Volume: 734
Volume: 736

Volume: 738
Volume: 740
Volume: 742

Volume: 744
Volume: 746
Volume: 748

Volume: 750
Volume: 752
Volume: 754

Volume: 756
Volume: 758
Volume: 760

Volume: 762
Volume: 764
Volume: 766

Volume: 768
Volume: 770
Volume: 772

Volume: 774
Volume: 776
Volume: 778

Volume: 780
Volume: 782
Volume: 784

Volume: 786
Volume: 788
Volume: 790

Volume: 792
Volume: 794
Volume: 796

Volume: 798
Volume: 800
Volume: 802

Volume: 804
Volume: 806
Volume: 808

Volume: 810
Volume: 812
Volume: 814

Volume: 816
Volume: 818
Volume: 820

Volume: 822
Volume: 824
Volume: 826

Volume: 828
Volume: 830
Volume: 832

Volume: 834
Volume: 836
Volume: 838

Volume: 840
Volume: 842
Volume: 844

Volume: 846
Volume: 848
Volume: 850

Volume: 852
Volume: 854
Volume: 856

Volume: 858
Volume: 860
Volume: 862

Volume: 864
Volume: 866
Volume: 868

Volume: 870
Volume: 872
Volume: 874

Volume: 876
Volume: 878
Volume: 880

Volume: 882
Volume: 884
Volume: 886

Volume: 888
Volume: 890
Volume: 892

Volume: 894
Volume: 896
Volume: 898

Volume: 900
Volume: 902
Volume: 904

Volume: 906
Volume: 908
Volume: 910

Volume: 912
Volume: 914
Volume: 916

Volume: 918
Volume: 920
Volume: 922

Volume: 924
Volume: 926
Volume: 928

Volume: 930
Volume: 932
Volume: 934

Volume: 936
Volume: 938
Volume: 940

Volume: 942
Volume: 944
Volume: 946

Volume: 948
Volume: 950
Volume: 952

Volume: 954
Volume: 956
Volume: 958

Volume: 960
Volume: 962
Volume: 964

Volume: 966
Volume: 968
Volume: 970

Volume: 972
Volume: 974
Volume: 976

Volume: 978
Volume: 980
Volume: 982

Volume: 984
Volume: 986
Volume: 988

Volume: 990
Volume: 992
Volume: 994

Volume: 996
Volume: 998
Volume: 1000

Volume: 1002
Volume: 1004
Volume: 1006

Volume: 1008
Volume: 1010
Volume: 1012

Volume: 1014
Volume: 1016
Volume: 1018

Volume: 1020
Volume: 1022
Volume: 1024

Volume: 1026
Volume: 1028
Volume: 1030

Volume: 1032
Volume: 1034
Volume: 1036

Volume: 1038
Volume: 1040
Volume: 1042

Volume: 1044
Volume: 1046
Volume: 1048

Volume: 1050
Volume: 1052
Volume: 1054

Volume: 1056
Volume: 1058
Volume: 1060

Volume: 1062
Volume: 1064
Volume: 1066

Volume: 1068
Volume: 1070
Volume: 1072

Volume: 1074
Volume: 1076
Volume: 1078

Volume: 1080
Volume: 1082
Volume: 1084

Volume: 1086
Volume: 1088
Volume: 1090

Volume: 1092
Volume: 1094
Volume: 1096

Volume: 1098
Volume: 1100
Volume: 1102

Volume: 1104
Volume: 1106
Volume: 1108

Volume: 1110
Volume: 1112
Volume: 1114

Volume: 1116
Volume: 1118
Volume: 1120

Volume: 1122
Volume: 1124
Volume: 1126

Volume: 1128
Volume: 1130
Volume: 1132

Volume: 1134
Volume: 1136
Volume: 1138

Volume: 1140
Volume: 1142
Volume: 1144

Volume: 1146
Volume: 1148
Volume: 1150

Volume: 1152
Volume: 1154
Volume: 1156

Volume: 1158
Volume: 1160
Volume: 1162

Volume: 1164
Volume: 1166
Volume: 1168

Volume: 1170
Volume: 1172
Volume: 1174

Volume: 1176
Volume: 1178
Volume: 1180

Volume: 1182
Volume: 1184
Volume: 1186

Volume: 1188
Volume: 1190
Volume: 1192

Volume: 1194
Volume: 1196
Volume: 1198

Volume: 1200
Volume: 1202
Volume: 1204

Volume: 1206
Volume: 1208
Volume: 1210

Volume: 1212
Volume: 1214
Volume: 1216

Volume: 1218
Volume: 1220
Volume: 1222

Volume: 1224
Volume: 1226
Volume: 1228

Volume: 1230
Volume: 1232
Volume: 1234

Volume: 1236
Volume: 1238
Volume: 1240

Volume: 1242
Volume: 1244
Volume: 1246

Volume: 1248
Volume: 1250
Volume: 1252

Volume: 1254
Volume: 1256
Volume: 1258

Volume: 1260
Volume: 1262
Volume: 1264

Volume: 1266
Volume: 1268
Volume: 1270

Volume: 1272
Volume: 1274
Volume: 1276

Volume: 1278
Volume: 1280
Volume: 1282

Volume: 1284
Volume: 1286
Volume: 1288

Volume: 1290
Volume: 1292
Volume: 1294

Volume: 1296
Volume: 1298
Volume: 1300

Volume: 1302
Volume: 1304
Volume: 1306

Volume: 1308
Volume: 1310
Volume: 1312

Volume: 1314
Volume: 1316
Volume: 1318

Volume: 1320
Volume: 1322
Volume: 1324

Volume: 1326
Volume: 1328
Volume: 1330

Volume: 1332
Volume: 1334
Volume: 1336

Volume: 1338
Volume: 1340
Volume: 1342

Volume: 1344
Volume: 1346
Volume: 1348

Volume: 1350
Volume: 1352
Volume: 1354

Volume: 1356
Volume: 1358
Volume: 1360

Volume: 1362
Volume: 1364
Volume: 1366

Volume: 1368
Volume: 1370
Volume: 1372

Volume: 1374
Volume: 1376
Volume: 1378

Volume: 1380
Volume: 1382
Volume: 1384

Volume: 1386
Volume: 1388
Volume: 1390

Volume: 1392
Volume: 1394
Volume: 1396

Volume: 1398
Volume: 1400
Volume: 1402

Volume: 1404
Volume: 1406
Volume: 1408

Volume: 1410
Volume: 1412
Volume: 1414

Volume: 1416
Volume: 1418
Volume: 1420

Volume: 1422
Volume: 1424
Volume: 1426

Volume: 1428
Volume: 1430
Volume: 1432

Volume: 1434
Volume: 1436
Volume: 1438

Volume: 1440
Volume: 1442
Volume: 1444

Volume: 1446
Volume: 1448
Volume: 1450

Volume: 1452
Volume: 1454
Volume: 1456

Volume: 1458
Volume: 1460
Volume: 1462

Volume: 1464
Volume: 1466
Volume: 1468

Volume: 1470
Volume: 1472
Volume: 1474

Volume: 1476
Volume: 1478
Volume: 1480

Volume: 1482
Volume: 1484
Volume: 1486

Volume: 1488
Volume: 1490
Volume: 1492

Volume: 1494
Volume: 1496
Volume: 1498

Volume: 1500
Volume: 1502
Volume: 1504

Volume: 1506
Volume: 1508
Volume: 1510

Volume: 1512
Volume: 1514
Volume: 1516

Volume: 1518
Volume: 1520
Volume: 1522

Volume: 1524
Volume: 1526
Volume: 1528

Volume: 1530
Volume: 1532
Volume: 1534

Volume: 1536
Volume: 1538
Volume: 1540

Volume: 1542
Volume: 1544
Volume: 1546

Volume: 1548
Volume: 1550
Volume: 1552

Volume: 1554
Volume: 1556
Volume: 1558

Volume: 1560
Volume: 1562
Volume: 1564

Volume: 1566
Volume: 1568
Volume: 1570

Volume: 1572
Volume: 1574
Volume: 1576

Volume: 1578
Volume: 1580
Volume: 1582

Volume: 1584
Volume: 1586
Volume: 1588

Volume: 1590
Volume: 1592
Volume: 1594

Volume: 1596
Volume: 1598
Volume: 1600

Volume: 1602
Volume: 1604
Volume: 1606

Volume: 1608
Volume: 1610
Volume: 1612

Volume: 1614
Volume: 1616
Volume: 1618

Volume: 1620
Volume: 1622
Volume: 1624

Volume: 1626
Volume: 1628
Volume: 1630

Volume: 1632
Volume: 1634
Volume: 1636

Volume: 1638
Volume: 1640
Volume: 1642

Volume: 1644
Volume: 1646
Volume: 1648

Volume: 1650
Volume: 1652
Volume: 1654

Volume: 1656
Volume: 1658
Volume: 1660

Volume: 1662
Volume: 1664
Volume: 1666

Volume: 1668
Volume: 1670
Volume: 1672

Volume: 1674
Volume: 1676
Volume: 1678

Volume: 1680
Volume: 1682
Volume: 1684

Volume: 1686
Volume: 1688
Volume: 1690

Volume: 1692
Volume: 1694
Volume: 1696

Volume: 1698
Volume: 1700
Volume: 1702

Volume: 1704
Volume: 1706
Volume: 1708

Volume: 1710
Volume: 1712
Volume: 1714

Volume: 1716
Volume: 1718
Volume: 1720

Volume: 1722
Volume: 1724
Volume: 1726

Volume: 1728
Volume: 1730
Volume: 1732

Volume: 1734
Volume: 1736
Volume: 1738

Volume: 1740
Volume: 1742
Volume: 1744

Volume: 1746
Volume: 1748
Volume: 1750

Volume: 1752
Volume: 1754
Volume: 1756

Volume: 1758
Volume: 1760
Volume: 1762

Volume: 1764
Volume: 1766
Volume: 1768

Volume: 1770
Volume: 1772
Volume: 1774

Volume: 1776
Volume: 1778
Volume: 1780

Volume: 1782
Volume: 1784
Volume: 1786

Volume: 1788
Volume: 1790
Volume: 1792

Volume: 1794
Volume: 1796
Volume: 1798

Volume: 1800
Volume: 1802
Volume: 1804

Volume: 1806
Volume: 1808
Volume: 1810

Volume: 1812
Volume: 1814
Volume: 1816

Volume: 1818
Volume: 1820
Volume: 1822

Volume: 1824
Volume: 1826
Volume: 1828

Volume: 1830
Volume: 1832
Volume: 1834

Volume: 1836
Volume: 1838
Volume: 1840

Volume: 1842
Volume: 1844
Volume: 1846

Volume: 1848
Volume: 1850
Volume: 1852

Volume: 1854
Volume: 1856
Volume: 1858

Volume: 1860
Volume: 1862
Volume: 1864

Volume: 1866
Volume: 1868
Volume: 1870

Volume: 1872
Volume: 1874
Volume: 1876

Volume: 1878
Volume: 1880
Volume: 1882

Volume: 1884
Volume: 1886
Volume: 1888

Volume: 1890
Volume: 1892
Volume: 1894

Volume: 1896
Volume: 1898
Volume: 1900

Volume: 1902
Volume: 1904
Volume: 1906

Volume: 1908
Volume: 1910
Volume: 1912

Volume: 1914
Volume: 1916
Volume: 1918

Volume: 1920
Volume: 1922
Volume: 1924

Volume: 1926
Volume: 1928
Volume: 1930

Volume: 1932
Volume: 1934
Volume: 1936

Volume: 1938
Volume: 1940
Volume: 1942

Volume: 1944
Volume: 1946
Volume: 1948

Volume: 1950
Volume: 1952
Volume: 1954

Volume: 1956
Volume: 1958
Volume: 1960

Volume: 1962
Volume: 1964
Volume: 1966

Volume: 1968
Volume: 1970
Volume: 1972

Volume: 1974
Volume: 1976
Volume: 1978

Volume: 1980
Volume: 1982
Volume: 1984

Volume: 1986
Volume: 1988
Volume: 1990

Volume: 1992
Volume: 1994
Volume: 1996

Volume: 1998
Volume: 2000
Volume: 2002

Volume: 2004
Volume: 2006
Volume: 2008

Volume: 2010
Volume: 2012
Volume: 2014

Volume: 2016
Volume: 2018
Volume: 2020

Volume: 2022
Volume: 2024
Volume: 2026

Volume: 2028
Volume: 2030
Volume: 2032

Volume: 2034
Volume: 2036
Volume: 2038

Volume: 2040
Volume: 2042
Volume: 2044

Volume: 2046
Volume: 2048
Volume: 2050

Volume: 2052
Volume: 2054
Volume: 2056

Volume: 2058
Volume: 2060
Volume: 2062

Volume: 2064
Volume: 2066
Volume: 2068

Volume: 2070
Volume: 2072
Volume: 2074

Volume: 2076
Volume: 2078
Volume: 2080

Volume: 2082
Volume: 2084
Volume: 2086

Volume: 2088
Volume: 2090
Volume: 2092

Volume: 2094
Volume: 2096
Volume: 2098

Volume: 2100
Volume: 2102
Volume: 2104

Volume: 2106
Volume: 2108
Volume: 2110

Volume: 2112
Volume: 2114
Volume: 2116

Volume: 2118
Volume: 2120
Volume: 2122

Volume: 2124
Volume: 2126
Volume: 2128

Volume: 2130
Volume: 2132
Volume: 2134

Volume: 2136
Volume: 2138
Volume: 2140

Volume: 2142
Volume: 2144
Volume: 2146

Volume: 2148
Volume: 2150
Volume: 2152

Volume: 2154
Volume: 2156
Volume: 2158

Volume: 2160
Volume: 2162
Volume: 2164

Volume: 2166
Volume: 2168
Volume: 2170

Volume: 2172
Volume: 2174
Volume: 2176

Volume: 2178
Volume: 2180
Volume: 2182

Volume: 2184
Volume: 2186
Volume: 2188

Volume: 2190
Volume: 2192
Volume: 2194

Volume: 2196
Volume: 2198
Volume: 2200

Volume: 2202
Volume: 2204
Volume: 2206

Volume: 2208
Volume: 2210
Volume: 2212

Volume: 2214
Volume: 2216
Volume: 2218

Volume: 2220
Volume: 2222
Volume: 2224

Volume: 2226
Volume: 2228
Volume: 2230

Volume: 2232
Volume: 2234
Volume: 2236

Volume: 2238
Volume: 2240
Volume: 2242

Volume: 2244
Volume: 2246
Volume: 2248

Volume: 2250
Volume: 2252
Volume: 2254

Volume: 2256
Volume: 2258
Volume: 2260

Volume: 2262
Volume: 2264
Volume: 2266

Volume: 2268
Volume: 2270
Volume: 2272

Volume: 2274
Volume: 2276
Volume: 2278

Volume: 2280
Volume: 2282
Volume: 2284

Volume: 2286
Volume: 2288
Volume: 2290

Volume: 2292
Volume: 2294
Volume: 2296

Volume: 2298
Volume: 2300
Volume: 2302

Volume: 2304
Volume: 2306
Volume: 2308

Volume: 2310
Volume: 2312
Volume: 2314

Volume: 2316
Volume: 2318

Contents

For your safety and that of your patients	4
Intended use	5
Quick start in an emergency	7
Operating concept	9
Preparing for use	17
Operation	35
Monitor functions	49
Troubleshooting (messages - cause - remedy)	71
Care	79
Check operational readiness	101
What's what	107
Technical data	115
Descriptions	123
Index	143

For Your Safety and that of Your Patients

Strictly follow the Instructions for Use

Any use of the apparatus requires full understanding and strict observation of these instructions. The apparatus is only to be used for purposes specified here.

Maintenance

The apparatus must be inspected and serviced regularly by trained service personnel at six monthly intervals (and a record kept).

Repair and general overhaul of the apparatus may only be carried out by trained service personnel.

We recommend that a service contract be obtained with DrägerService and that all repairs also be carried out by them. Only authentic Dräger spare parts may be used for maintenance.

Observe chapter "Maintenance Intervals".

Accessories

Do not use accessory parts other than those in the order list.

Not for use in areas of explosion hazard

This apparatus is neither approved nor certified for use in areas where combustible or explosive gas mixtures are likely to occur.

Safe connection with other electrical equipment

Electrical connections to equipment which is not listed in these Instructions for Use should only be made following consultations with the respective manufacturers or an expert.

Liability for proper function or damage

The liability for the proper function of the apparatus is irrevocably transferred to the owner or operator to the extent that the apparatus is serviced or repaired by personnel not employed or authorized by DrägerService or if the apparatus is used in a manner not conforming to its intended use.

Dräger cannot be held responsible for damage caused by non-compliance with the recommendations given above. The warranty and liability provisions of the terms of sale and delivery of Dräger are likewise not modified by the recommendations given above.

Dräger Medizintechnik GmbH

Intended use

Anaesthetic Workstation Cato®

Universally applicable anaesthetic workstation for

- Inhalation anaesthesia in semi-closed systems
- Inhalation anaesthesia in virtually closed systems with »low flow« and »minimal flow« techniques for minimum gas and anaesthetic consumption, with:
- Inhalation anaesthesia in non-rebreathing systems with separate fresh gas outlet for connecting e.g. Kuhn system.
- Automatic ventilation (IPPV)
- Synchronized intermittent mandatory ventilation (SIMV)
- Manual ventilation (MAN)
- Spontaneous breathing (SPONT)

The workstation must only be used under the supervision of qualified medical staff, so that help is available immediately if any faults or malfunctions occur.

Explosive anaesthetic agents, such as ether or cyclopropane, must not be used due to the risk of fire!

The equipment cannot distinguish between different anaesthetics. Dräger cannot accept any liability if the wrong anaesthetic is used!

Additional electric devices clipped into the top of the unit must be connected to the base unit via an equipotential bonding (earthing) conductor.

Do not use mobile phones within a distance of 10 metres from the machine.

Mobile phones can cause interference to electrical and electronic medical appliances, thereby putting patients at risk.*

Do not use Cato® for nuclear spin tomography.
The functioning of the apparatus may be impaired.

Since this equipment is not approved for use with inflammable anaesthetics (ether, cyclopropane, etc.), it is not necessary to use antistatic (conductive) breathing hoses or face masks.

Conductive breathing hoses and face masks may cause burns during high-frequency surgery and are therefore not recommended for this equipment.

The workstation should be moved **using the handles only**.

* Dräger medical appliances comply with the interference immunity requirements of the specific standards for the products or EN 60601-1-2 (IEC 601-1-2). However, depending on the design of the mobile phone and situation of use, field strengths may occur in the immediate environment of a mobile phone that exceed the values of the standards quoted and therefore cause interference.

Measurement and monitoring functions:

- Measurement of the ventilation parameters: pressure, flow, O₂ concentration (inspiratory and expiratory)
- Continuous measurement of the CO₂ concentration and N₂O/anaesthetic concentration (halothane, enflurane, isoflurane, sevoflurane, desflurane). The flow rate for sampling the measuring gas can be varied and is returned to the circulation.
- Automatic adjustment of the alarm limits for automatic ventilation IPPV.
- Anaesthetic vaporizer with automatic Vapor recognition¹⁾.

Optional:

- Continuous non-invasive measurement of the functional O₂ saturation.
- Measurement of the inspiratory breathing gas temperature.
- Expiratory O₂ value.

The following values are indicated:

- Continuous curve for airway pressure, peak and plateau pressure, mean pressure and PEEP.
- Patient compliance.
- Expiratory minute volume, tidal volume and respiration rate.
- Expiratory flow curve.
- Inspiratory and expiratory O₂ concentration.
- Inspiratory and expiratory concentration of N₂O and anaesthetic halothane, enflurane, isoflurane, sevoflurane and desflurane.
- Inspiratory and end-expiratory CO₂ concentration (inCO₂ and etCO₂).
- Continuous CO₂ curve.
- List entries and trend displays.

Optional:

- Functional O₂ saturation, pulse rate, plethysmogram.
- Inspiratory breathing gas temperature.
- Expiratory O₂ value.

1) Refer to the separate Instructions for Use for the Vapor!

The following parameters are monitored:

- Airway pressure.
- Expiratory minute volume.
- Inspiratory O₂ concentration.
- Inspiratory and expiratory CO₂ concentration.
- Inspiratory anaesthetic concentration.

Optional:

- Functional O₂ saturation and pulse.
- Inspiratory breathing gas temperature with invariable upper alarm limit.

Accessories

– **Patient monitoring**

Optional monitor PM 8060 vitara¹⁾.

Parameter box for patient monitoring and measurement of the haemodynamic patient values.

- ECG/arrhythmia analysis.
- Pulse rate.
- Respiration.
- Invasive (2 channels) and non-invasive blood pressure.
- Functional O₂ saturation and pulse.
- Body temperature (2 channels).

– **Aquapor¹⁾**

for humidifying and heating the breathing gas.

– **Secretion aspirator¹⁾**

– **Vapor¹⁾**

Anaesthetic vaporiser for halothane, enflurane, isoflurane and sevoflurane.

– **Connection for two "Vapor" anaesthetic vaporisers¹⁾**

– **Devapor¹⁾**

Anaesthetic vaporiser for desflurane.


– **Anaesthetic gas scavenging system¹⁾**

– **Uninterruptible power supply¹⁾**

1) Refer to the separate Instructions for Use for this equipment!

Quick start in an emergency

- Plug the gas connectors into the gas supply wall sockets.
- 1 The O₂, AIR and N₂O pressure gauges must be in the green range.
- Plug the power plug into the mains.
- 2 Press the master switch.
All the LEDs on the ventilator light up.
- 3 Press the key for the desired operating mode on the ventilator –
Recommendation: hold down  for longer than 1 second.
- 4 The following message is displayed in the display window of the ventilator:
Test 3 x discont (example)
Number of times that the self-test has been interrupted for a quick start since it was last completed successfully.
- Set the selector switch to »N₂O« or »AIR«.
- 5 Deliver fresh gas.
- 6 Press O₂ flush – if necessary –
to fill the system and the breathing bag rapidly.
- 7 Switch the pressure limiting valve (APL) to »MAN«.
- 8 Set maximum pressure. Turn the lever of the pressure limiting valve clockwise until the indicator (plate) stands at maximum pressure.

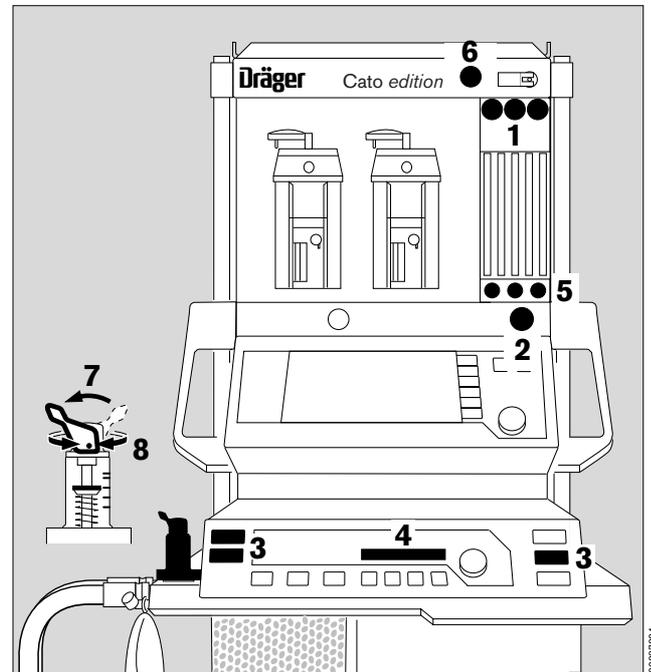

No more than ten consecutive quick-starts are allowed between two complete self-tests. This maximum is only permitted if the previous completed self-test revealed no fault.

After the 10th consecutive quick-start, the following message appears in the ventilator display window:

last cancel

If a further attempt is made to cancel the self-test and carry out a quick start, the following message is displayed:

COMPLETE TEST

Quick-start not permitted.

A complete self-test must be carried out before start-up is possible.

However, in all cases, manual ventilation is always possible.

Quick-start can be started at any time, even while a self-test is in progress.

Quick start in an emergency

Power failure

Gas failure

Auto-WakeUp

Power failure (manual ventilation is still possible)

- The master power switch must be pressed.
The audible power failure warning is muted after 45 seconds.
- Deliver fresh gas and set the pressure limiting valve (APL). If necessary press the O₂ Flush key (»O₂ +«).



If there is a power failure, the ventilator piston is forced back to its end position by the airway pressure, thereby increasing the system volume by max. 1.4 litre.

Gas failure

If AIR (medical compressed air) fails

- Cato automatically switches over to O₂.

If O₂ fails

- Cato automatically switches over to AIR. An audible warning is emitted (O₂ shortage warning). N₂O delivery is blocked:

If O₂ and air fail:



Ventilate the patient immediately with the separate emergency ventilation bag!

Auto-WakeUp

Manual ventilation cannot be performed in standby mode. Any attempt to perform manual ventilation in standby mode is detected by the system on account of the pressure thrust in the breathing bag and it automatically switches to »**MAN/SPONT**« mode in which manual ventilation can then be performed. The background lighting of the flow measuring tubes goes on at the same time.

The system is protected against excess static pressures:

The system pressure is relieved automatically if an excess static pressure of more than 30 mbar is present for more than 60 seconds.

Dräger recommends that manual ventilation should always be started by pressing the »**MAN/SPONT**« key.

Operating concept

Contents

	Page
Operating concept, general	10
Master switch for electricity supply	10
Operating concept of the ventilator	11
Keys with dedicated function	11
Display window without dialogue function	12
Display window with dialogue function	12
Operating concept of the monitor	13
Keys with dedicated function	13
Displays	13
Measured values with grey numerals	13
Screen structure	14
Screen saver	14
Rotary control.....	14
The various screen displays	16

Operating concept, general

Master switch for electricity supply

Master switch

- 1 Press to switch on

Dialogue:

Monitor and ventilator feature a dialogue with the user mediated by:

- keys,
- rotary controls,
- displays and
- beeps.

Basic conditions of the operating state are established by adjusting the delivery valves or pressure limitation (e.g. APL valve).

- **Keys:** For direct command input

- **Rotary control:**

For selection by ...

... turning

This causes a cursor frame (on the screen) to be moved or a variable numerical value (on the display window of the ventilator or on the screen) to be changed.

... pressing

The value selected with the rotary control is adopted as a valid parameter or a process is started or ended.

- **Displays:** For presenting all information on the screen and on the display windows of the ventilator.



- **Tone sequences:**

As an acoustic supplement to the messages. They are coupled with certain sounds or tone sequences, according to priority classes.

Tone sequences accompanying warnings are output continuously, every 30 seconds with caution messages and only once in conjunction with advisory messages. These are to draw the user's attention to the messages which appear simultaneously in the displays.

European Standard EN 740 stipulates use of EN tones.

Alternatively there are tones in keeping with Dräger conventions available.

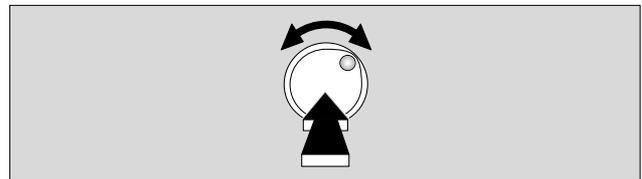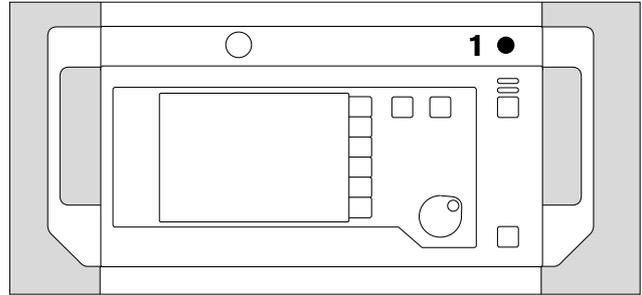

Operating concept of the ventilator

Keys with dedicated function – for setting the operating modes

Left-hand side:

**MAN
SPONT** Key for manual ventilation or
 spontaneous breathing.



IPPV Key for IPPV mode.

Right-hand side:

TEST Key for leakage test and compliance
 measurement.

SIMV Key for SIMV mode.

Standby key.

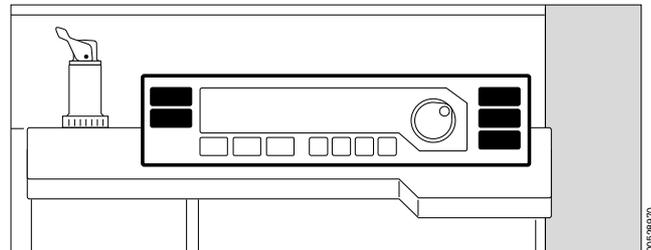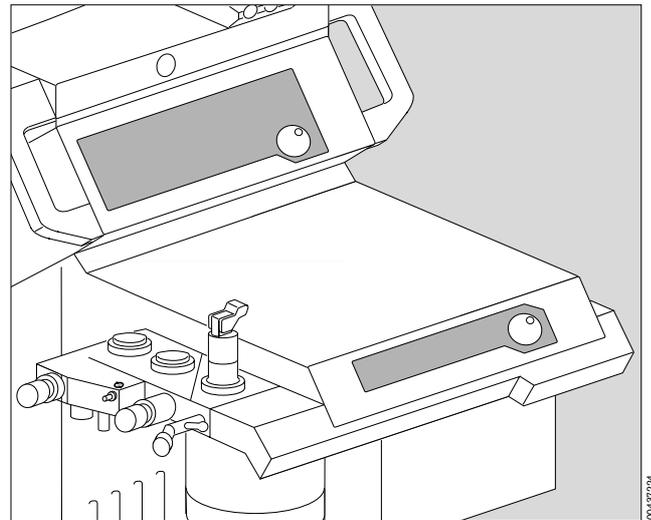

for setting ventilation parameters

Below the display window:

Pmax Key for setting the maximum pressure
 for IPPV and SIMV ventilation.

Vt Key for setting the tidal volume.


f_{IPPV} Key for setting the ventilation frequency
 in IPPV mode.

T_i : T_E Key for setting the time ratio between
 inspiration and expiration.

T_{IP} : T_i Key for setting the relative inspiratory
 pause.

PEEP Key for setting the PEEP pressure for
 IPPV mode.

f_{IMV} Key for setting the ventilation frequency
 in SIMV mode.

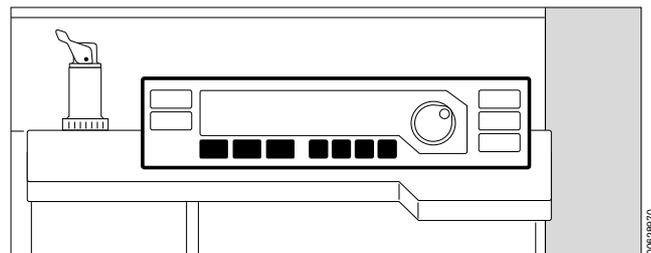



Display window without dialogue function

Top left:

Continuous indication of the relative piston movement (in % referred to the set stroke volume VT)

The set operating parameters correspond with the keys below:

- Indication of the maximum pressure P_{max} in mbar.
- Indication of the tidal volume VT in mL or L.
- Indication of the ventilation frequency fIPPV in breaths per minute.

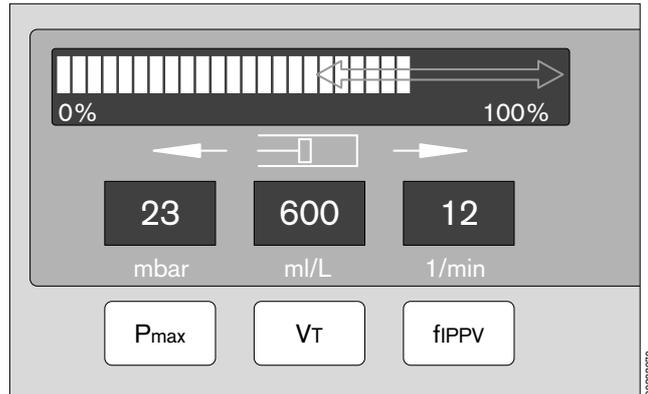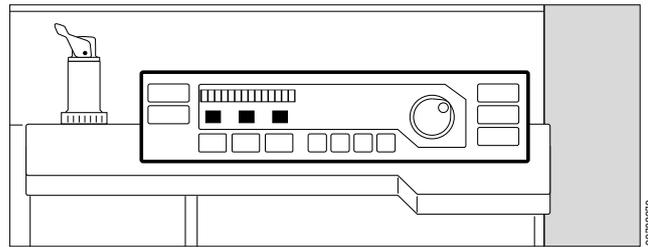

Display window with dialogue function (in combination with the rotary control)

Example: adjusting the maximum pressure




In the black field, beside the rotary control:

- 1 The set value appears on the right and left-hand sides of the field when a parameter key (P_{max} , VT, fIPPV) is pressed. Here: »23«.
- 2 The value on the right-hand side is changed by turning the rotary control. Here: »28«.
The old and new values are consequently always displayed together.
- 3 The value on the right («28») is confirmed as the definitive value by pressing the control.

If the rotary control is not pressed and not turned again, the machine is reset after 10 seconds without changing the setting.

- This dialogue window also displays advisory messages (see page 78) –

Example: »Paed. hoses !«:

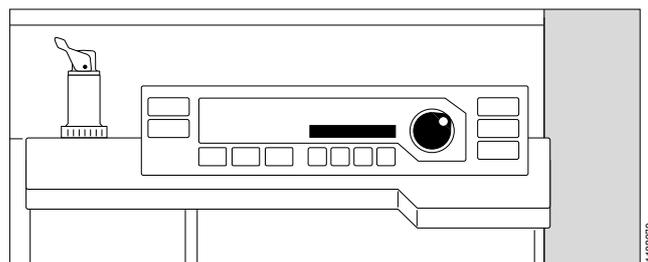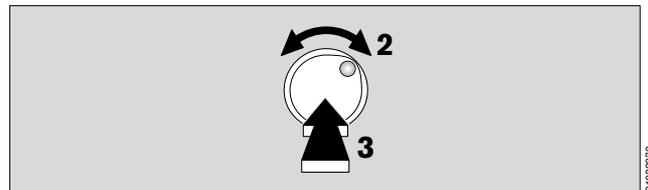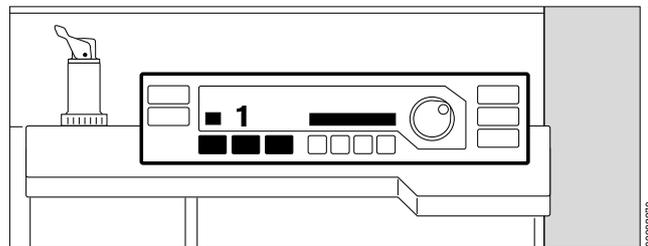

Operating concept of the monitor

Keys with dedicated function (Hardkeys)

The right-hand side is reserved for operating elements, the left-hand side for displays.



This key switches the monitor from standby to measuring mode and vice versa.

The monitor mode depends on the ventilator mode:

Standby can only be selected on the monitor if the ventilator is also in standby.
 The monitor starts up when the ventilator is started.



This key is used to deactivate the alarm tone for two minutes. It is reactivated by pressing the key again. The yellow LED in the key lights up while alarms are suppressed.


Inside the dark area, there are two keys acting directly on the screen contents:



This key is used to switch directly from one screen to the next in succession.



This key is always used to call up the »Standard screen« (see page 57).

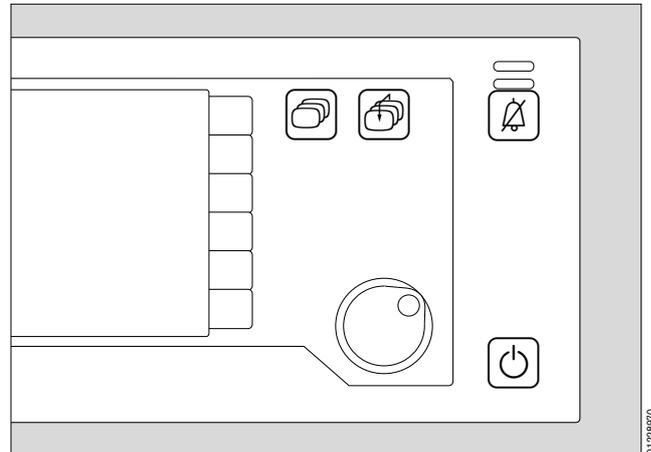

01228970


Displays



00537224

Two bar-shaped indicator lamps are located above the  key: these lamps continue to indicate the alarm states even when the acoustic alarm is switched off.

Red (upper) lamp, flashing:	Warning	!!!
Yellow (lower) lamp, flashing:	Caution	!!
Yellow lamp, constant:	Advisory	!

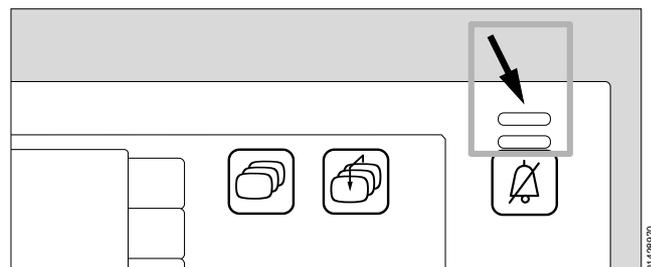

01428970


Measured values with grey numerals

Measured values generated by an uncalibrated sensor are shown in grey type. This may be due to the following causes:

- Self-test has been aborted.
- Automatic sensor calibration is in progress.
- The measuring equipment for the anaesthetic gas composition has not yet reached the required working temperature when the system is started.


Screen structure

- **Status field** top:
contains information on the current alarm mode of the monitor.
- **Alarm field** top:
indicates any alarms and their priority.
- **Graphic field** left:
for curves and bar graphs.
- **Measured value field** right:
for the most important numerical values.
- **Operator prompts** bottom right:
prompts to guide the operator.
- **Softkeys** right:
for rapid selection of the functions displayed on the screen.

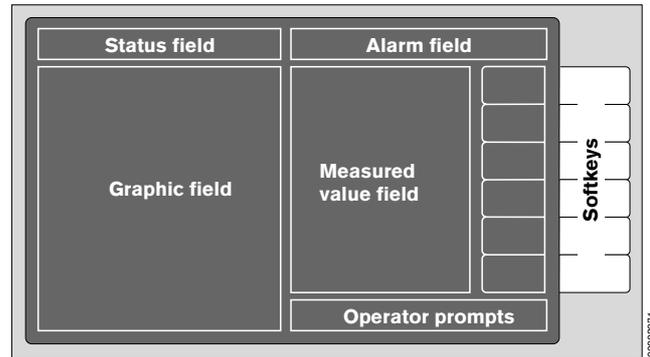

Screen saver

If none of the operating elements on the monitor is operated in »standby« for approx. 2 minutes, the screen switches off and becomes dark. The yellow LED in the standby key and the word »standby« on the ventilator light up. The monitor display is immediately restored as soon as any key is pressed.

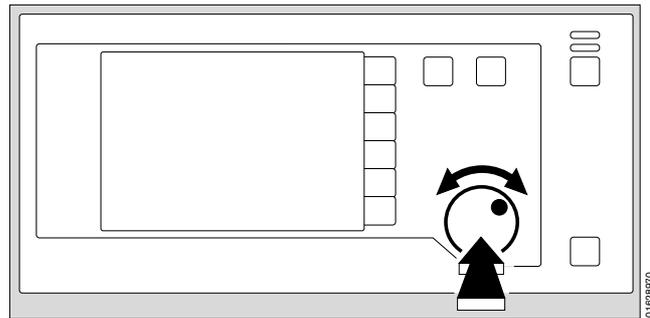

Rotary control

Selection and adjustment with a single control.

For example:



Adjusting the volume of the pulse tone

after calling up the menu of default values via the softkey »config.« in standby (after entering a code).


- Turn rotary control = selection.
The cursor frame moves horizontally in the dashed area.



- Press rotary control = confirm selection.
The selection is confirmed and appears in dark type on a light background. The cursor frame is positioned over the arrow symbol (→) to the next higher menu.
- Turn rotary control = the cursor frame moves vertically inside the dashed area.
Select pulse tone.

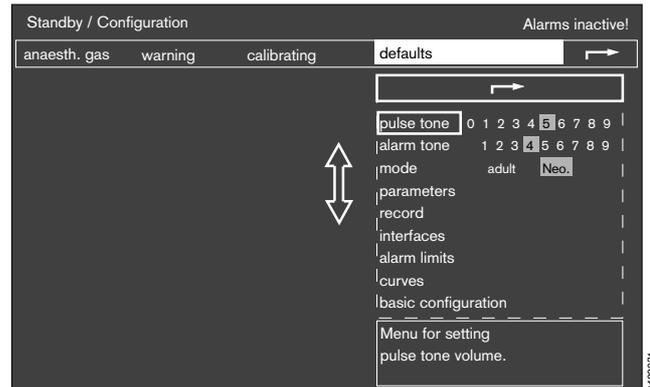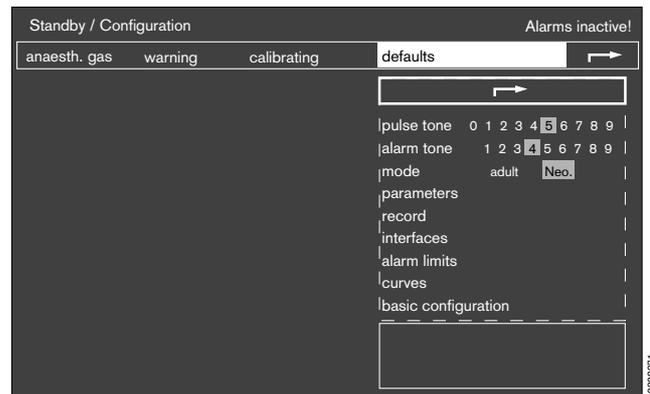


- Press rotary control = confirm selection.
 The selection is confirmed and appears in dark type on a light background.
- Turn rotary control = select new setting.
- Press rotary control = confirm selection.
 The cursor frame is now on the arrow symbol (➡) to the next higher menu.

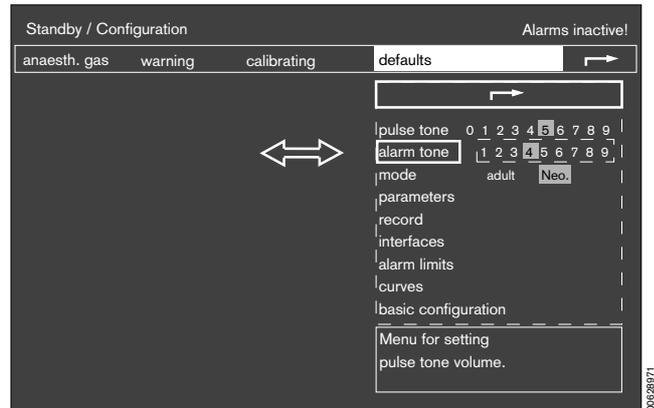


- Turn rotary control = select new setting.
 Or:
- Press rotary control again = close submenu »defaults«.
 The cursor frame is on the arrow symbol (➡) to the next higher menu.

The current configuration is indicated by the fields with grey background.

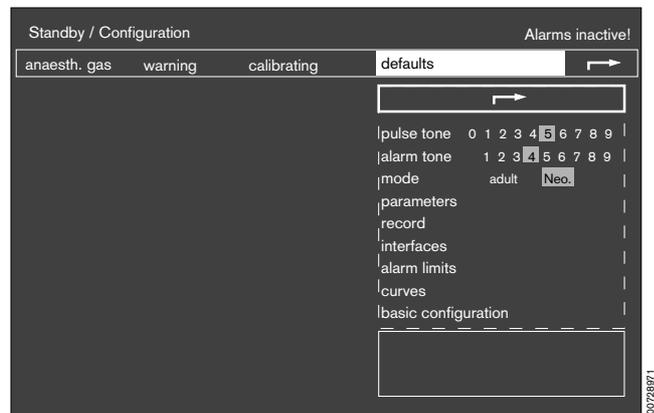


- Press rotary control again = exit menu.

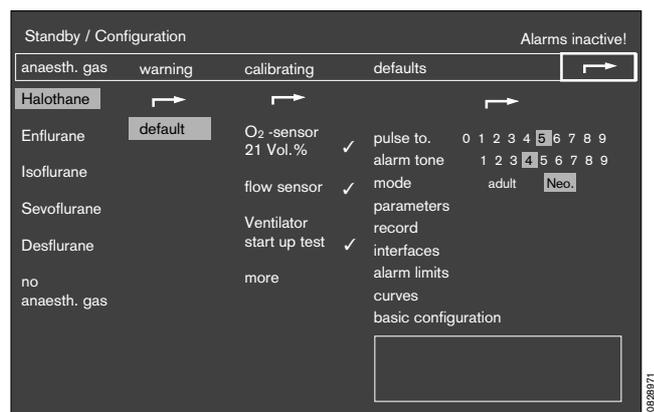


The various screen displays

- 1 The three different screen displays are invoked by pressing .
- 2 Press  to return to the standard screen from any screen display.

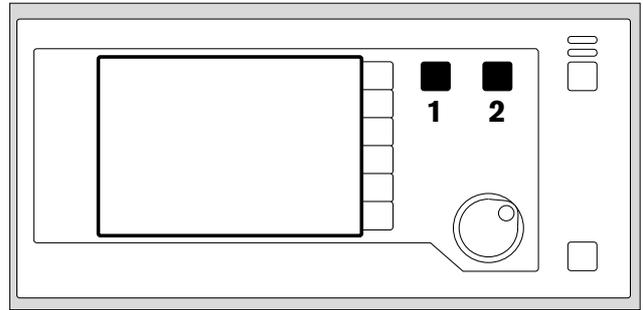


The standard screen

with the **CO₂ curve** and **another** selectable curve. The most important measured values are grouped together on the right.

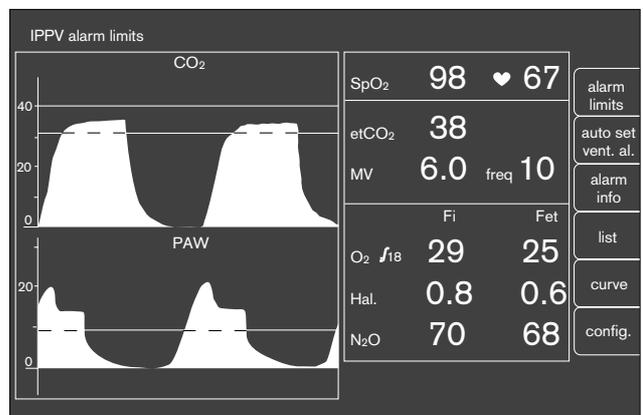


The data screen

contains **all** measured values with their units of measure; simplifies the completion of the anaesthesia record.

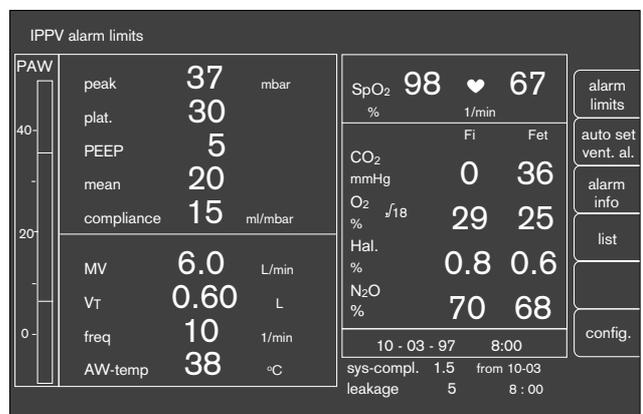

The trend screen

for displaying the **changes in measured values** since measurement started. The **current measured values** are shown on the right.

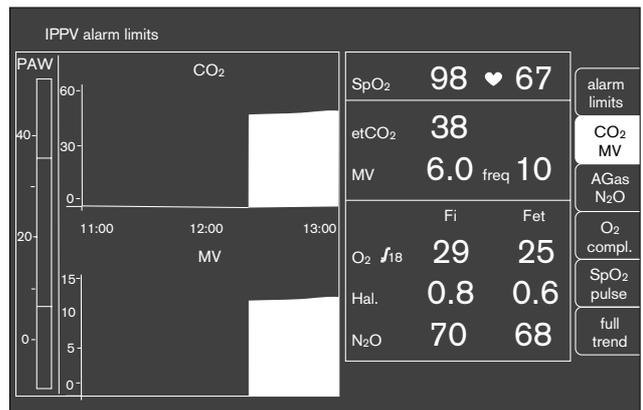

Preparing for use

Contents

	Page
Connecting the equipment	18
Electricity and gas supply	18
Auxiliary electrical equipment	18
Anaesthetic gas scavenging system (AGS)	19
Anaesthetic agent vaporiser	19
Uninterruptible power supply	20
External equipment	20
Checking readiness for operation with checklist	21
Vapor	22
Anaesthetic gas scavenging	22
Breathing system	23
Soda lime	23
Emergency ventilation bag	23
Water traps	24
Reserve gas cylinders (optional)	24
Pipeline gas supply	24
Gas delivery	24
Oxygen Ratio Control (ORC)	25
O ₂ Flush	25
Secretion aspirator (optional)	25
Power supply	26
Self-test	26
Fresh gas - External outlet (optional)	27
Selecting anaesthetic agent	28
Automatic calibration of O ₂ /flow sensors	29
Manual calibration of the O ₂ sensor	29
Manual calibration of the flow sensor	30
Ventilator start-up test	31

Connecting the equipment

The equipment must have been stripped down and tested beforehand!

Electricity and gas supply

- 1 Plug the power cable into the mains socket.
- 2 Connect a ground lead for equipotential bonding to one of the four pins on the rear of the workstation when it is used for intracranial or intracardiac surgery.

The other end must be connected to the specified point in the operating theatre.

- 3 Screw hoses for O₂, AIR and N₂O into the rear of the equipment and plug connectors into the wall sockets.
- Check that the supply pressure is adequate on the pressure gauges on the front (pointers must be in the green area).
- 4 Holder for anaesthetic gas scavenging.

Auxiliary electrical equipment

- 5 Connect to the three auxiliary power sockets (max. current per socket: 2 A). The auxiliary power sockets are not controlled by the main switch.

If there is a power failure, the auxiliary power sockets will be de-energised, because they are not powered by the uninterruptible power supply (UPS).

Do not connect high-frequency surgery appliances to the auxiliary power sockets.


Do not connect any other multiple sockets, e.g. multiple socket adapters, to the auxiliary sockets.

The connection of equipment to the auxiliary sockets causes an increase in leakage current.

The total leakage current in the power line must not exceed 500 µA (EN 60601-1).


- 6 Sub-D socket for connecting the uninterruptible power supply (UPS), see page 20. On connection to the Sub-D socket, the UPS can be switched off using the main switch.

The auxiliary power sockets and UPS are not installed in the Cato ceiling version.

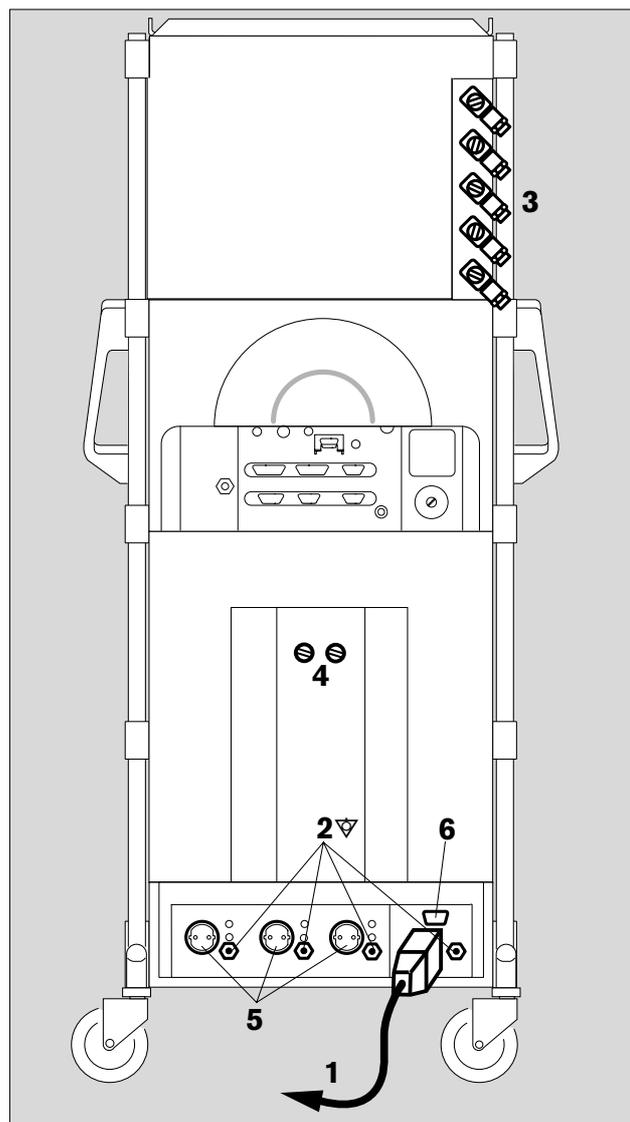


Anaesthetic gas scavenging system (AGS)

- Connect the transfer hose to the scavenging adapter – first time only, it then remains in place.
- 1 Insert the scavenging adapter in the breathing system from below until it engages.
- 2 Route the transfer hose round the equipment to the rear.

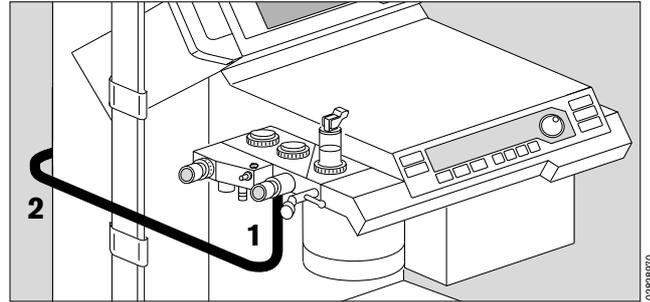


- 3 Plug the transfer hose to the port on the collecting system.
- 4 Connect the suction hose (sampling hose) to the port on the collecting system.
- 5 Connect the anaesthetic scavenging connector to the sampling hose.
- 6 Make sure that the second connector on the collecting system is sealed with the screw plug.
- Follow the specific Instructions for Use of the anaesthetic gas scavenging system.

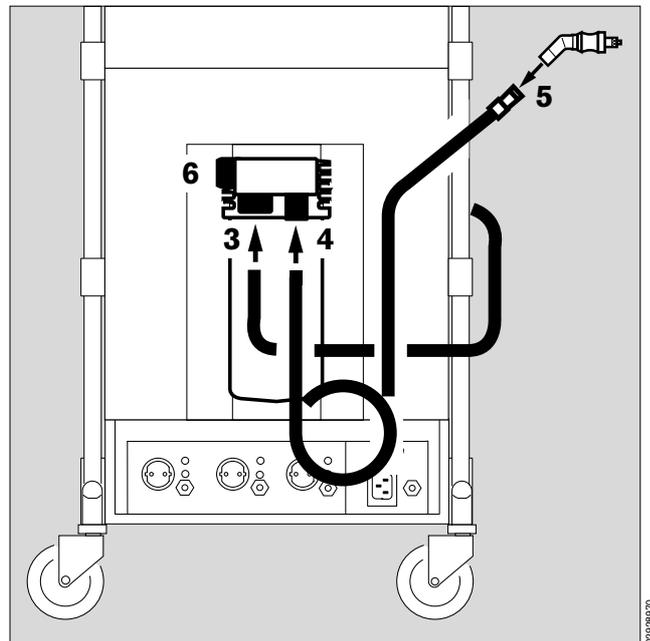

Anaesthetic agent vaporiser

The illustration shows a Vapor 2000

- If using the double connector for two Vapors, the automatic Vapor identification system is disabled.
- Only use Vapors listed in the Order List.
- Follow the specific Instructions for Use of the Vapor.
- For each anaesthetic agent, only use the specified Vapor.
- 7 Always insert the sealing plug and
- 8 secure with lever.
- 9 Engage control dial at »0« when no fresh gas is set.
- 10 Always secure the Vapor with the locking lever (turn lever to the left as far as it will go).

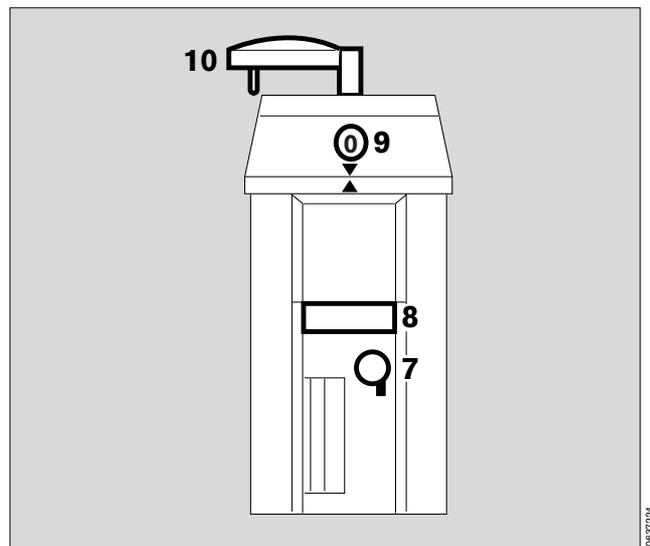

Uninterruptible power supply


(optional, see separate Instructions for Use)

Make sure that the status line is connected, so that the uninterruptible power supply (UPS) is controlled by Cato.

In the event of power failure, the machine is powered by the battery of the UPS, but the auxiliary mains socket on the back of the Cato will be de-energised. However, auxiliary equipment powered by the refrigerator connector on the side of the Cato will still be powered.

- 1 Plug the power plug of the Cato into the socket of the UPS.
- 2 Plug the power plug of the UPS into the mains socket.

The UPS can supply the Cato and connected auxiliaries with electrical energy for about 45 minutes. It is automatically activated in the event of a power failure.

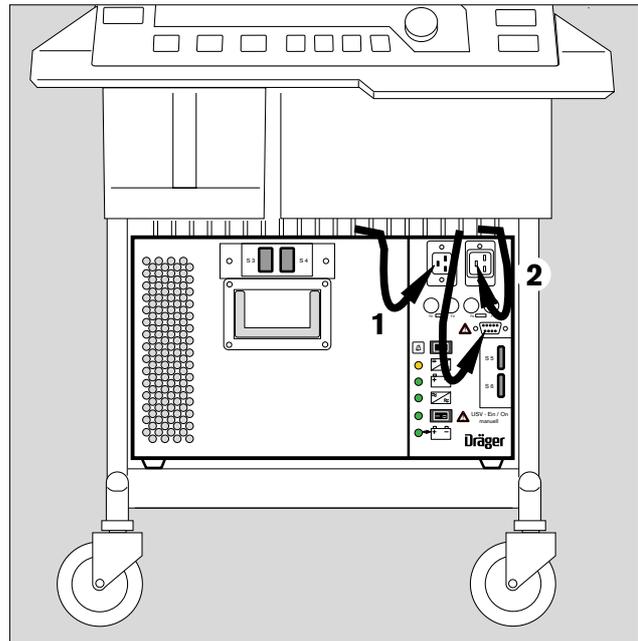

External equipment

See page 53 for the configuration of the interfaces.

Connection via the protocol interface

- 3 with data cable for printers with **serial interface**, e.g. Desk-Jet (Hewlett-Packard)

or


e.g. the PM 8060 Vitera Patient Monitor with the MEDIBUS protocol.

See page 53 for configuration.

Connection via the Dräger RS 232 C MEDIBUS interface

e.g. to connect the PM 8060 Vitera Patient Monitor.

- 4 Connect with data cable.
- The equipment plugs must be secured with the screws provided.

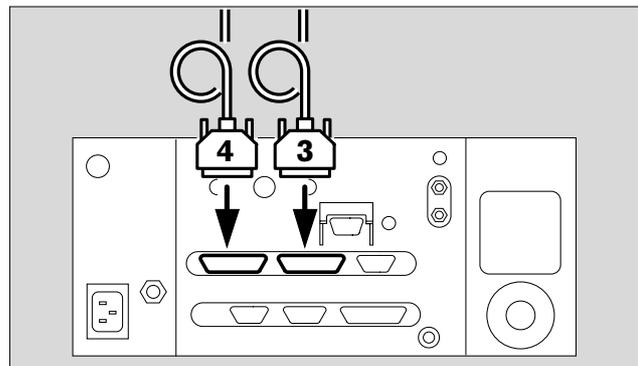

Checking readiness for operation with checklist

It is assumed that the following conditions have been met:

- The equipment and its accessory parts have been cleaned and disinfected – refer to page..... 83 onwards
- The unit is fully equipped for the application in question – refer to page..... 89 onwards
- The user has a good knowledge of the Instructions for Use and has been trained to use the equipment.
- An emergency ventilation bag with appropriate mask is available on the equipment.
- A checklist is affixed to the equipment.

The equipment must always be checked against the checklist before it is used!

This check is mandatory as specified by EN 740!

Duration: approx. 5 minutes (depending on the scope of calibration).

The equipment is subsequently in »standby« mode when ready for use. Anaesthesia ventilation can immediately be started at any time in this mode – refer to page..... 36 onwards

- Update the checklist by adding or deleting points in accordance with the Instructions for Use, the equipment type and configuration concerned and the various supplementary units connected. It will then contain all the requisite checks.
- Enter the model designation and serial number of the equipment.
- Tick off ✓ the results of the checks in the ACTUAL column.
- Remember to sign and date the checklist!

Test sequence

- Note any alterations and additions!
- Note the Instructions for Use of the individual units!
- If the checks do not proceed as planned, restore the required status.

Preparing for use
Checking readiness for operation with checklist
Vapor
Anaesthetic gas scavenging

Vapor

The illustration shows a Vapor 2000

- 1 The control dial is engaged at »0«.
 - 2 Filling level OK – check filling level in viewglass.
- Last inspection less than six months ago.

Safety fill:


- 3 The sealing plug is inserted and secured with the lever.

Plug adapter:

- The plug adapter lies horizontally and flush all round on the sealing rings of the plug connection.


Interlock:

- 4 Plug-in system is locked – locking lever turned clockwise until it engages.

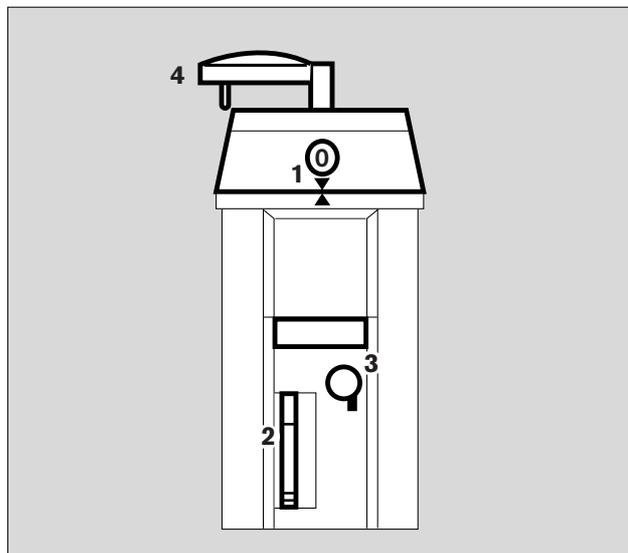

Anaesthetic gas scavenging


- Is indicator in wall socket green? (Only when using Dräger systems; note sounds of gas flow in other cases.)
 - Hose connector engaged below breathing system?
 - The auxiliary air holes in the tube below the connector must not be sealed, otherwise the breathing system will be drained!
- Note the separate Instructions for Use for the anaesthetic gas scavenging system.

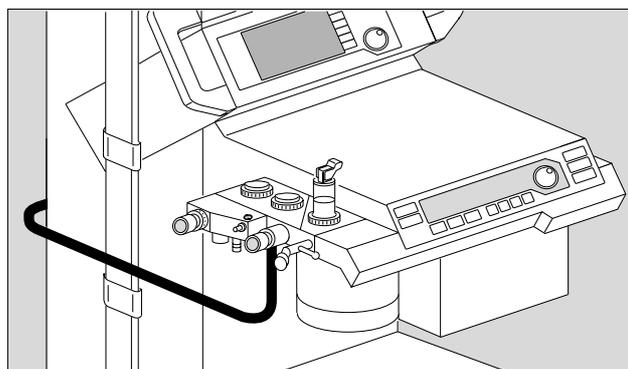

Breathing system

Lift off table top:

- 1 Lever set to the position shown in black.

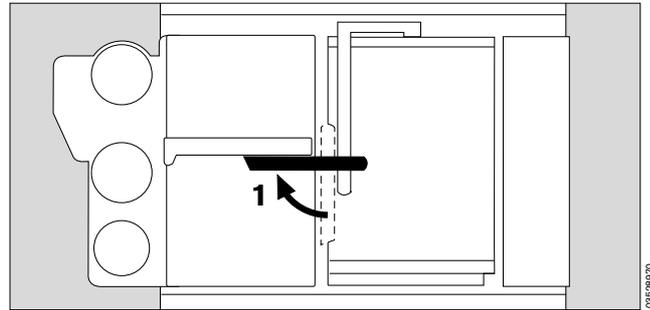

The following are complete:

- 2 Hose with manual ventilation bag (Connected from below – not shown) Symbol:

- Correct breathing hoses installed. (Adult or infant hoses)



- 3 Pressure measuring hose with filter connected.

- 4 Measured gas return hose connected. Symbol:

- 5 Fresh gas hose plugged in. (Connected from below – not shown)

- 6 Valve discs inserted.

- Pressure limiting valve (APL) present. Symbol:
- Inspiratory microbial filter. Symbol:
- Expiratory microbial filter. Symbol:

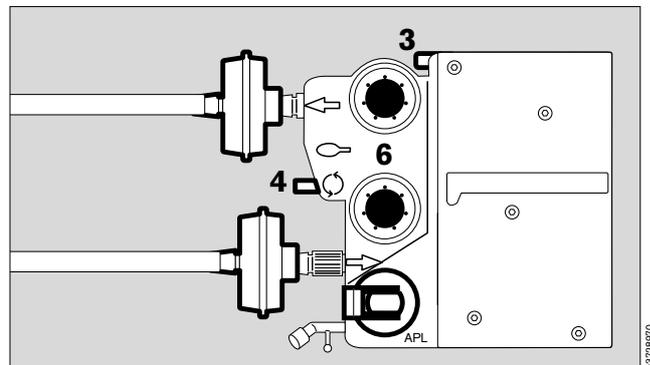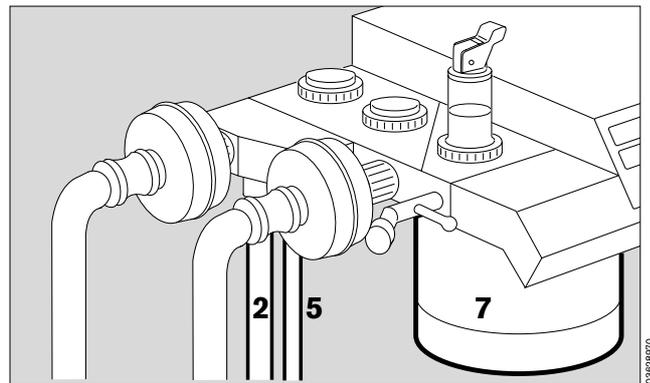

Soda lime

- 7 Lime has not noticeably changed colour (purple).
 - Filling level adequate (up to the mark).
 - Soda lime container is securely tightened – **up to the end stop** (clockwise).

Emergency ventilation bag

(not shown)

- Bag is complete with mask and hung from the side of the Cato.
- Bag functions correctly.

✓ Check and tick off.

Preparing for use
 Checking readiness for operation with checklist
 Water traps, Reserve gas cylinders
 Pipeline gas supply, Gas delivery

Water traps


- 1 Water traps are recommended in both the inspiratory and the expiratory lines during prolonged anaesthesia, low-flow anaesthesia and when using humidified breathing gas.
- Water traps must be fitted at the lowest point in the hose and hang downwards.
 - Check regularly and drain if necessary.



Observe hygiene regulations – risk of infection!

The hose system remains sealed. The container must be replaced securely!

»Tips on reducing condensation« on page 136.

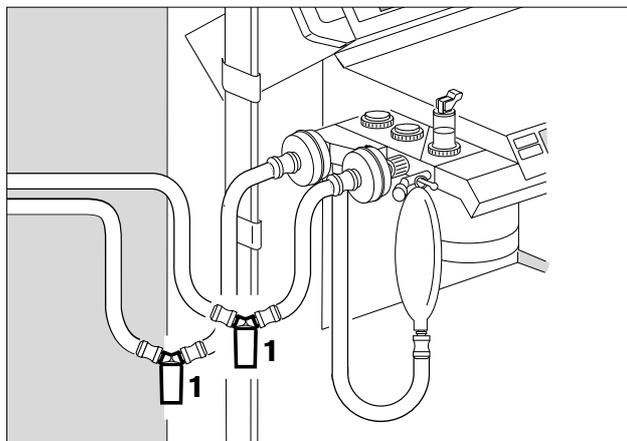

Reserve gas cylinders (optional)


- Open cylinder valves.
- ✓ Check and tick off:
 - Pressure indicator on O₂ cylinder exceeds 50 bar? –
 - Pressure indicator on N₂O cylinder exceeds 30 bar? –
 Replace cylinders if not.
- Close cylinder valves!

Pipeline gas supply

- Have connectors been pressed right into the wall sockets for O₂, AIR and N₂O (not in holding position!)?
- ✓ Check and tick off:
 - 2 Pointers of all three pressure indicators are in the green range.

Gas delivery

- 3 Switch over to AIR.
- 4 Open O₂ and AIR delivery valves until more than 9 L/min are indicated!
- 5 Open N₂O delivery valve completely.
- ✓ Check and tick off:
 - Does N₂O measuring tube indicate 0?
- 3 Switch over to N₂O.
- ✓ Check and tick off:
 - Does N₂O measuring tube indicate more than 9 L/min?
 - Does AIR measuring tube indicate 0?

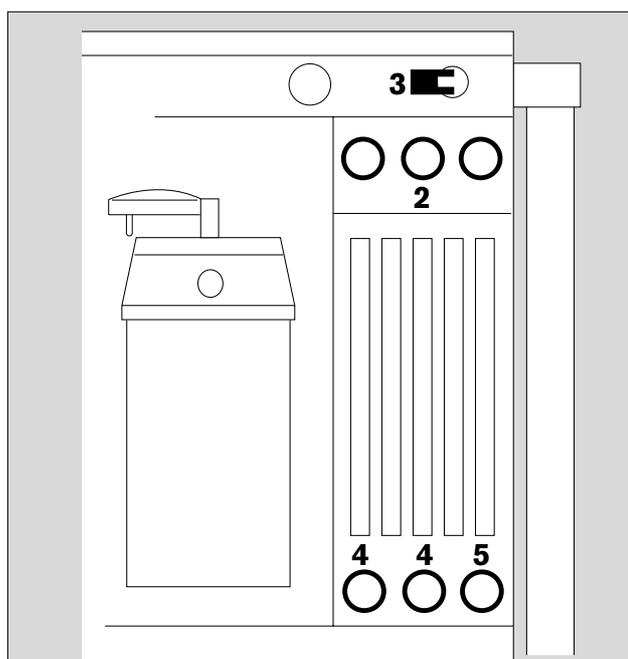


Oxygen Ratio Control (ORC)

- 1 Slowly close O₂ delivery valve –
 - check:
 - the N₂O flow decreases to less than 0.8 L/min proportionally with the O₂ flow for ORC low-flow, the N₂O flow decreases to »O« proportionally with the O₂ flow for S-ORC.
- 2 Switch over to »AIR«.
 N₂O flow decreases to »O«.
- Close N₂O and AIR delivery valve.

O₂ flush

- 3 Press button »O₂+« –
 - Is there a distinctly audible flow noise?
 - Does the manual ventilation bag inflate?
- ✓ Check and tick off!

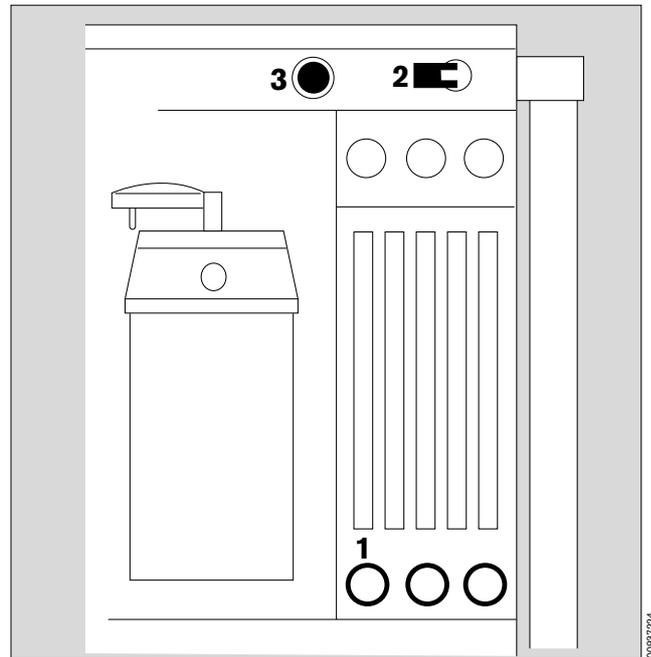


Secretion aspirator (optional)

- 4 Open ejector valve –
 - 5 Seal the **aspiration holes** on aspirator hose with your finger (or fold over the hose).
- Negative pressure indicated approx. –0.8 bar?
- ✓ Check and tick off!
 - Close ejector valve.



Secretion aspirator may only be used in »MAN/SPONT« mode or with disconnected Y-piece.

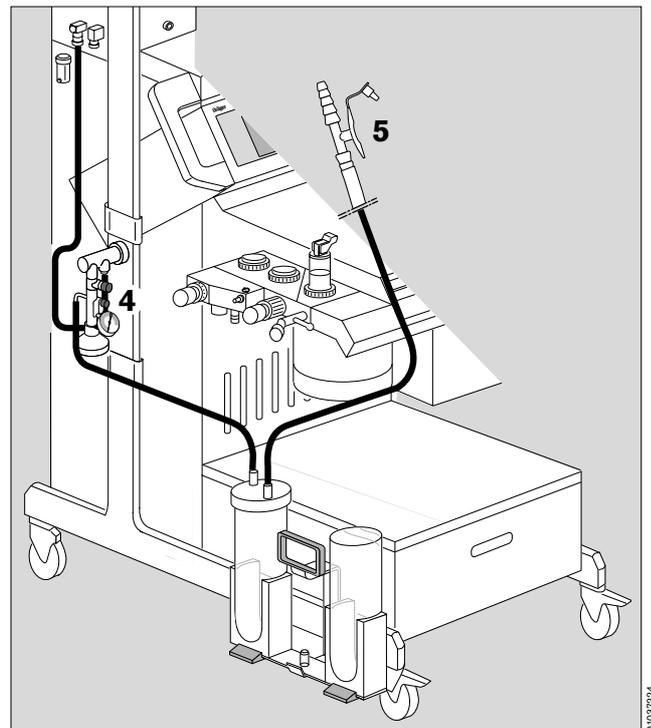


Preparing for use
 Checking readiness for operation with checklist
 Power supply
 Self-test

Power supply

- Equipotential bonding conductor connected?
- Mains plug connected?

- 1 Switch on equipment –
Press main switch to I.

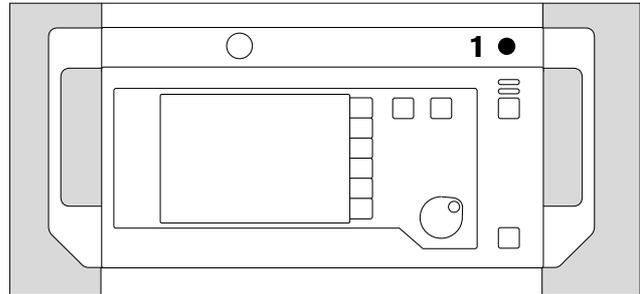

Self-test

✓ Check the signals and tick off the following on the checklist:

- Lamp and alarm test, in succession:

Ventilator:

- Indication of the software version, all indicators light up for approx. 2 seconds while a single tone sounds.

Monitor:

- The machine runs through the self-test:

All LEDs and display elements light up for approx. 2 seconds. The LED in the Standby key  continues to light up.

- Two alarm tones sound.
- The internal program memories are tested.
- The self-test is completed after approx. 1 minute.

The following display appears on the monitor:



»Self-test«


is displayed on the ventilator.

If the self-test reveals a **fault of no relevance to the safety of the equipment** and not affecting any of the measuring functions, the following message appears on the monitor:

»Ready within limits«

together with a specific error message (refer to pages 74 onwards). However, the equipment can be operated nevertheless.

- Press the rotary control.
Call DrägerService.

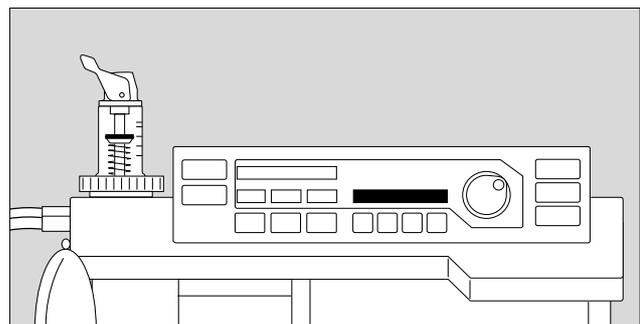


If the self-test reveals a **fault impairing the equipment's safety**, the following message appears on the monitor:

»Not ready«.

The machine cannot switch to Standby and cannot be switched to measuring mode. DrägerService must be called immediately.


- The configuration menu is displayed after the self-test (example):

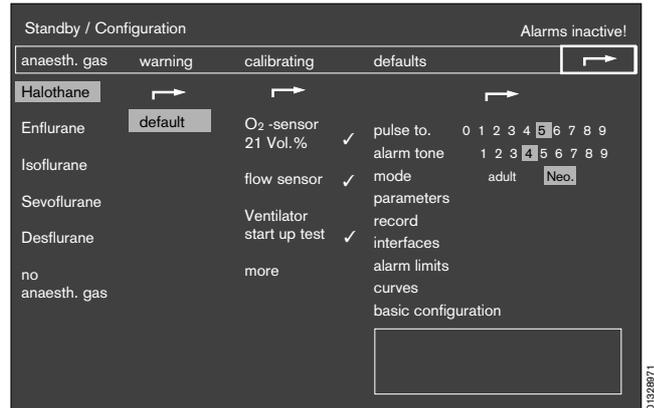

Fresh gas - External outlet (optional)

Fresh gas outlet for connecting to semi-open systems.

- 1 Fit the hose of the semi-open system to the external fresh gas outlet.
- Make sure the anaesthetic gas scavenging hose is connected.
- 2 Set the desired O₂ and N₂O flow-rates on the flowmeter block.
 - 3 Press »MAN SPONT« on the ventilator.
 - 4 The lamp in the »FRESH GAS OUTLET« key does not light up.
- Check that pressure is building up in the circle system.
- 4 Switch on the fresh gas outlet – Press »FRESH GAS OUTLET« key. The lamp in the key lights up.
- Check that pressure builds up in the semi-open system.

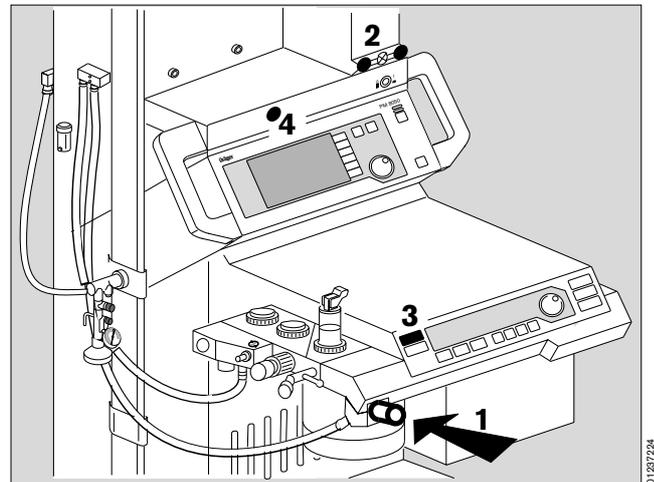

Preparing for use
Checking readiness for operation with checklist
Selecting anaesthetic agent

Selecting anaesthetic agent

The machine automatically identifies the newer Vapor models with code for

halothane, enflurane and isoflurane,
sevoflurane and desflurane.

Sevoflurane and desflurane are only identified by equipment built after July 1994 or which has been upgraded.

Uncoded Vapors must be selected via the configuration menu on the monitor.

The setting **»no anaesth. gas«** must be selected in the configuration menu for ventilation without volatile anaesthetic. This setting is activated automatically if a Vapor is not connected.

The plug-in system of the new Vapor models features an optical code allowing the machine to identify the Vapor:

- when the equipment is started up
- when the Vapor is changed during operation.

Incorrect anaesthetics are not detected!

However, the user can quite deliberately select a different Vapor instead of that identified automatically.

The following advisory message is then displayed in the alarm field on the monitor:

»AGT NOT SEL !«

If the Vapor has not been encoded:

The message

»Vapor not identified«

is displayed in the user advisory field of the configuration menu in **»standby«** mode.

- Set the appropriate anaesthetic for the Vapor used.

If a Vapor has not been connected:

The message

»Vapor not present«

is displayed in the user advisory field of the configuration menu in **»standby«** mode.

The following symbols are used:

- ? = Enquires whether an action has been performed or a setting made.
- ⊖ = Waiting period. The selected test step is being performed by the machine.
- ✓ = The action has been completed successfully or is not required.

Automatic calibration of the O₂/flow sensors

Sidestream calibration of the O₂ measurement (O₂ sampling) and flow measurement are performed automatically.

Automatic flow calibration is performed during the first breaths after starting ventilation. There is therefore no cause for panic if the measured values for the minute ventilation still appear in grey (i.e. uncalibrated values, see page 13) immediately after the self-test.

CO₂ measurement must function correctly before automatic flow calibration can be performed, otherwise the flow sensor must be calibrated by hand.

Manual calibration of the O₂ sensor




The O₂ sensor must be calibrated by hand if O₂ measurement has been set in the inspiratory line (see "Parameters" on page 53).

Calibrate O₂ sensor with 21% O₂ by volume = air

- The O₂ sensor can be calibrated while flow calibration is still in progress.

- 1 Remove the sensor from the inspiratory valve and expose it to the ambient air – place it on the table and wait at least two minutes.
- 2 Use the rotary control to select »**calibrating**« in the »**Standby/Configuration**« menu and press to confirm.
 - Move the cursor frame to »**O₂ sensor**« 21Vol.%« by means of the rotary control.
 - Press rotary control to confirm: calibration starts.

- 1 Then plug O₂ sensor onto the inspiratory valve again.
- ✓ Tick off in checklist.

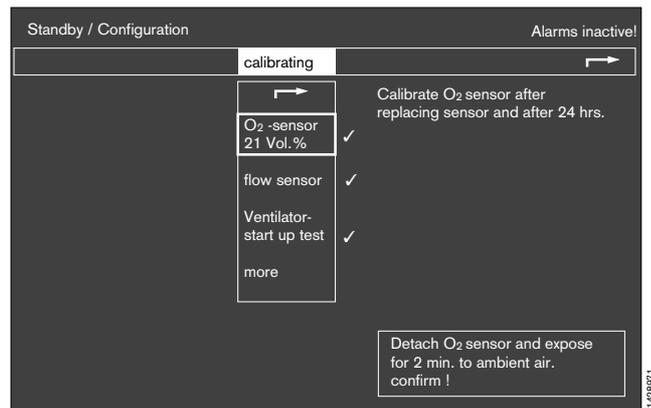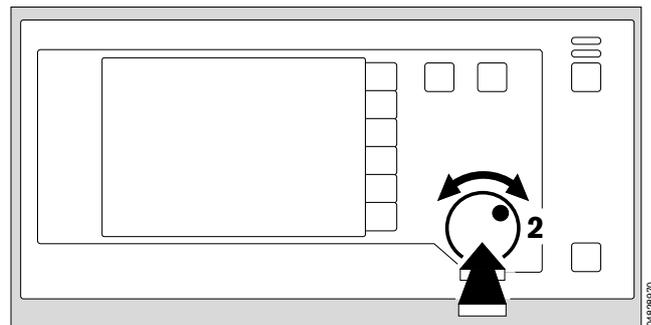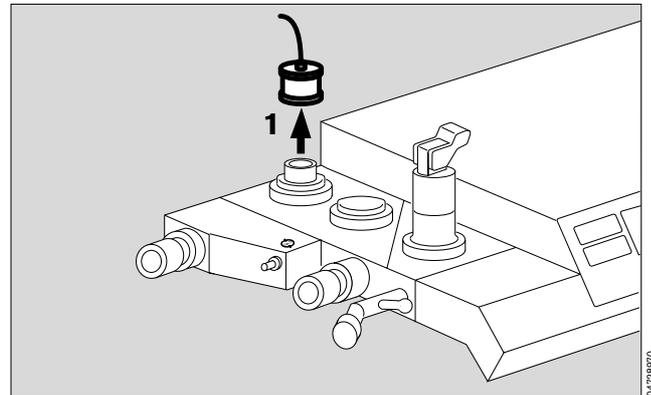

Manual calibration of the flow sensor

- The flow sensor can be calibrated by hand while O₂ calibration is still in progress, either in the machine or after being removed from the machine.

Without removal:

On the anaesthetic unit:


- 1 Set AIR/N₂O selector to »AIR«.
 - 2 Close the delivery valves for O₂ and N₂O, open the delivery valve for air and thoroughly flush the breathing system with air.
 - 3 Close the delivery valve for air.
- Press the rotary control on the monitor to start calibration.

The question mark (?) disappears and is replaced by a timer icon (⊖) on the screen.

A tick (✓) appears on the screen when calibration has been completed correctly.



- ✓ Tick off in checklist.

The cursor frame has automatically jumped to »Ventilator start-up test«.

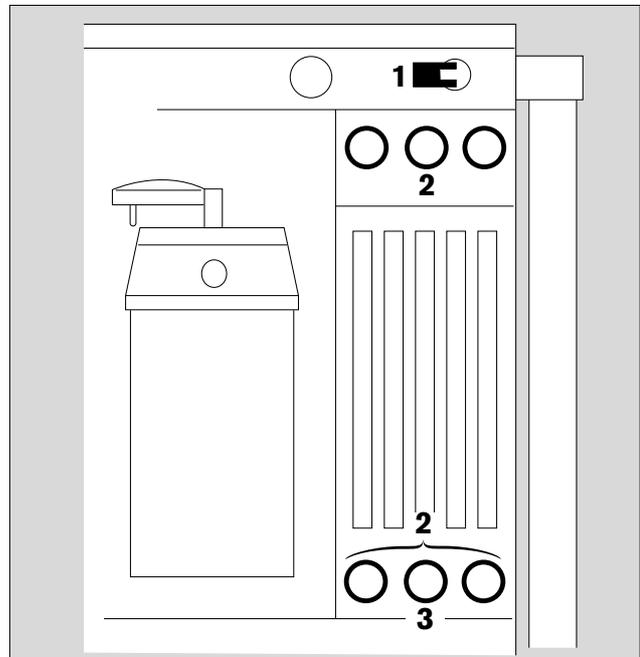

With removal:

- 4 Remove flow sensor:
 - Unscrew expiration nozzle
 - Remove flow sensor
 - Briefly swing it to flush with ambient air
 - Hold it horizontally with the cable connection pointing downwards (calibration in installation position improves the measuring accuracy)
 - Seal off one or both sides as shown on the right, preferably with your thumb or palm.
 - Press the rotary control on the monitor to start calibration; continue as above.
- 4 Replace flow sensor!

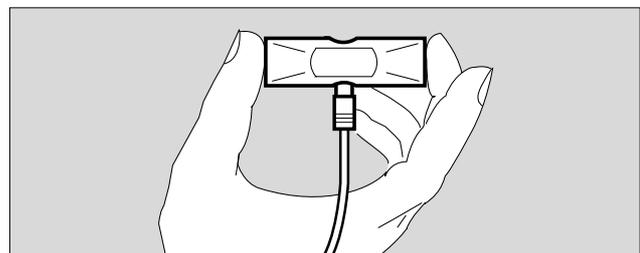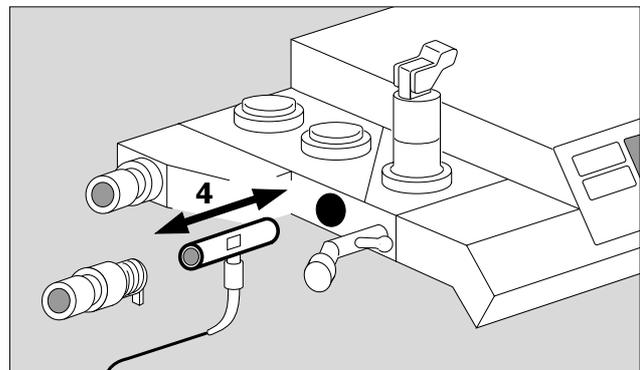

Ventilator start-up test



The ventilator must not be connected to the patient during this test!

Quantitative test to establish the integrity of the anaesthetic system.

The test cannot be performed until O₂ and flow calibration is complete and the relevant sensors have been installed in the anaesthetic system.

- 1 Use the rotary control to select «Ventilator start-up test» under »Calibrating« in the »Standby/Configuration« menu and press to confirm. (If this has not already been done automatically after O₂ and flow calibration).

- Screen display:

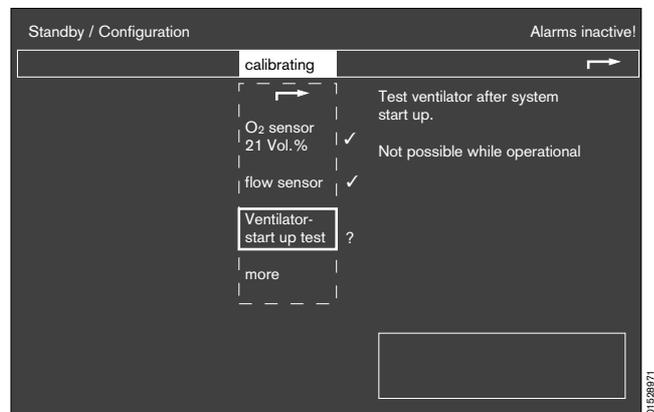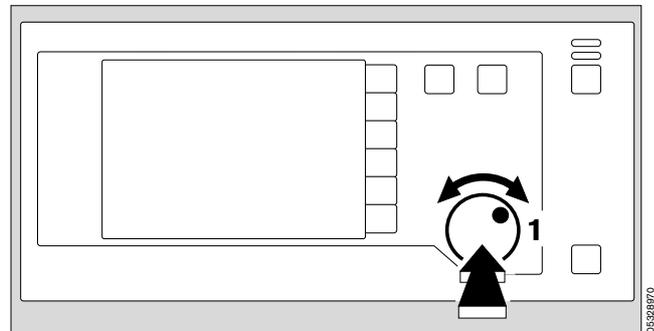



On ventilator:

- 2 Display: Digits 1 to 10 = test blocks during the self-test – refer to the flow chart on pages 124/125 for further details.

Messages in the display are accompanied by a brief advisory tone which is repeated every 15 seconds if the operator does not undertake some kind of action on the equipment.

The test pressure is displayed as an analog and digital value.

Display (example):

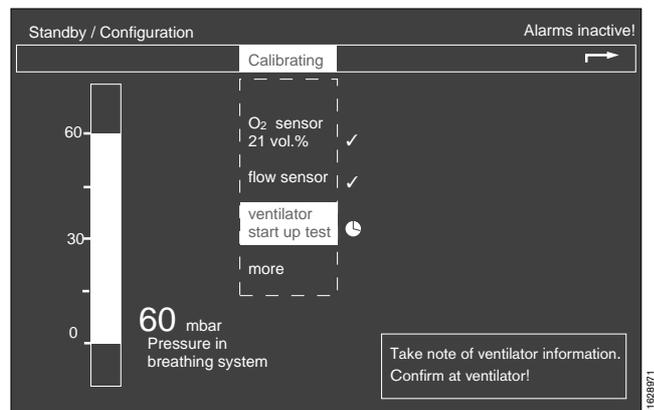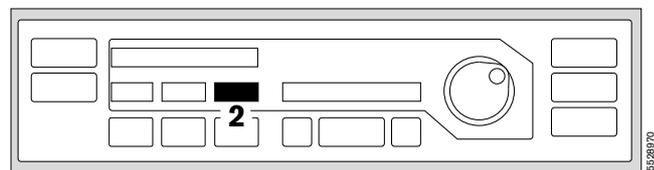

Preparing for use
 Checking readiness for operation with checklist
 Ventilator start-up test

The dialogue continues on the ventilator:

- 1 Test block No. 3 and display:

Fresh gas occl?

- 3 Close all three delivery valves!

Confirm:

- 2 Press rotary control on ventilator!

- 1 Display:

APL = 30 mbar?

- 4 Set pressure limiting valve to »MAN« and 30 mbar!

Confirm:

- 2 Press rotary control on ventilator!

- 1 Display:

Y-piece open?

- 5 Remove Y-piece from cone or exercise thorax!

Confirm:

- 2 Press rotary control on ventilator!

- 1 Display:

Self-test

- 1 Display:

Ypiece occluded?

- 5 Plug Y-piece onto cone!
 (The sampling line is connected to the Y-piece and the water trap.)

Confirm:

- 2 Press rotary control on ventilator!

- 1 Display with test block No. 4:


Self-test

The test then continues automatically, block by block:


- wait – approx. 1 minute!

- 1 Display:


Leaktest IPPV

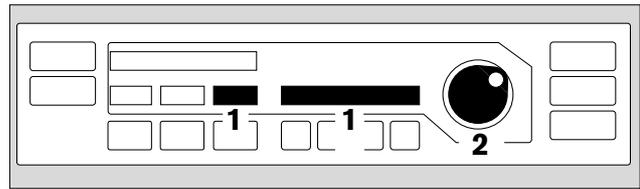

05728970

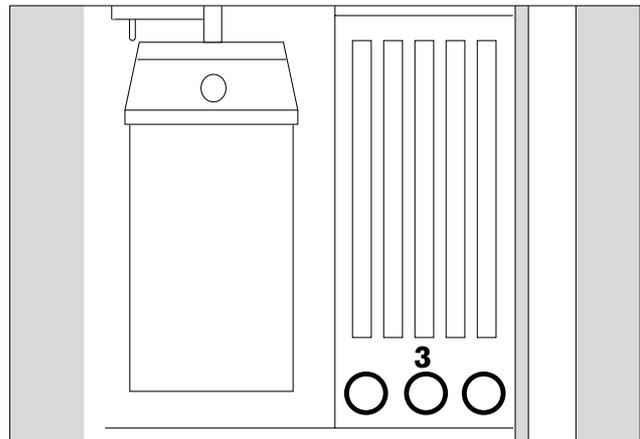

01437224

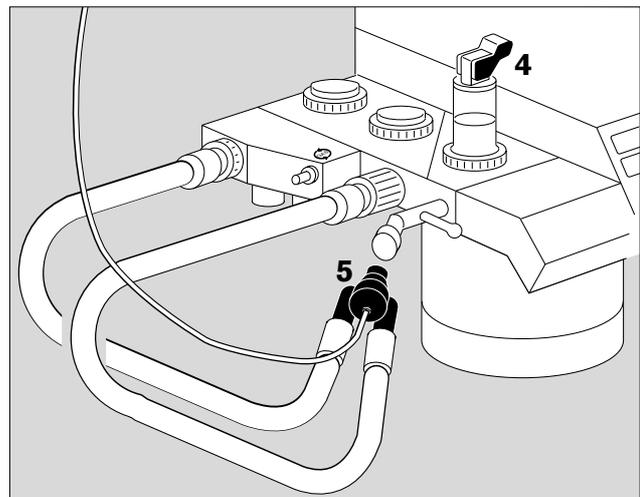

05628970

1 Display:

Leak IPPV = xxx mL

If greater than 175 mL/min:

Repair leak, particularly for low-flow, or continue if not problematic –

Confirm:

2 Press rotary control on ventilator!


1 Display with test block No. 9:

Leaktest MAN

1 Display:

Self-test

- Wait – approx. 1 minute!

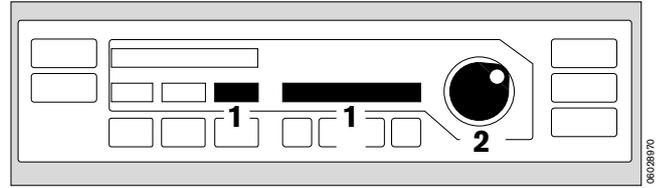

A tick (✓) appears on the screen when the ventilator test has been completed successfully.


The prompt »Confirm« appears.

- Press rotary control on monitor.

The cursor jumps to the arrow symbol on the configuration menu and the vertical pressure indication disappears.


- Press the rotary control on the monitor again to confirm and exit the configuration menu. Configuration is now complete.

- ✓ Tick off the ventilator self-test and monitor self-test on the checklist.

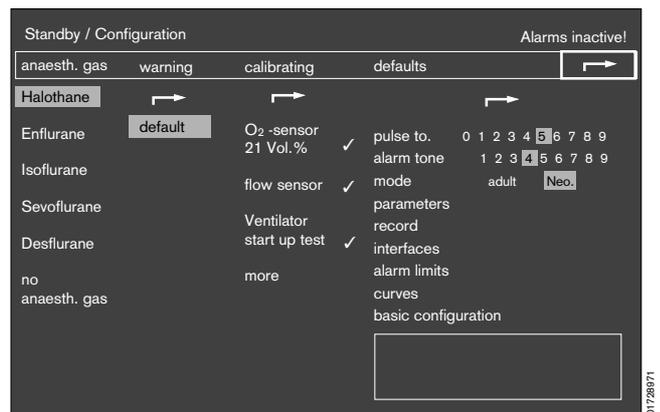


Preparing for use
Checking readiness for operation with checklist
Ventilator start-up test

1 Press rotary control on monitor –

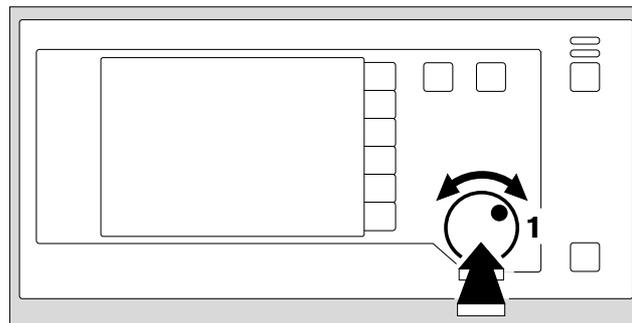

The machine changes to »standby« mode and the following display appears (example):

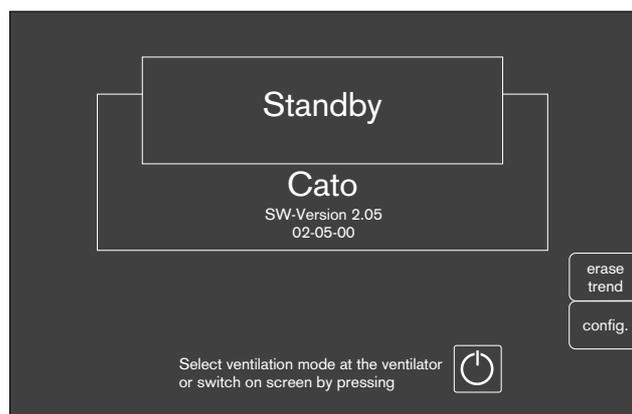

2 On the ventilator:

Standby

»Standby« means that:

- the system can immediately be switched to any operating mode;
- gas consumption is zero (neither drive gas nor anaesthetic gas);
- power consumption is marginal;
- the piston cylinder unit is in withdrawal position.


Manual ventilation is not possible in standby mode!

Any attempt to undertake manual ventilation in »standby« mode is immediately detected and causes the system to start operation in MAN/SPONT mode. (»Auto-WakeUp« function, see page 8).

The equipment is now ready for operation!

The monitor and ventilator self-tests have been completed successfully.

- ✓ Tick off in checklist, select the settings required for the patient and
- ✓ tick off in checklist.
- Remember to sign and date the checklist!

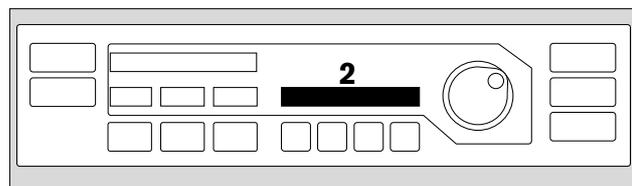

Operation

Contents

	Page
Manual / Spontaneous	36
Select Manual / Spontaneous mode	36
Spontaneous breathing	36
Manual ventilation	37
Manual ventilation following a power failure	37
Emergency ventilation following a gas failure	37
Fresh gas - External outlet (optional)	38
IPPV	39
Selecting IPPV mode	39
Adjusting ventilation parameters	39
Automatic compliance correction	40
Ventilation with pressure limitation (PCV)	40
Limit settings	40
SIMV	41
Selecting SIMV mode	41
Paediatric use	42
Anaesthesia ventilation with the Kuhn system	43
Monitoring with reduced alarm limits	44
Checking water traps	44
at the water separator in the gas sampling hose	44
in the breathing hoses	44
Checking the soda lime	45
Anaesthetic vaporiser	45
Secretion aspirator (optional)	46
Changing patients	46
Set machine to »standby«	48
After use	48

Operation
 Manual / Spontaneous
 Select Manual / Spontaneous mode
 Spontaneous breathing


Manual / Spontaneous

Before connecting a patient


- Check workstation with checklist (see page 21 onwards).
- Check that breathing system is complete (see page 23) and
- perform leakage test (see page 31 onwards).

Select Manual / Spontaneous mode

- 1 Press »MAN SPONT« on ventilator for at least one second (or press »MAN SPONT« and confirm with the rotary control. Setting by Dräger Service).
- 2 Display in dialogue field:
MAN/SPONT

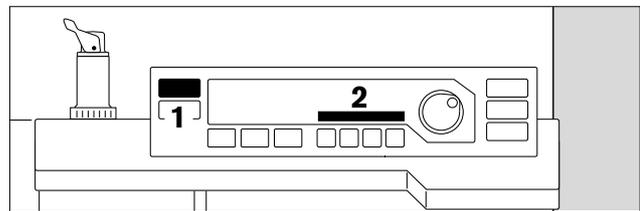

The standard screen appears with the alarm limits for MAN/SPONT mode.

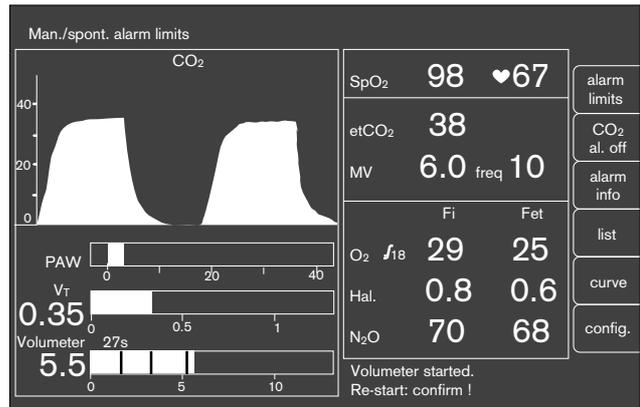


Spontaneous breathing

»PEEP« and pressure limitation »P_{max}« are inactive.

- 3 Set pressure limiting valve APL to »SPONT«. It is now open, regardless of the set pressure.

To fill system:

- 4 Press »O₂ +« to inflate the breathing bag rapidly.
- 5 Set fresh gas – detailed information on setting the fresh gas flow can be found in the Annex on page 128.

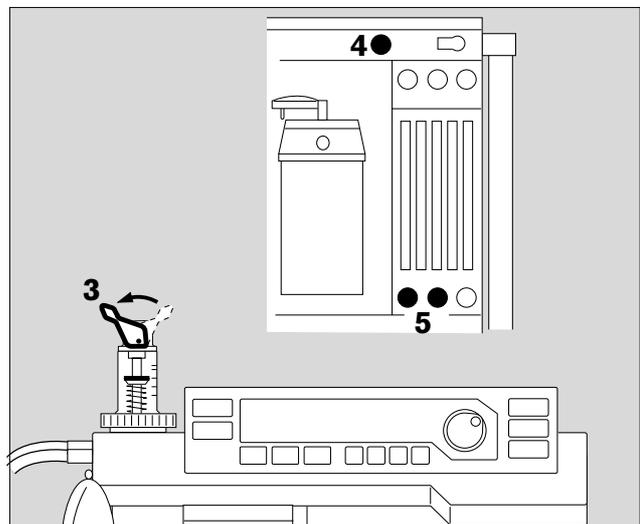

Manual ventilation


with breathing bag

»PEEP« and pressure limitation »P_{max}« are inactive.
The airway pressure is limited via the pressure limiting valve APL.

- 1 Set pressure limiting valve APL to »MAN« and set required ventilation pressure: turn valve head.

To fill system:

- 2 Press »O₂ +«.
 - 3 Set fresh gas with O₂, N₂O or AIR delivery valve –
- start manual ventilation.

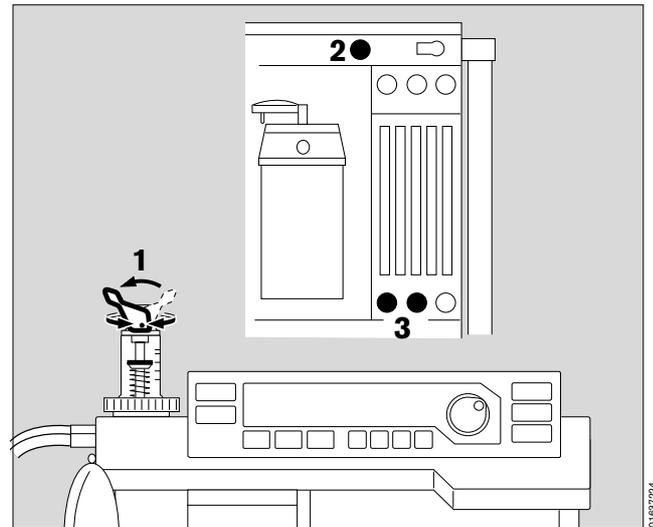

Manual ventilation following a power failure

with breathing bag

(Only relevant if the machine is not equipped with an uninterruptible power supply).

In the event of a power failure, the airway pressure pushes the ventilator piston into its limit position, thus increasing the system volume by up to 1.4 litres!

To compensate this:

- 1 Set pressure limiting valve APL to »MAN« –
 - 3 temporarily increase fresh gas flow or
 - 2 press »O₂ +«.
 - 3 Set fresh gas with O₂, N₂O or AIR delivery valve –
- start manual ventilation.

Emergency ventilation following a gas failure

AIR changeover

The ventilator drive and ejector generating a negative pressure in the secretion aspirator are normally supplied with gas from the compressed air supply (AIR). If AIR fails, the system automatically switches over to O₂.

N₂O lock and O₂ shortage alarm

If O₂ fails, an O₂ shortage is signalled and the supply of N₂O disabled. The measuring tube block automatically switches to AIR, regardless of the lever setting.



The patient must immediately be ventilated with the separate emergency ventilation bag if both O₂ and AIR fail.

Fresh gas - External outlet (optional)


Only available in "MAN/SPONT" mode

- 1 Connect the anaesthetic gas scavenging hose, see page 92.
- 2 Press »MAN SPONT« on the ventilator.
- 3 Switch on the external fresh gas outlet – Press »FRESH GAS OUTLET«
The lamp in the key lights up.
The circle system is switched off.
- 4 Monitoring the breathing gas (O₂ and anaesthetic gases) in semi-open mode with aspiration measurement at the fresh gas supply connector of the semi-open system, and with the screen in HLM mode.




Intake measurement is performed without connection to the breathing phase. There is no pressure monitoring.

When switching on and in the event of a power failure lasting longer than 2 minutes, Cato switches automatically to the internal fresh gas outlet.

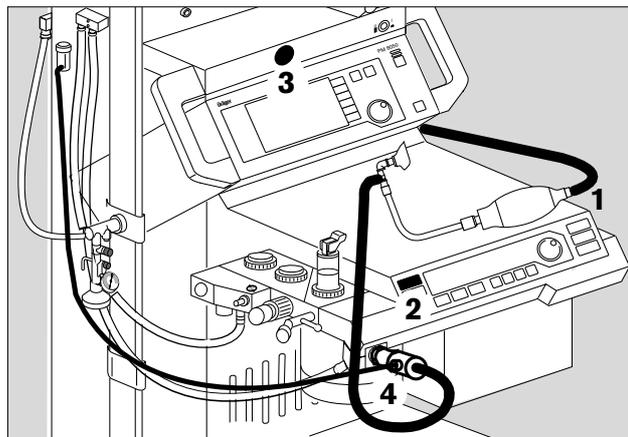

01737224

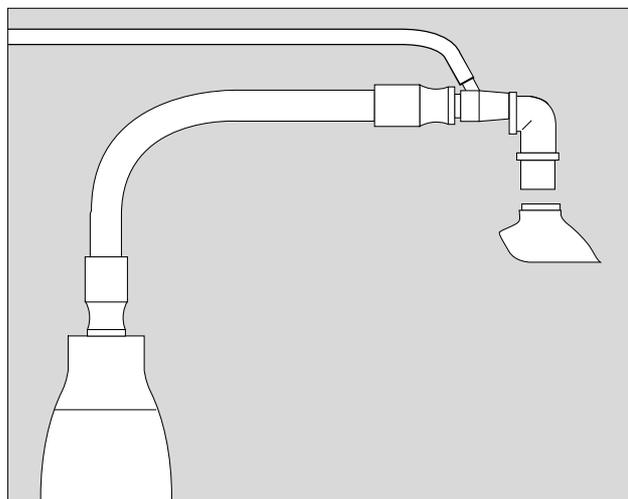

07028870

IPPV

Selecting IPPV mode



Secretion aspirator may only be used in »MAN/SPONT« mode or with disconnected Y-piece.

After switching on, the ventilation parameters set prior to delivery or subsequently programmed by DrägerService are active in IPPV mode.

- 1 Set the values required for the patient.

Default settings upon delivery of new equipment:
(may be altered by DrägerService if requested by customer)

VT	Tidal volume	0.6 L
f _{IPPV}	IPPV frequency	12 breaths/min.
P _{max}	Maximum ventilation pressure	25 mbar
Ti:TE	Inspiration/expiration time ratio	1 : 1.7
TiP:Ti	Inspiration pause/inspiration time ratio	10 %

- 2 Set PEEP – if necessary.



- 3 Press »IPPV«.

- 4 Display in dialogue field:
IPPV Mode ?

- 5 Press rotary control to confirm – ventilation starts.

The parameters P_{max}, VT, f_{IPPV}, PEEP and piston movement are displayed on the ventilator.

The monitor is also started and the IPPV alarm limits are activated.

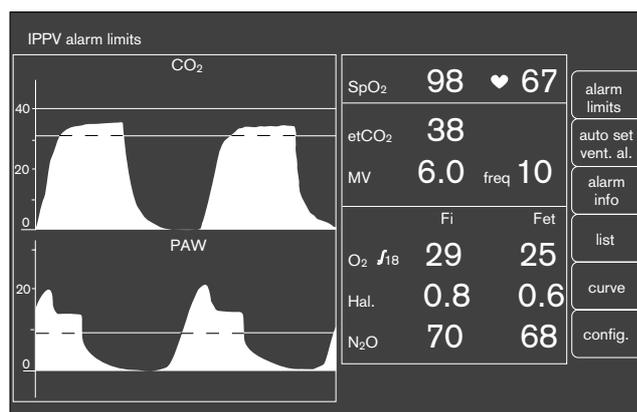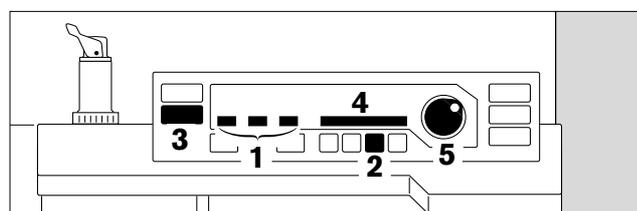

Adjusting ventilation parameters (P_{max} for example):

- 6 Press »P_{max}«.

- 1 The set value appears in the window above, while
- 4 the set value (left), the parameter and its unit of measure (middle) and the value to be adjusted (right) appear in the dialogue window.

- 5 Turn rotary control:


- 4 the value on the right changes –

- 5 continue turning until the required maximum pressure is reached. Press rotary control to confirm this value –

- 1 the new value is displayed –

if the new value is not confirmed, it will not be adopted by the system and the display disappears after approx. 10 seconds.

- f_{IPPV}, VT, Ti:TE, TiP:Ti, PEEP and fIMV are adjusted like P_{max} after pressing the corresponding parameter key.

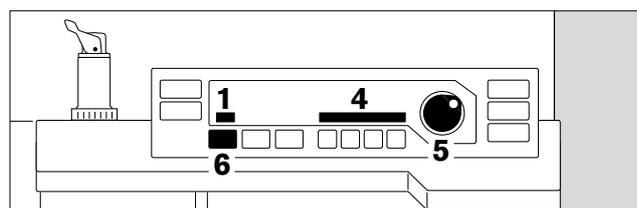

Automatic compliance correction

Only set the effective tidal volume!

The compliance of the breathing system and of the hoses used is established by the equipment during the self-test or if a leakage test is started manually. The reduction in tidal volume due to system compliance is then corrected automatically during ventilation so that the patient actually receives the set tidal volume.

The leakage test should therefore be repeated whenever changes have been made in the hoses.



The patient must always be disconnected and the system set to »standby« before starting the leakage test! (see page 48 onwards)

A detailed description of automatic compensation of the system compliance can be found in the Annex on page 129.

Ventilation with pressure limitation

When the set maximum ventilation pressure P_{max} has been reached, the inspiratory stroke is adjusted so that the pressure remains constant up to the end of inspiration.


The set tidal volume is not fully applied in this case!

1 Display on ventilator:

Pressure limit

2 The bar graph on the ventilator does not reach 100%.

If the pressure increases by more than 5 mbar above the maximum ventilation pressure P_{max} , e.g. because the patient coughs, inspiration is immediately stopped and expiration starts.

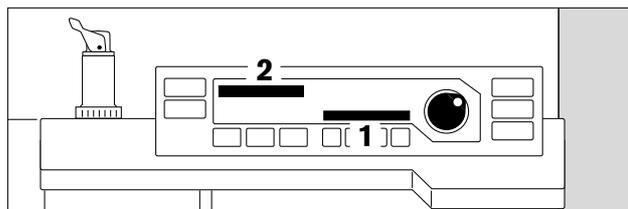

Limit settings

– P_{max} , PEEP

The minimum difference between P_{max} and PEEP equals 10 mbar. Settings resulting in a smaller pressure difference are not accepted by the system.

– Max. inspiratory flow

The tidal volume, frequency, I:E ratio and inspiratory pause time cannot be set to values resulting in an inspiratory flow of more than 75 L/min.

– Max. minute volume

The tidal volume and frequency cannot be set to values resulting in a minute volume of more than 25 L/min.

SIMV

Selecting SIMV mode



Secretion aspirator may only be used in »MAN/SPONT« mode or with disconnected Y-piece.

In order to prevent the mechanical mandatory ventilation stroke from being applied during the expiratory spontaneous breathing phase, a special trigger ensures that the mandatory ventilation stroke is controlled by the patient and consequently synchronized with spontaneous breathing (this is described in detail on page 133).

PEEP is not active in SIMV mode!

After switching on, the ventilation parameters programmed upon delivery are active in SIMV mode.

The ventilation parameters of the preceding mode remain active when changing from IPPV to SIMV mode and vice versa!

Settings upon delivery of new equipment:

VT	Tidal volume	0.6 L
fIMV	IMV frequency	12 breaths/min.
P _{max}	Maximum ventilation pressure	25 mbar
Ti:TE	Inspiration/expiration time ratio	1 : 1.7
TiP:Ti	Inspiration pause/inspiration time ratio	10 %

If the ventilation frequency is equal to or greater than 6 breaths per minute in »SIMV« mode, the IPPV alarm limits become active and the alarm mode »**IPPV alarm limits**« is automatically displayed on the monitor.

If the ventilation frequency is less than 6 breaths per minute in »SIMV« mode, special SIMV alarm limits become active and the advisory message »**SIMV alarm limits**« flashes in the status field of the monitor for five seconds. This message does not require confirmation.


When setting frequencies of **more than 5 breaths per minute**, the system **automatically** reverts to the IPPV alarm limits and indicates this on the monitor.

The ventilation parameters are adjusted in the same way as in IPPV mode:

- 1 Press »SIMV«.
- 2 Display in dialogue field:
SIMV Mode ?
- 3 Press rotary control to confirm – ventilation starts.

The parameters P_{max}, VT, fIMV, PEEP and piston movement are displayed on the ventilator – the monitor is also started and the alarm limits for SIMV are activated.

Refer also to the chapter describing the »**Alarm concept**« on page 65.

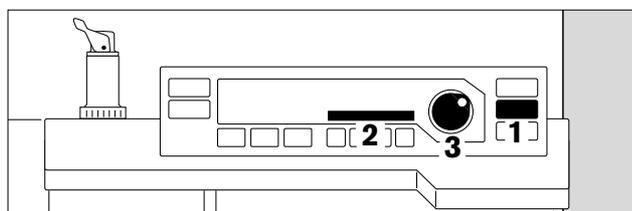

07525870

Paediatric use

Infant hose set

Infant hoses must be used for ventilation volumes of less than 200 mL.

If a breathing gas humidifier is used, water traps must be installed at the lowest points of the breathing hoses on both the inspiratory and the expiratory side.

Avoid pressure peaks

The fresh gas is stored in the breathing bag during inspiration. The pressure built up in the breathing bag when working with high flow rates and long inspiration times may be higher than the end-inspiratory pressure in the patient, particularly when using a 0.5 L breathing bag.

Even at a fresh gas flow of 4 L/min, a pressure peak may arise at the beginning of the expiration phase as fresh gas streams out of the breathing bag. This is particularly possible in combination with long inspiration times and can be avoided by reducing the fresh gas flow or using a 1.5 L breathing bag.

The adjustment intervals and metering accuracy for the tidal volume VT depend on the selected range.

Range	Interval	Metering accuracy
< 20 mL	1 mL	± 30% or ± 6 mL
20 to 50 mL	2 mL	± 10% or ± 10 mL
50 to 100 mL	5 mL	± 10% or ± 10 mL
100 to 990 mL	10 mL	± 5% or ± 15 mL
1 L to 1.4 L	0.01L	± 5% or ± 15 mL

The system must calculate its new system compliance following a change of hoses.



The patient must be disconnected for this purpose and the leakage test started!
(see overleaf)

The ventilation parameters are adjusted in the same way as in IPPV mode.


If a tidal volume of less than 200 mL is selected from a higher setting, the system automatically generates the

1 display:


Paed. hoses !

● Fit infant hoses –

2 press rotary control to confirm.

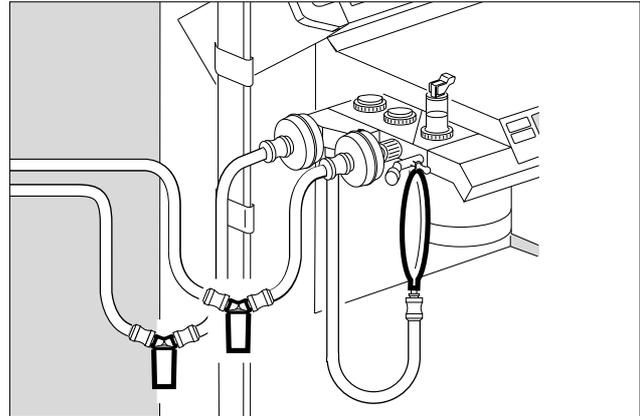

07/288790

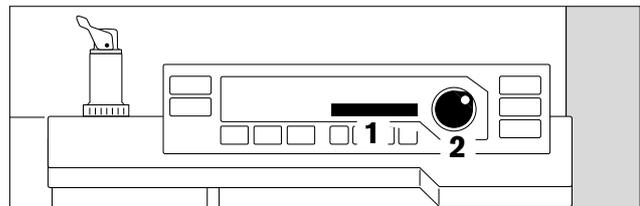

07/288790

Then switch to »standby« (see page 48) and call up the leakage test so that the new compliance can be calculated.

Invoking the leakage test

- 1 Press »TEST« on the ventilator for at least three seconds.
- 2 Display in dialogue field on ventilator:

Leaktest IPPV


followed by:

Leak IPPV = xx mL

The calculated leakage and compliance values are saved and displayed on the data screen.

The manual ventilation bag and its hose are not included in the test!

The system switches back to »standby« after the test.

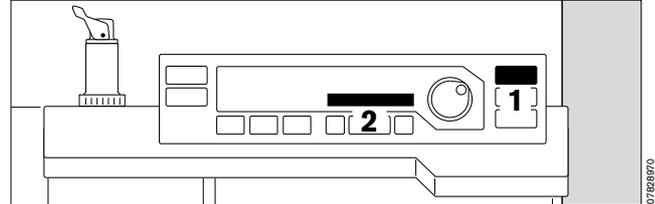

Anaesthesia ventilation with the Kuhn system

If the machine does not have an external fresh gas outlet (optional - see page 38), anaesthesia ventilation with the Kuhn system is performed as follows.:

- Prepare the Kuhn set in accordance with separate Instructions for Use –
- connect anaesthetic gas scavenging hose (see page 92).
- 3 Press »MAN/SPONT« on ventilator for at least one second –

- 4 set pressure limiting valve APL to »MAN«.

- Connect inspiration hose via



- 5 connecting sleeve – Symbol: ←

- 6 Expiration connector remains open – Symbol: →

- 7 Seal manual ventilation connector with cap. Symbol: ☉

The pressure indicated on the monitor is not identical with the actual airway pressure. Reason: higher flow resistance of the fresh gas hose in the Kuhn system. The higher the fresh gas flow, the greater the difference.

- 8 The anaesthetic and O₂ concentration must be monitored by means of intake measurement with the monitor in HLM mode.

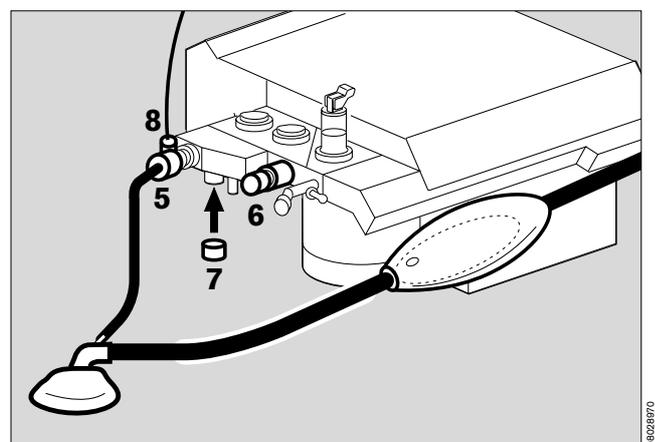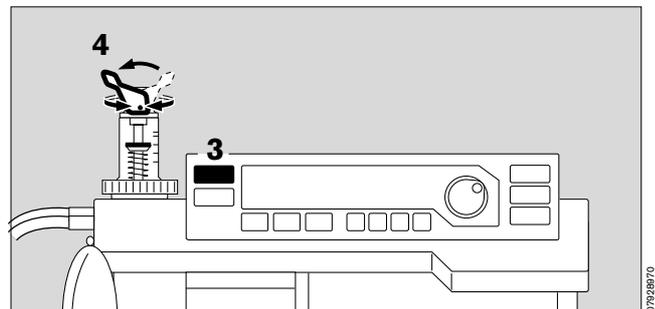

Monitoring with reduced alarm limits

The »**IPPV alarm limits**« are automatically activated on the monitor when a ventilation frequency **equal to or greater than 6** breaths per minute is set in »**IPPV**« and »**SIMV**« modes.

The »**SIMV alarm limits**« are automatically activated on the monitor if a ventilation frequency of less than 6 breaths per minute is set in »**SIMV**« mode.

The »**MAN/SPONT alarm limits**« are automatically activated on the monitor in »**MAN/SPONT**« mode. This prevents false alarms (see »**Alarm concept**«, page 65 onwards).

Checking water traps

WaterLock

(refer also to the separate Instructions for Use)


When fouled

or
if **CO₂ LINE ? !** is output although the sample line is clear.


- Replace water trap:
 - when severely fouled,
 - if the error message persists even after draining the water trap,
 - when the maximum service period of four weeks is exceeded.
- Grasp the ribbed sides of the water trap and draw it out.
- Remove a new water trap from the package.
- Enter the date in the field provided on the water trap.
- Grasp the ribbed sides of the water trap and slide it into the holder – until it tangibly engages.
- Empty water trap:
 - Insert an empty syringe without cannula, at least 20 mL, into the connector.
 - Extract the water, remove syringe and dispose of the full syringe as ordinary domestic waste.
 - Push water trap back into holder – until it tangibly engages.

Disposal:

- Water traps which have been dismantled must be disposed of as ordinary commercial waste.
Waste code number 91101.

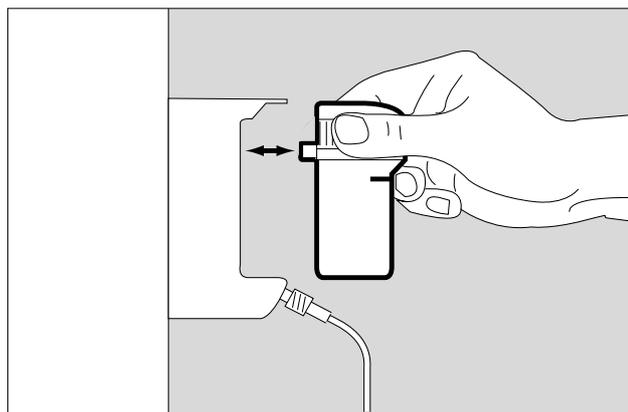

01857224

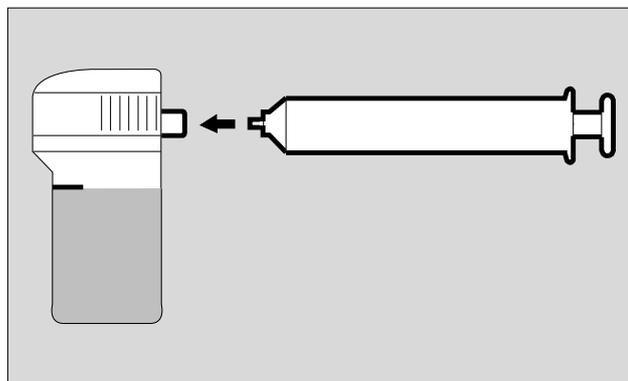

04137224

in the breathing hoses


- Water traps must be fitted at the lowest point in the hose and hang downwards.
- They must be checked regularly and drained if necessary.



Note hygiene regulations – risk of infection!

The hose system remains sealed. Container must be replaced securely.

"Tips on reducing condensation", see page 136.

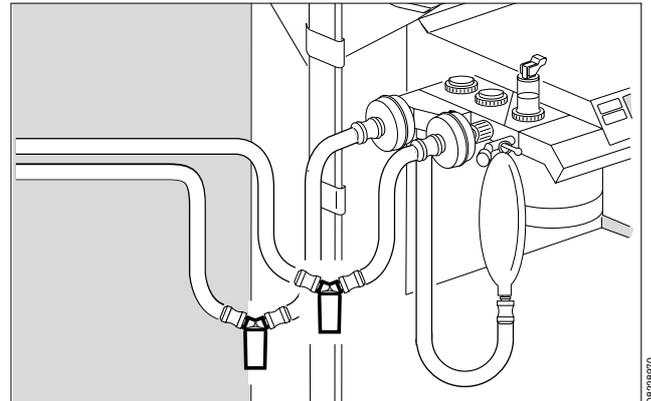

Checking the soda lime

The soda lime turns purple from the bottom upwards when saturated with CO₂. To monitor CO₂ absorption, the FiCO₂ value is displayed in "data screen" mode. An alarm limit of 5 mmHg is set for monitoring CO₂ absorption.

The soda lime must be changed when two-thirds of the charge has changed colour! The colour may also fade again due to drying out after prolonged breaks.


Changing soda lime during operation

(see also page 91)

- Prepare a replacement soda lime container.
 - Occlude the Vapor and stop N₂O delivery.
 - Set ventilator to »MAN/SPONT« and the pressure limiting valve to »SPONT«.
- 1 Briefly turn the soda lime container anticlockwise and pull it downwards.
- Insert the replacement soda lime container from below and turn clockwise as far as possible.
 - Gas and machine settings may be cancelled.
 - Remove spent soda lime from the container.



Note hygiene regulations when dealing with infectious patients – risk of infection!

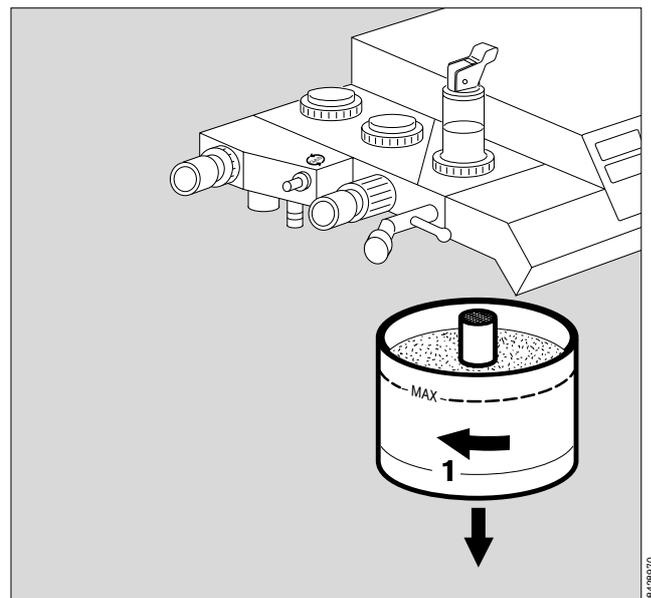

Anaesthetic vaporiser


The illustration shows a Vapor 2000.

- Control dial with scale in % anaesthetic agent by volume.
- »0« button – engages in zero position »0«.
- Locking lever – Vapor is locked onto the plug adapter.

Metering anaesthetic agent:

- 1 Press »0« button and turn control dial anticlockwise – until required setting is reached.

If Vapor has to be removed – turn control dial to »T«.
Swing locking lever forwards – Vapor can now be removed.

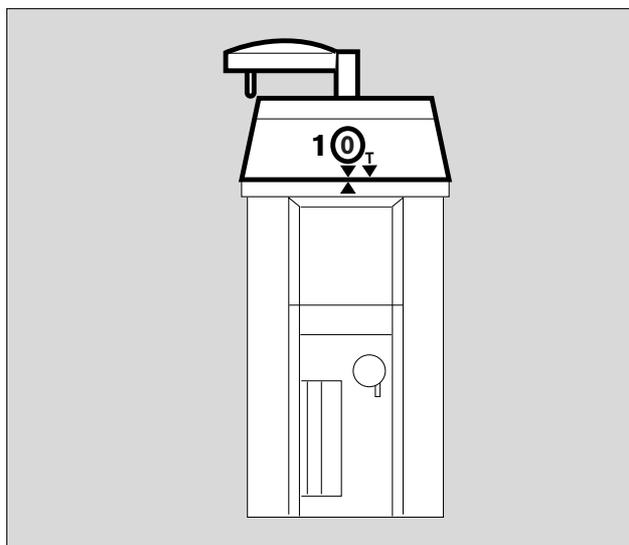


Secretion aspirator (optional)

To prevent negative pressures in the breathing system and in the patient's lungs, we recommend that you only carry out suction with the Y-piece disconnected from the patient tube.

- 2 Open ejector valve –
 - 3 cover the »Fingertip« with your index finger.
- The aspiration capacity can be adjusted very accurately with the aid of auxiliary air on this surface.
 - Check filling level of aspiration vessel regularly and drain if necessary.



Note hygiene regulations – risk of infection!

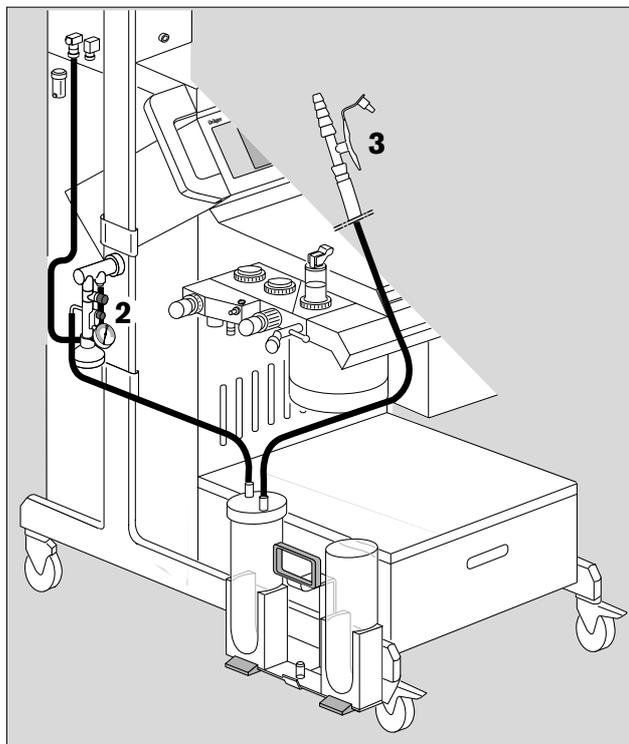

Changing patients

Changing parts

Refer to page 83 onwards for a schematic overview and description of methods for cleaning and disinfection.

Maintenance intervals: page 106.

- Machine in »standby« mode –
- Vapor: control dial set to »0« = Off.

The T-piece and filter are not required when using a Y-piece with Luer lock.

After treating an infectious patient

the entire machine must be cleaned, disinfected and sterilized.



Note hygiene regulations – risk of infection!

After treating a non-infectious patient

the following parts must be replaced before continuing with the next patient:

- Tube or mask
 - Y-piece
 - Both breathing hoses
 - Temperature sensor and cable if applicable
- 1 T-piece of the measured gas hose and filter, if used (disposable articles, household refuse)


- 2 Drain:
Insert an empty syringe without cannula, at least 20 mL, into the connector.

- Extract the water, remove syringe and dispose of the full syringe as ordinary domestic waste.
- Push water trap back into holder – until it tangibly engages.


An overfull container or a faulty water trap could lead to failure in sidestream measurement.

The following must also be changed when working without microbial filter:


- 3 O₂ sensor with connecting lead, if used.
 - 4 Flow sensor.
Disconnect lead from flow sensor, it remains on the machine.
- Breathing bag with hose
 - Breathing system:
Disconnect fresh gas hose, it remains on the machine.
 - Pressure measuring hose and filter: must be replaced. Hose connector remains on the machine.
 - Soda lime container – turn anticlockwise and pull down – must be replaced.
 - Anaesthetic gas scavenging hose – release (latch in front of connector) and pull off, also at the central supply point.

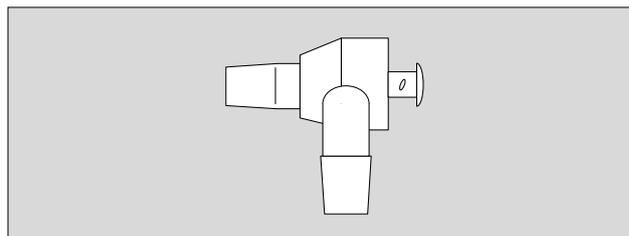


00728970

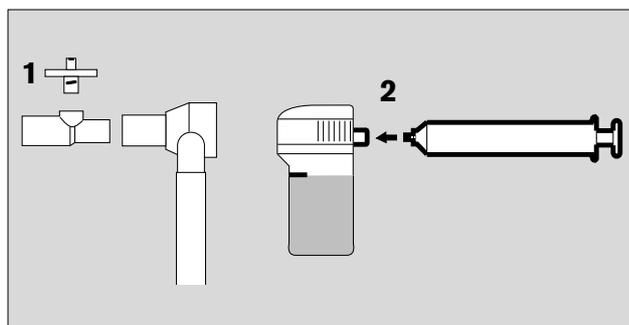

02107224

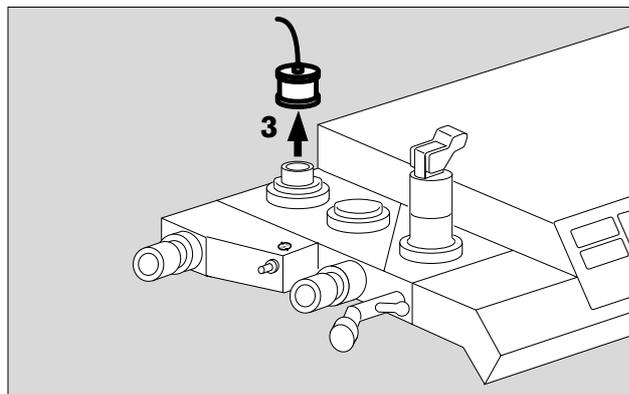

00828970

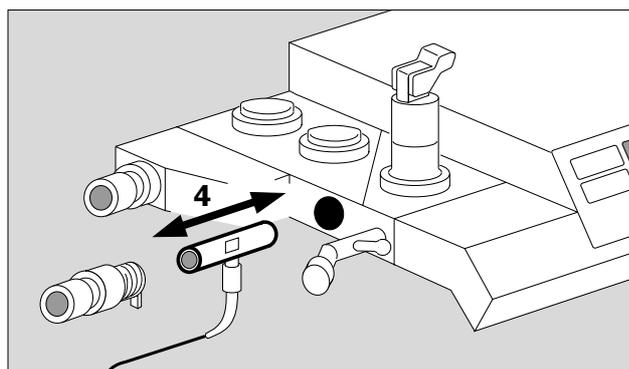

00828970

Operation
Set machine to »standby«
After use

Set machine to »standby«

In »standby« mode,

- the system can immediately be switched to any operating mode,
- gas consumption is zero,
- power consumption is marginal and
- the piston cylinder unit is in withdrawal position.

- 1 Press  on the ventilator for at least three seconds.
- 2 Display
Standby

The flow tubes are no longer lit up.

The monitor remains operational and should be set to »standby« by pressing  if not required.

The monitor can only be switched to standby if the ventilator is already in standby!



During breaks:

- Set fresh gas flow to »O«.
- On Vapor: set control dial to »0«.

After use

If operation is interrupted for several hours, we recommend that you do not leave the Cato in »Standby« mode but switch it off completely.

- Turn the master switch to »O« = Off – the switch symbol is no longer white. After switching off, an OFF delay displays the message »Power off« in the Ventilator display for about 10 seconds.
- Disconnect the gas hoses from the wall sockets, or close the cylinder valves of the gas cylinders.
- Roll up hoses and hang them over the holder at the back of the machine.
- Disconnect the mains plug.

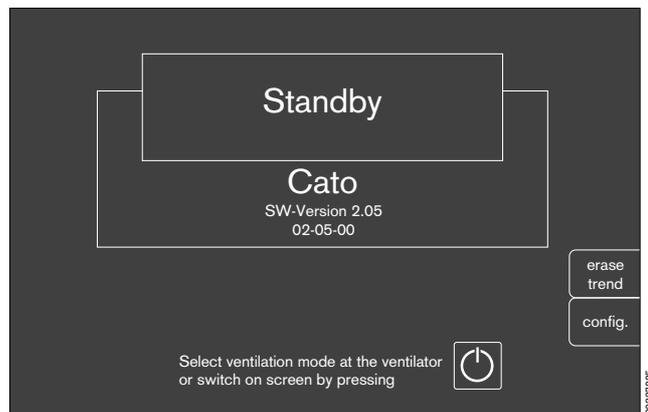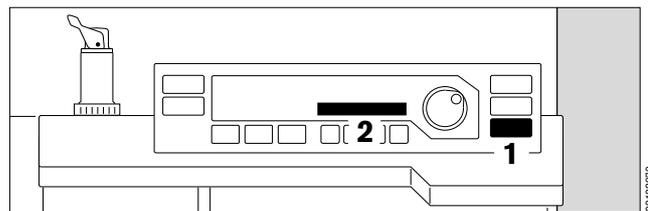

Monitor functions

Contents

	Page
Screen saver	50
Standby screen	50
Configuring monitor functions in »standby«	51
Invoking the configuration menu	51
Changing default values	51
Pulse tone, alarm tone, mode	52
Parameters, record, interfaces	53
Alarm limits, curves, basic configuration	54
Calibration	55
Alarms, anaesthetic agent	56
Monitor functions during operation	57
Changing to measuring mode from »standby«	57
Select standard screen, select data screen	57
Select trend screen with zoom function	58
Other trend combinations	59
Return to full trend	59
Erase trend memory	59
Softkeys	60
Displaying and setting alarm limits during operation	60
Activate and deactivate CO ₂ alarms	61
Auto set/Ventilation alarm	61
Alarm information	62
Select list screen	62
Select curves	63
Using the volumeter function	63
Configuration during operation	64
Alarm concept	65
Alarm priority	65
Displaying all alarms	66
Suppressing the alarm tone	66
CO ₂ alarm on/off	66
Adjusting to the ventilation mode	67
Heart/lung mode (HLM)	68


Screen saver

If none of the operating elements on the Cato is operated in »standby« for approx. 2 minutes, the screen switches off and becomes dark. The yellow LED in the standby key and the word »Standby« on the ventilator light up. The monitor display is immediately restored as soon as any key is pressed.

Standby screen

The »standby« screen contains two softkeys with which to invoke menus for

- erasing the trend and list memory,
- monitor configuration.

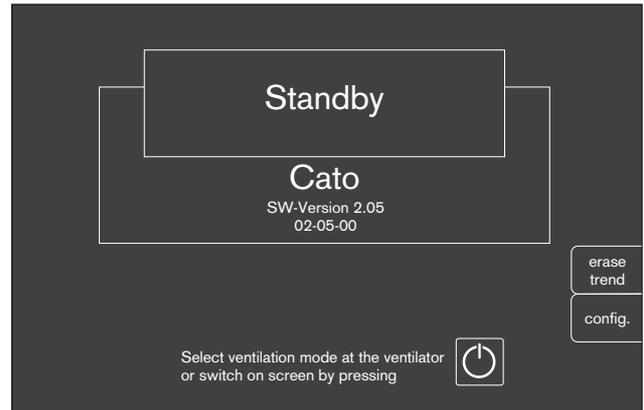

Erase trend

e.g. for a new patient:

The trend memory and list are erased together! This is only possible in »standby«!

- Press  on the screen in order to switch to »standby«. This is only possible if the ventilator is also on »standby«!



- Press the softkey »erase trend«.

The system enquires again whether the trend is indeed to be erased.

- Press the softkey »erase« to confirm.

The former screen is restored without change if the softkey »Do not erase« is pressed.

The saved anaesthetic concentration trend is erased when changing the anaesthetic vaporizer!

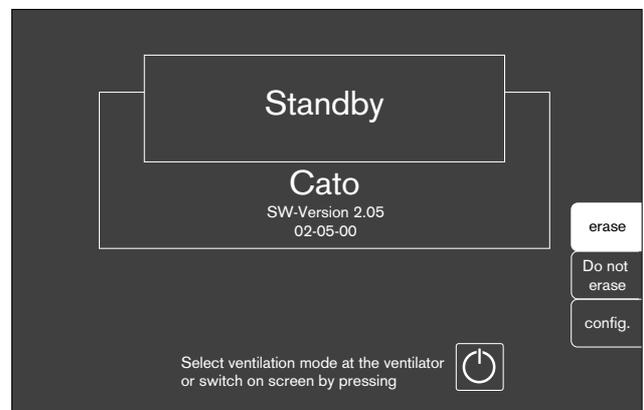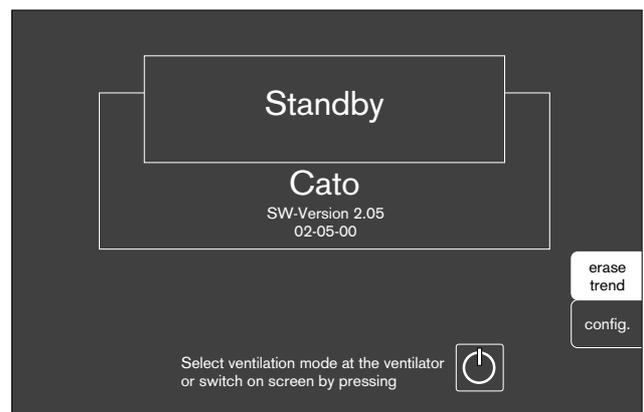

Configuring monitor functions in »standby«

Invoking the configuration menu

- The configuration menu is called up by pressing the softkey »config«.

Configurations which are set under »defaults« in »standby« mode remain active whenever the machine is switched on again.

All other configurations, on the other hand, only apply temporarily.

The screen shown alongside appears on the monitor:

The

- defaults,
- calibration,
- alarms and
- anaesthetic gas




can be altered with this menu.

These settings are always called up and changed in the same way:

- **Turn rotary control** to displace the cursor frame.
- **Press rotary control** to confirm the selection made with the cursor frame.
- **Fields with grey background** represent the currently valid settings.
- **Fields with white background** show the menu steps via which the momentary field was reached.
- **The arrow (↔)** means return to the preceding menu level.

Changing default values

- Turn rotary control on monitor until the cursor frame is located over »defaults«.
- Press rotary control.
 The operator is then prompted to enter a four-digit password (specified during the initial training) in order to prevent unauthorized changes to the basic functions. This function can be deactivated by DrägerService.
- Use the rotary control to select and confirm individual numbers from the line displayed below. The password is represented by asterisks (* * *) below the line. When the correct password has been entered, the cursor frame jumps to the selection field for the default values.

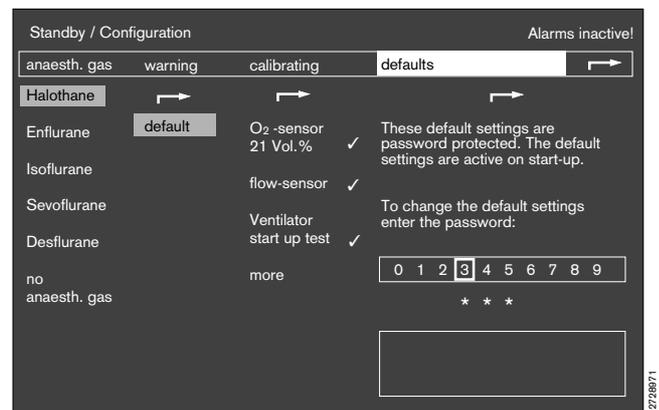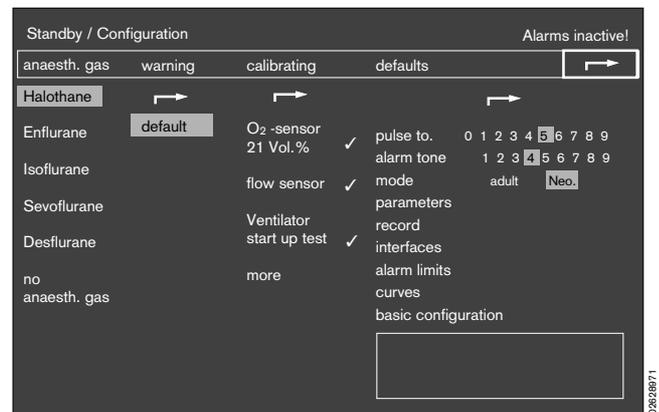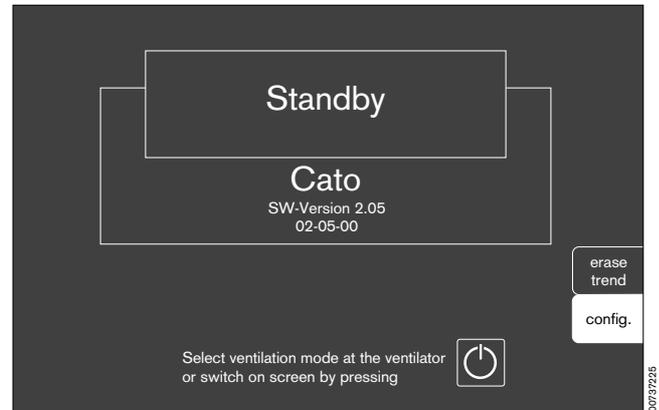

Monitor functions

Configuring monitor functions in »standby«


Changing default values

Pulse tone, Alarm tone, Mode

The vertical selection area is displayed.

The menu for configuring the defaults appears with settings:

- **pulse tone** for setting the volume of the pulse tone.
- **alarm tone** for setting the volume of the alarm tone.
- **mode** for changing over between adult and neonate mode.
- **parameters** for setting the measuring functions.
- **record** for setting the list entries and printer output.
- **interfaces** for setting the parameters for data transfer.
- **alarm limits** for selecting the alarm limits.
- **curves** for selecting the curve display.
- **basic config.** for setting the date, time, language and tone.

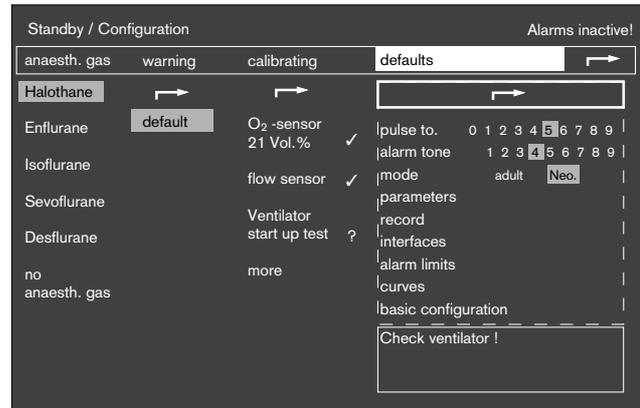


Pulse tone

The volume of the pulse tone is set here.

»0« corresponds to **OFF**, »1« is the lowest volume and »9« the highest volume.

The tone is generated at the corresponding volume as it is adjusted.

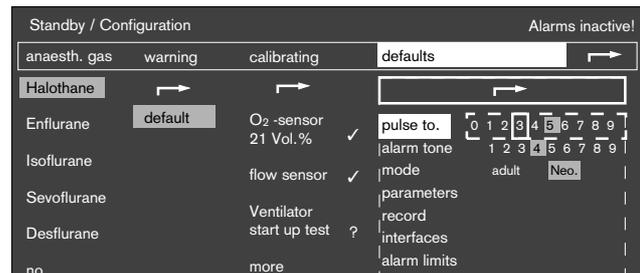


Alarm tone

The volume of the alarm tone is set here.

»1« is the lowest volume and »9« the highest volume.

The tone is generated at the corresponding volume as it is adjusted.

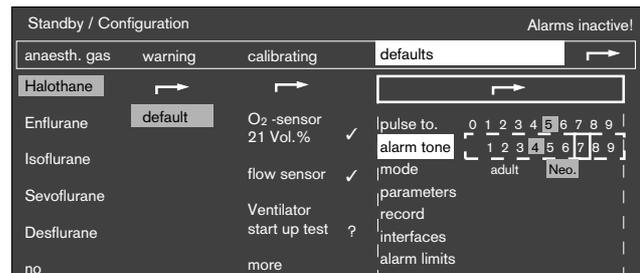


Mode

This function is used to change over between adult and neonate mode:

- **adult** Adult mode
- **neo** Neonate mode


The volumeter functions and trend displays are scaled at the same time.

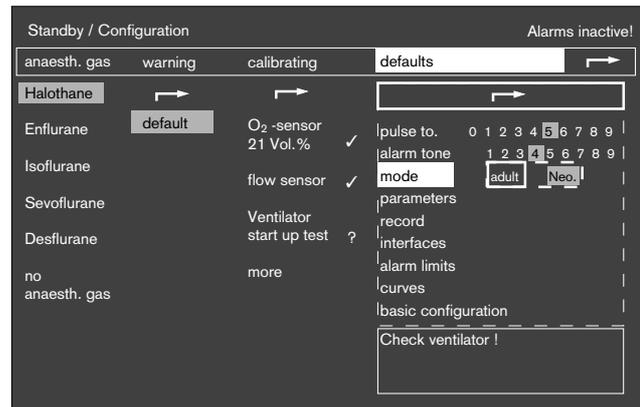

Parameters

The settings for the following parameters are defined via this menu:


- **SpO₂-measurement** on • off
- **sidestream measurement O₂** on • off
- **sample rate** 60 • 200 mL/min
- **CO₂ units** mmHg • kPa • % by volume

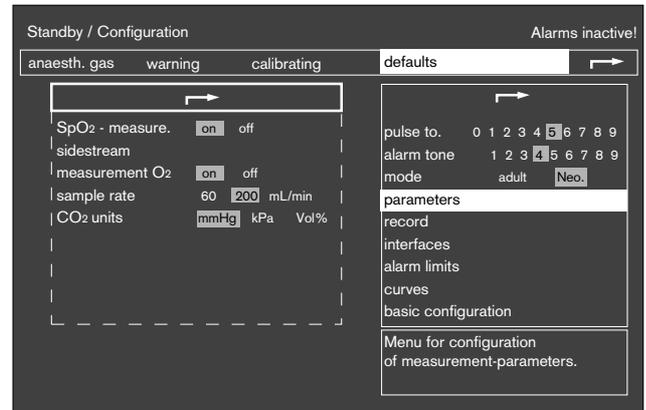

Record

This menu is used to define which event triggers an **automatic** entry in the record list or printout on the on-line printer:


- **time intervall** Entry upon expiry of a fixed time interval in minutes.
- **NiBP started** Entry after every NiBP measurement with new results.
- **warning started** Entry is made whenever a warning has been triggered.
- **caution started** Entry is made whenever a caution has been triggered.

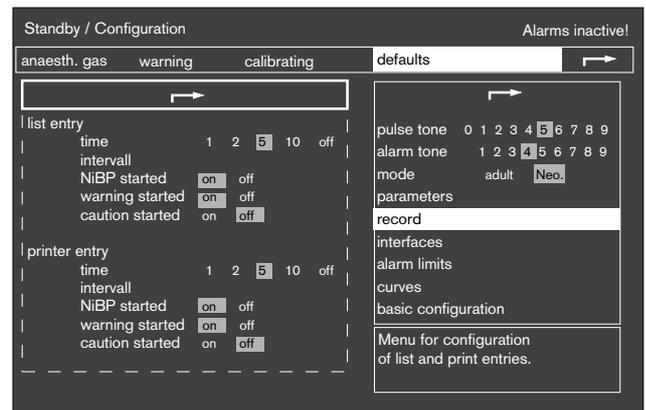

Interfaces

The interface for data transfer (Dräger MEDIBUS RS 232) and the printer port are configured here:

- **baud rate** Transmission rate (can be varied, see Instructions for Use of the device to be connected)
- **parity** Cannot be changed in MEDIBUS and is merely displayed for information.
- **data bits** Cannot be changed in MEDIBUS and is merely displayed for information.
- **stop bits** Cannot be changed in MEDIBUS and is merely displayed for information.
- **protocol** Allows the printer port to be used as a second MEDIBUS interface.

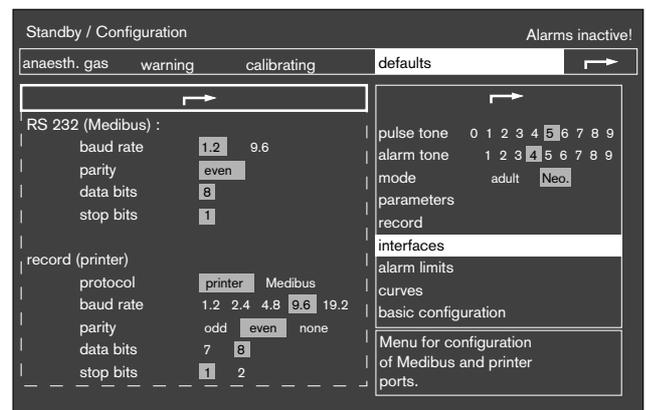

Monitor functions
Configuring monitor functions in »standby«
Alarm limits, Curves
Basic configuration


Alarm limits

This menu is used to specify the standard alarm limits.

These alarm limits are automatically active after

- switching on the Cato,
- selecting »default«.

Two dashes (--) in the table mean that this alarm limit is inactive and not monitored. It can be activated by turning the rotary control beyond the maximum or minimum value possible and confirming it.

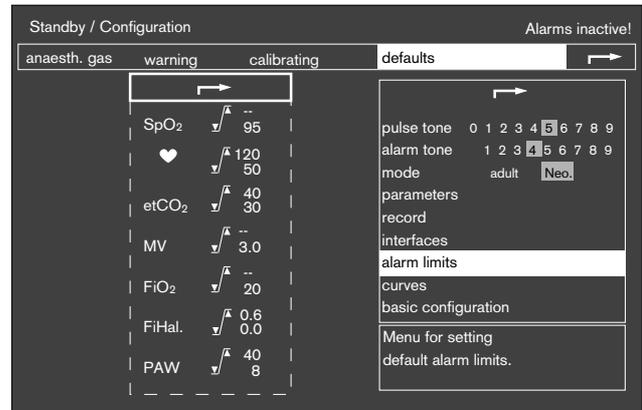


Curves

This menu is used to specify the standard curves.

These curves are automatically active after

- switching on the Cato.

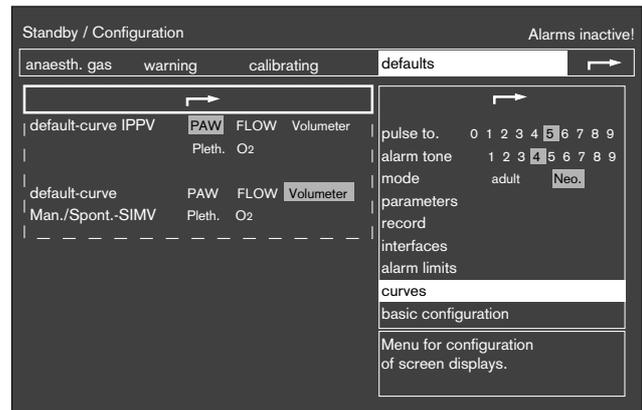

Basic configuration


The basic configuration comprises four items:

- **time** for the current time
- **date** for the current date
- **language** for the language version
 The same language version is also used by the ventilator.

The following languages are available:

English	GB
French	F
German	D
Dutch	NL
Italian	I
Spanish	E

- **tone-sequence** for selecting the tone sequences for warnings, cautions and advisories.

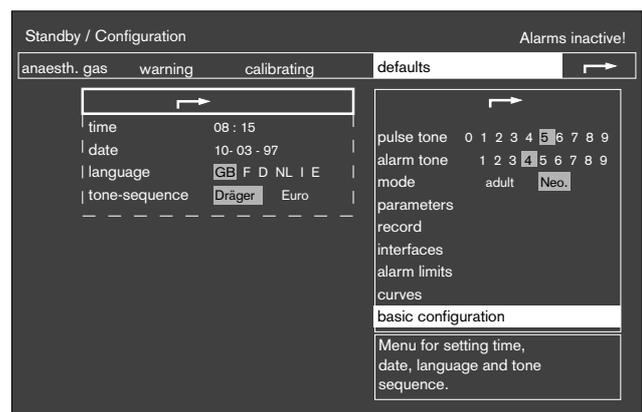

Calibration

This menu is used for the following tests and calibrations:




- Calibrate **O₂-sensor with 21 Vol.% O₂**
- Calibrate **flow sensor**
- Perform **Ventilator start up test**
- **more**

The following can be realized under »more«:

- O₂ sensor calibration with 100% O₂ by volume,
- linearity check of the O₂ sensor and
- calibration of the CO₂ sensor.

The significance of the symbols is as before:

- ? = Enquires whether an action has been performed or a setting made.
- ⊖ = Waiting period. The selected test step is being carried out by the system.
- ✓ = The action has been completed successfully or is not required.

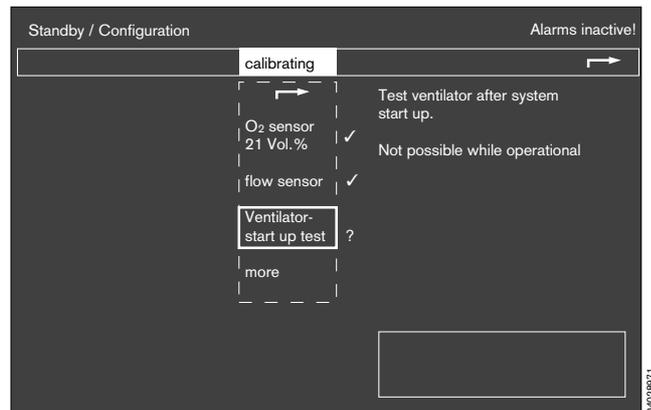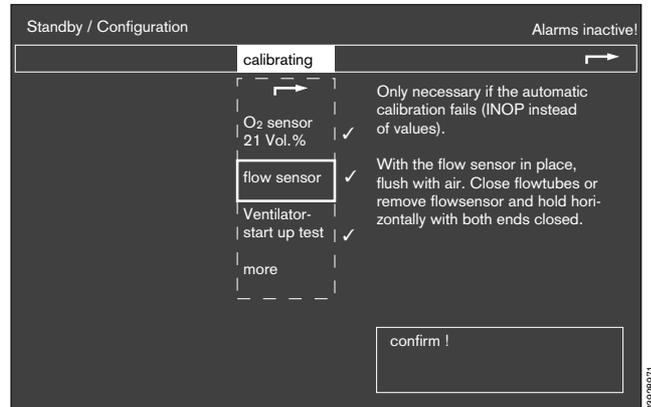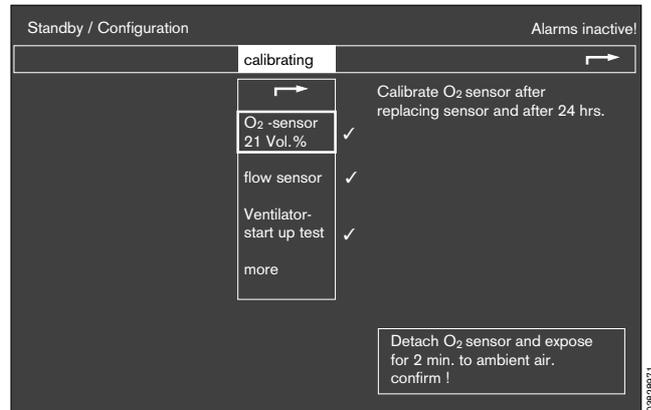


Monitor functions

Configuring monitor functions in »standby« Alarms, Anaesthetic agent

Alarms

The standard alarm limits are activated by selecting and confirming the field »defaults«. The individual settings in the »limits« menu are then overwritten.

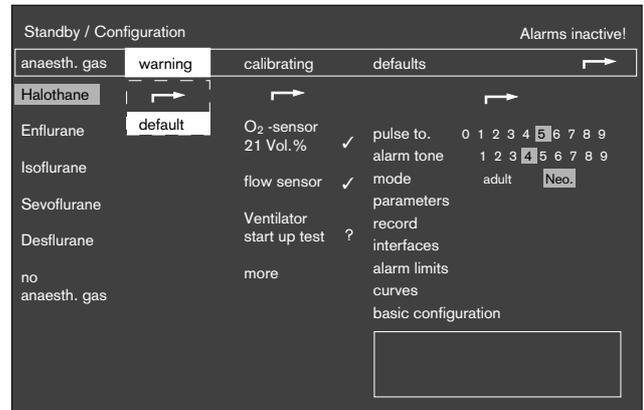

Anaesthetic agent

The anaesthetic vaporizer is automatically recognized by the Cato on account of an optical code. However, sevoflurane and desflurane vaporizers are only recognized by equipment built after July 1994 or which has been converted by DrägerService. The anaesthetic agent concerned is then highlighted by a grey background.


The machine can only recognize the Vapor, but not the anaesthetic agent!

The user is responsible for ensuring that the correct agent is used!

If the anaesthetic vaporizer does not feature an optical code which can be read electronically (older models can be upgraded by DrägerService on request), the anaesthetic agent must be selected by hand:

- Move the cursor frame to the required anaesthetic agent with the aid of the rotary control.
- Press the rotary control to confirm.

The stored trend for the anaesthetic agent concentration is automatically erased when changing the anaesthetic vaporizer!

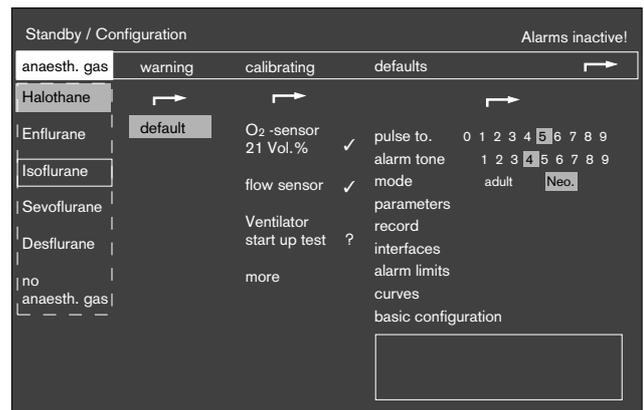

Monitor functions during operation

Changing to measuring mode from »standby«


- The monitor automatically changes to measuring mode when »standby« mode is deactivated on the ventilator or a ventilation pattern is set (»IPPV«, »SIMV«) or »MAN/SPONT«.

or

- Press »standby«  on the monitor; the yellow LED goes out.

- The **standard screen** with the basic parameters of importance for anaesthesia ventilation is displayed on the monitor.


The **data screen** and the **trend screen with zoom function** can also be selected. These screens are described below.

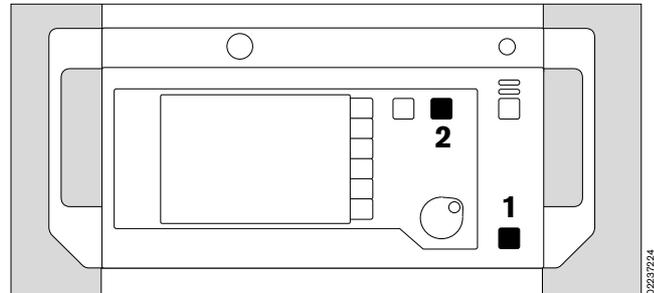

Select standard screen

The standard screen can be selected at any time:


- Press  on the monitor.

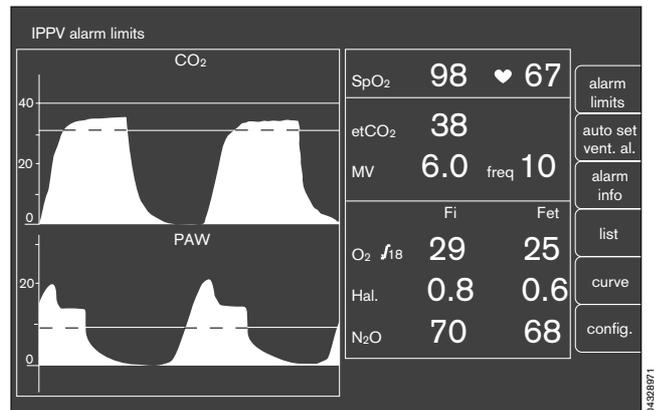

Select data screen


to display all measured values.

- Press  (repeatedly) until the **data screen** is displayed.

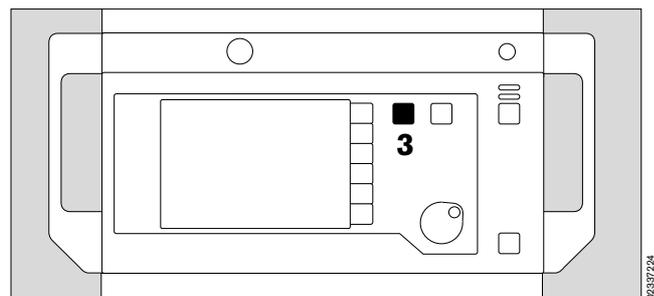

- Display (example):

The airway pressure P_{aw} is continuously indicated by the bar graph on the left-hand side of the screen.

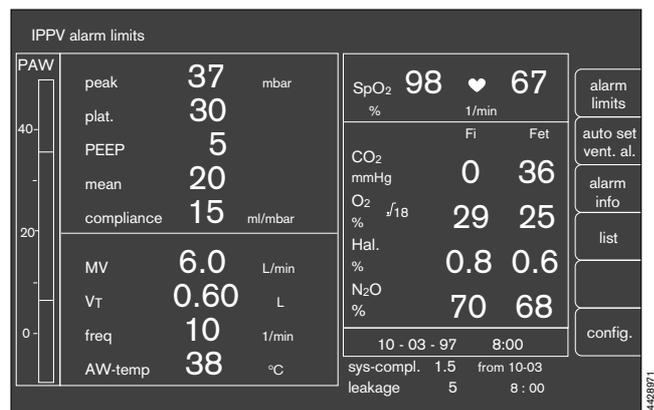

Select trend screen with zoom function


This screen presents the development of measured values over time since measurement commenced.

Values can be saved for a maximum of eight hours.

The following combinations can be displayed:

- CO₂ / AMV
- AGas / N₂O
- O₂ / Compliance
- SpO₂ / Pulse rate


- 1 Press  (repeatedly) until the trend screen is displayed.

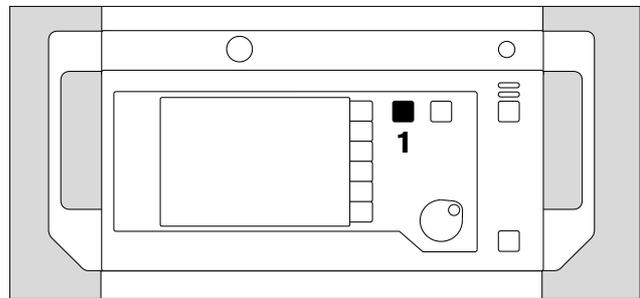

Example:

(The bottom softkey »full trend« is only available here)

The zoom function can also be called up when the equipment has been in operation for more than 30 minutes. This function makes it possible to magnify part of the time range (possibly several times). The magnified section is marked by a dashed border. The past is on the left.

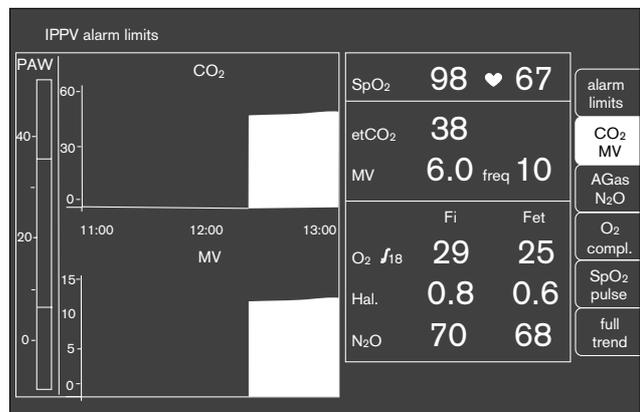


Example:

- Turn rotary knob – the dashed area moves until it includes the interval of interest.
- Press rotary knob – the dashed area is enlarged to the full display width.

When the monitor has been in operation for a sufficiently long time, a new dashed area appears and can be extended as described above.


The maximum has been reached when no new dashed border appears.

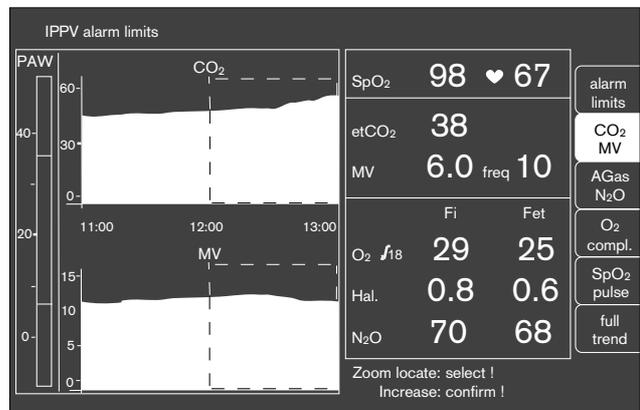

Show other trend combinations


- Press the corresponding softkey, in this case »Agas/N₂O«; the combination is then set against a light background and the new trend curves appear on the screen.

The softkey remains blank if a measuring function (e.g. SpO₂ measurement) is not available.

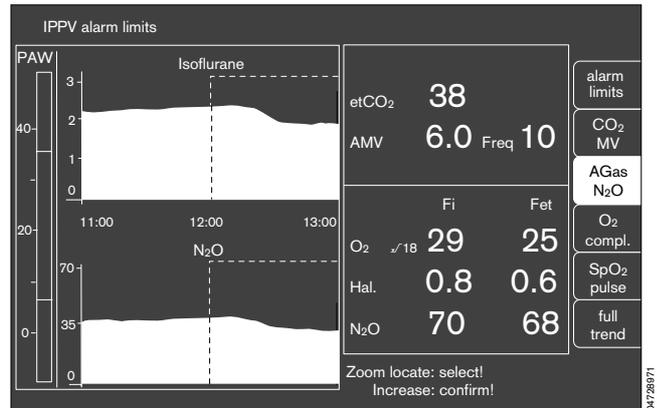

Return to full trend

- Press the softkey »full trend« in order to restore the complete trend display.


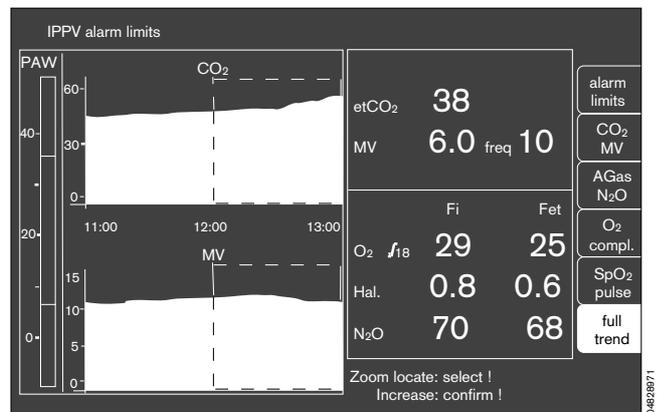


Erase trend memory

The trend memory and list are erased together! This is only possible in »standby« mode!

- Press  on the monitor in order to switch to »standby«. This is only possible if the ventilator is already on »standby«!
 - Press the softkey »erase trend«.
- The system enquires again whether the trend is indeed to be erased.
- Press the softkey »erase« to confirm.
- The former screen is redisplayed without changes if the softkey »Do not erase« is pressed.

The saved anaesthetic concentration trend is erased when the anaesthetic vaporizer is changed!

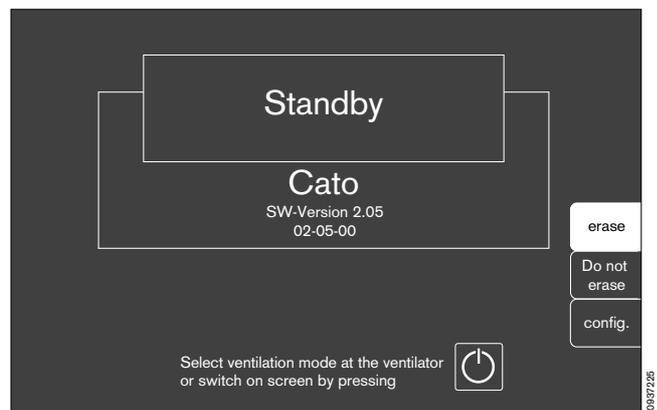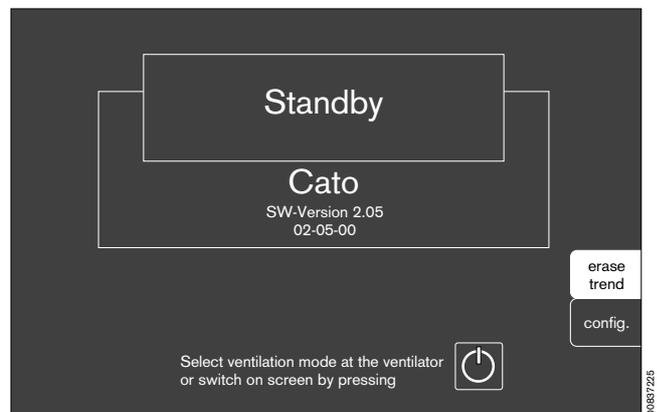


Softkeys

Displaying and setting alarm limits during operation

This can be done in all screens.

- So that measured values can be displayed together with their associated alarm limits.
- So that alarm limits can be corrected in line with changed values.
- Invoked via the softkey »alarm limits«.

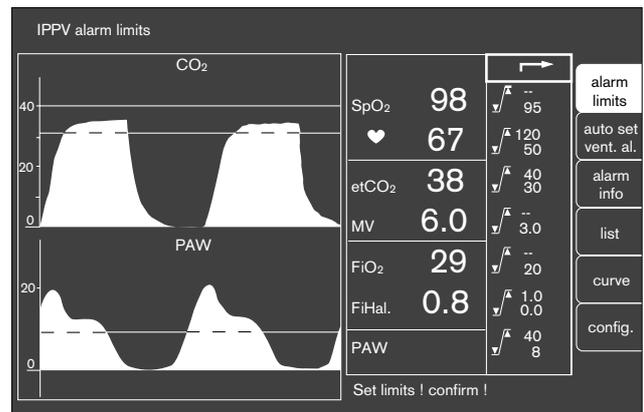

The display comprises the designation of the measured value (etCO₂ in this example), the value actually measured on the patient in large numerals (38 in this example) and the set upper and lower alarm limits as small numerals following the symbol for alarm limits ($\sqrt{\wedge}$).

A deactivated alarm limit is indicated by two dashes in the numbers field.

The set limit values only apply temporarily! They are overwritten by the standard alarm limits when


- the Cato is switched off;
- the setting »default« is selected and confirmed under »warning«.

To change alarm limits:

- Move the cursor frame to the value for the required alarm limit with the rotary control and press to confirm.
- The value can then be altered by turning the rotary control until the required value has been obtained.
- The value is confirmed by pressing the rotary control – this value now represents the active limit.

The active alarm limits are shown as a dashed line in the graphs displayed on the monitor.

See page 65 for information on adjusting alarm limits.

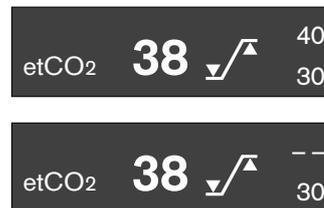

Activate and deactivate CO₂ alarms

In »MAN/SPONT« monitoring mode, the alarm limits for inspiratory and expiratory CO₂ are deactivated; only the CO₂-apnoea alarm is active. This alarm can also be deactivated by pressing the »CO₂ al. on/off« key.

In this case, there will be no CO₂ monitoring at all.

The CO₂-apnoea alarm can be reactivated by pressing the key again.

The CO₂ alarms are automatically activated on changing to »IPPV« or »SIMV« monitoring mode with a ventilation rate above 6 breaths/min.

The alarms remain deactivated when changing to »HLM« mode or »SIMV« with a ventilation rate of up to 6 breaths per minute.

When the CO₂ alarm is activated, the inspiratory CO₂ concentration of the breathing air is monitored with an invariable upper limit of 5 mmHg.



Auto set/Ventilation alarm

Prerequisite:

The ventilator must be in »IPPV« mode

The ventilation alarms with new alarm limits can be automatically adjusted to the current ventilator settings by pressing the softkey »auto set vent. al.«.



The alarm limits previously set in the »limits« menu are erased and cannot be reactivated! See also »Alarm concept« on page 65.

The new tolerance ranges generated for the current measured values depend on the parameter in question:

Parameter	New measured value tolerance		Unit
	Lower	Upper	
Paw	PEEP + (P _{max} - PEEP) / 4	P _{max} + 10	mbar
AMV	VT * f * 0.6 but always: over 0.5	VT * f * 1.4 over 2.0	L/min


The default alarm limits are not affected by Auto set and can be reactivated at any time via the functions »config«, »warning« and »defaults«.

Deactivated alarm limits (– –) are not activated by Auto set!

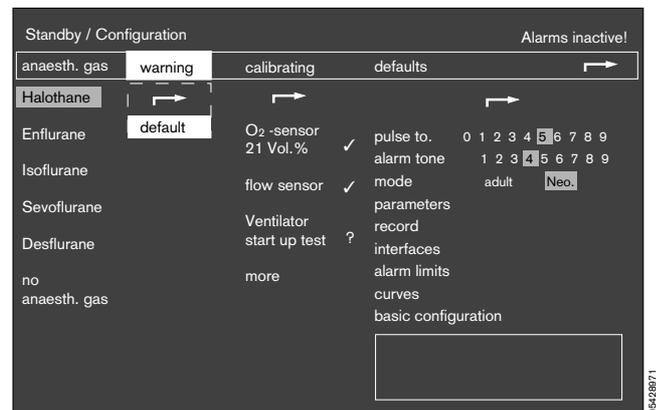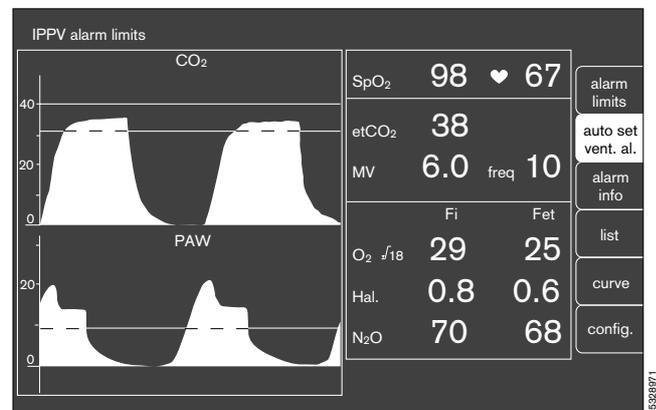

Alarm information

- The measured value field is erased and all active alarms listed when this softkey is pressed.
The display can only be viewed as long as the key is pressed.

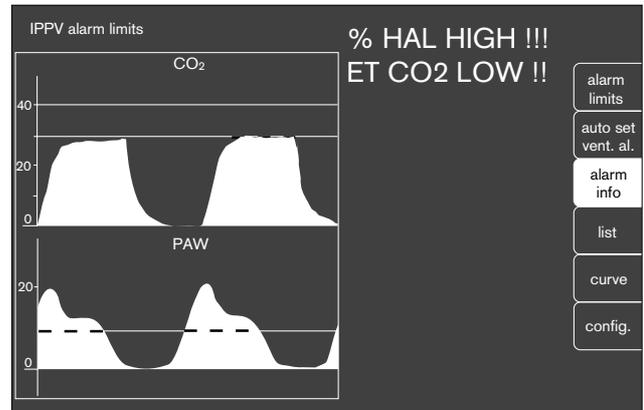


Select list screen

For retrospective documentation of former measured values and alarms. Generation of list entries is described in the section »record« on page 53.

On the standard screen,

- press the softkey »list«.

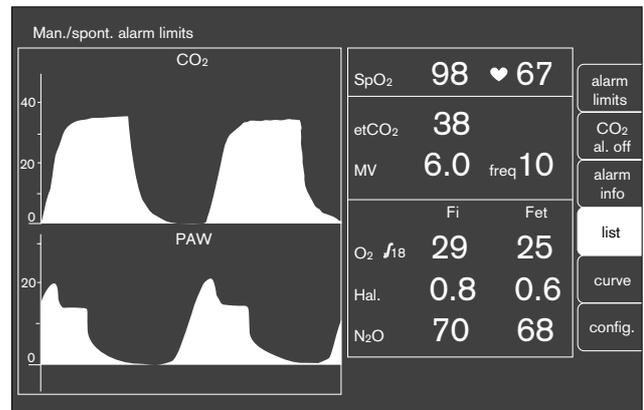

- The display changes and the list appears on the screen (example):

Return to previous page:

- Select »previous page« and confirm.

Move to next page:

- Select »next page« and confirm.

Erase list:

- Press »standby« key  and softkey »erase trend«. Press new softkey »erase« in response to system enquiry.

IPPV-alarm limits

next page previous page

warning	time	HR/pulse	NIBP/mmHg sys/m/dia	SpO ₂	etCO ₂	MV	O ₂ Fi/Fet	AGas Fi/Fet	PAW peak/Peep
% HAL HIGH !!!	12:13	65		99	40	6.0	30/26	1.6/0.6	33/10
% O ₂ LOW !!!	12:15	65		98	38	5.9	25/20	0.8/0.7	30/10
	12:16	66		99	40	6.1	30/26	0.8/0.7	30/10

05/2/8/71

Curve selection


Only possible in the standard screen!

A second curve for the bottom half of the screen can be selected from the menu presented here to complement the CO₂ concentration curve (CO₂) which is always displayed.

The following can be selected:

- **PAW** Airway pressure
- **flow** Expiratory flow
- **Volumeter** Display showing the minute ventilation plus Paw and VT as a bar graph. See below.
- **Pleth.** Plethysmogram (optional)
- **O₂** Oxygen concentration of the breathing gas (optional).

- Press the softkey »**curve**«. A selection menu appears. The option with grey background is currently set.
- Use the rotary control to move the cursor frame to the designation of the required curve.
- Press rotary control. The selected curve is shown on the screen.

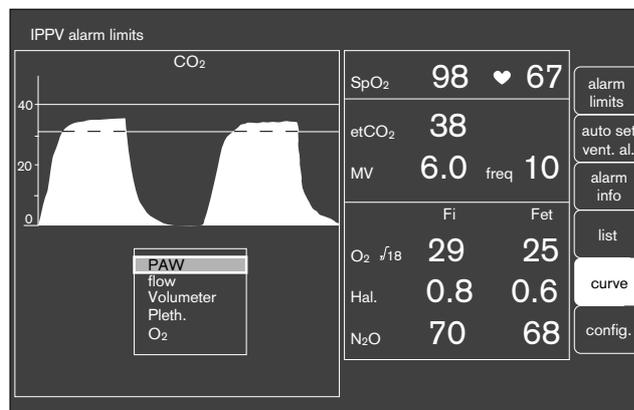

Volumeter function

Depending on mode, the length of the bar represents a maximum measured value:

Mode	Tidal volume V _T	Minute ventilation
Neonate	0.2 L	2 L/min
Adult	1.0 L	10 L/min

Pressure indication:

- The airway pressure is shown in the upper bar graph.

Tidal volumeter:

- The expiratory volume of a breath (V_T) is indicated graphically by the middle bar and numerically on the left.


Minute volumeter:

- The minute volume represented by the bottom bar runs for 60 seconds, during which the tidal volume is summed.

The time expired is shown in seconds above the bar, the total volume appearing on the left. The individual breaths are separated by segments in the bar graph. The volumeter automatically stops and outputs an advisory tone after 60 seconds. The measured values are displayed for 4 minutes and are then erased.

To start volumeter:

- Press rotary control. If the rotary control is pressed again during the 60 seconds, the values will be erased and the volumeter restarted.

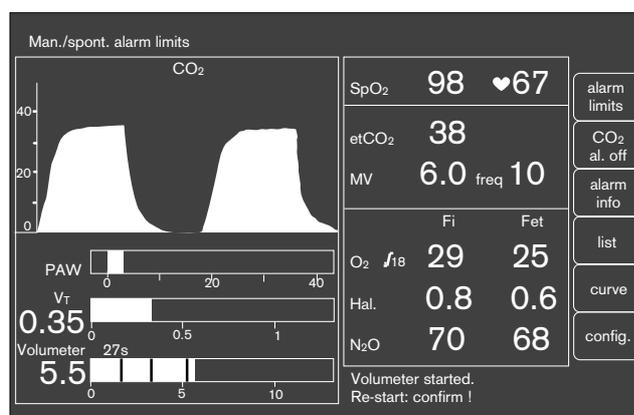

Configuration during operation

The menu displayed during operation shows the functions required for measuring mode. The functions are selected as described for other menus. The settings made here only apply for the present operation.

- Press the softkey »**config.**«

The configuration menu is displayed. The values shown against a grey background currently apply.

- Turn the rotary control to move the cursor frame as in the standby configuration menu. Refer to the description of that menu for further details (page 51 onwards).

set

- **pulse tone** For setting the volume.
- **alarm tone** For setting the volume.
- **mode** Change over between adult and neonate modes.
- **parameters** For setting the parameters, see page 53.
- **record** For determining which event triggers an entry in the record list or printout on the printer, see page 53.
- **activate defaults** Activates the configured default settings (see page 51 onwards).



calibrating

- **calibration** O₂ and flow sensor, as well as others.

warning

- **default** Activates all default alarm limits configured in »**standby**«.
- **CO₂** Activates/deactivates all CO₂ alarms (including »Apnoea«).
- **SpO₂** Activates/deactivates all SpO₂ alarms.
- **HLM mode** Activates/deactivates the heart/lung machine monitoring mode. See page 68.
- **P_{max}**
 - »**on**« Ventilator pressure limitation generates a caution message.
 - »**off**« Ventilator pressure limitation generates an advisory message.

The »**off**« setting is recommended for ventilation with intentional pressure limitation.
- **anaesth. gas** see »Configuring monitor functions in « standby »« on page 56.

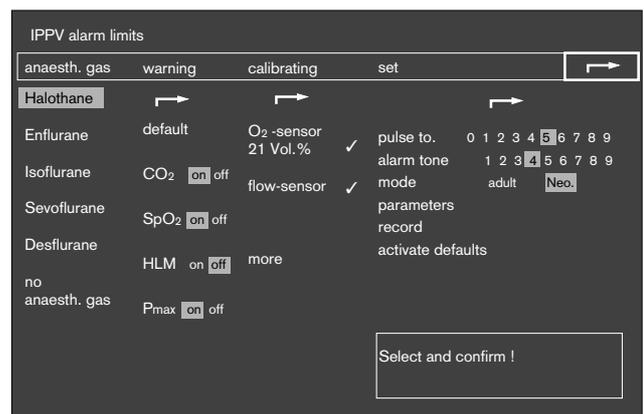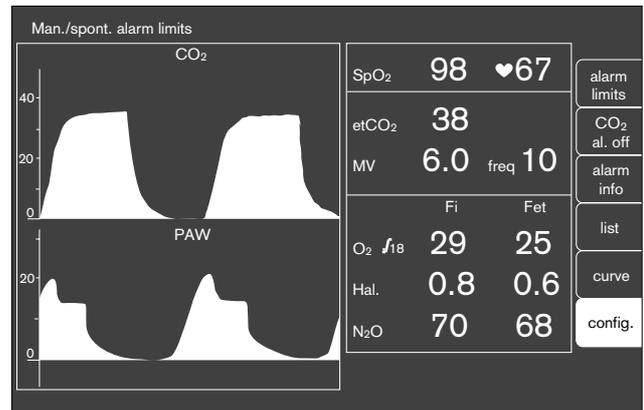

Alarm concept

Alarm priority

The monitor alarms are combined with those of the ventilator, arranged in order of priority and assigned to certain tones or tone sequences.

This information is presented in keeping with the situation, in separate fields for:

Warning marked »!!!«

Caution marked »!!«

Advisory marked »!«

Warnings are accompanied by a continuous tone, caution messages by a tone at 30 second intervals and advisory messages by a single tone. All tones are output at the same time as the messages appear in the displays.

European Standard EN 740 stipulates use of EN tones. Alternatively there are tones in keeping with Dräger conventions available.

Whenever a warning or caution message occurs, the corresponding LED lights up above the  key:

- 1 Warning Red, flashing
- 2 Caution Yellow, flashing
- 2 Advisory Yellow, continuous

The summary of alarm limit settings appears on the screen. The parameters which have been exceeded are shown in inverted form.

Example:

The lower alarm limit for etCO₂ is set at 30 but the actual value is lower. A value of 28 is measured.



- The settings menu appears automatically. The violated alarm limit is shown in inverted form.
- The symbol for the lower alarm limit  and the parameter designation »etCO₂« appear in the advisory field for caution messages.
- The yellow LED lights up.
- The corresponding tone is heard.

The value can be altered by turning the rotary control and confirmed by pressing the control.

The alarm limit can be confirmed directly if it is to remain unchanged. The limit value menu is always closed whenever a value is confirmed.

If several alarms occur simultaneously, the alarm limits which have been exceeded are shown against a grey background. When the first alarm has been confirmed (by pressing the rotary control on the monitor), the alarm limit of the alarm with the next lower priority is activated and displayed.


All alarms received are recorded for the anaesthesia record if list entry has been configured accordingly.

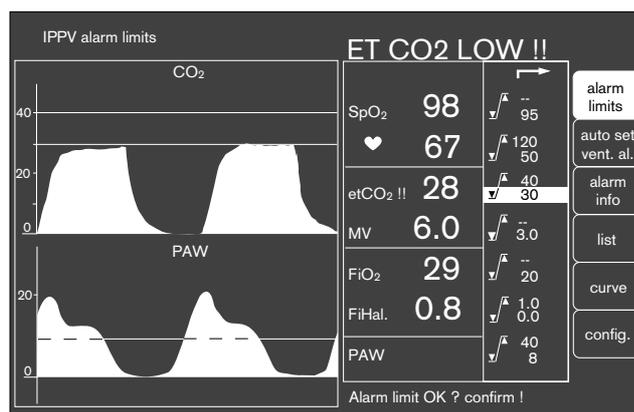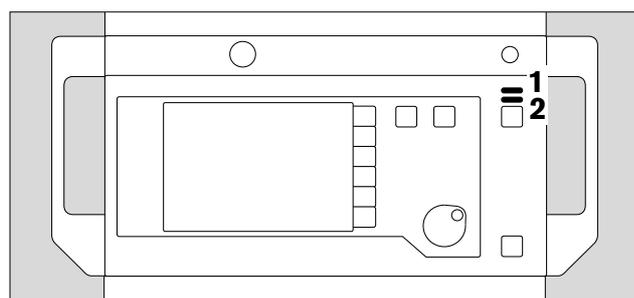

Monitor functions
 Alarm concept
 Alarm priority
 CO₂ alarm on/off

Displaying all alarms

- Press the softkey »alarm info« and keep pressed. The alarm texts of all warning, caution and advisory messages are displayed in order of priority.
- Release the softkey »alarm info«. The alarm texts disappear.

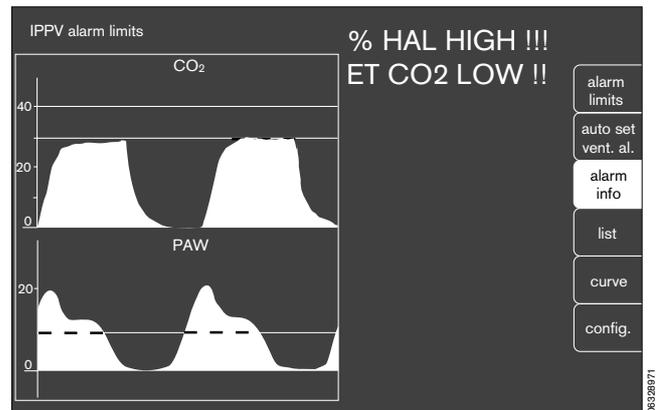


Suppressing the alarm tone

- 1 Press  on the monitor for which the yellow LED lights up. The alarm tone is then suppressed for **two minutes**. Any new messages occurring during this time, however, are indicated once with their appropriate specific tone.
- 2 The red (upper) or yellow (lower) LED continues to light up and the text remains on display.

To reactivate the alarm tone during the 2 minutes:

- 1 Press  again and the corresponding yellow LED goes out.

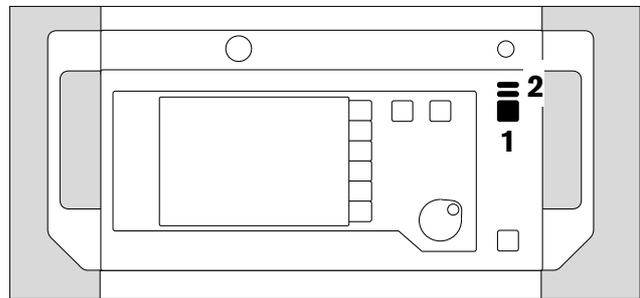


CO₂ alarm on/off

The CO₂ alarms can be deactivated by the »CO₂ al. on/off« key.

In this case, CO₂ is not monitored.

Press the key again to reactivate the alarm.

When the CO₂ alarm is on, the inspiratory CO₂ concentration in the breathing air is monitored with a fixed maximum limit of 5 mmHg!

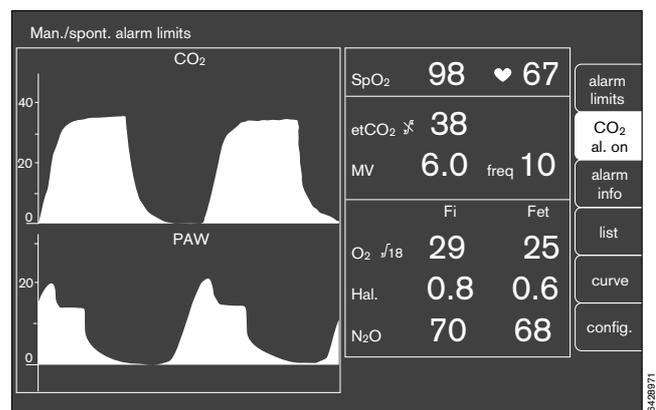

Adjusting to the ventilation mode

For »IPPV«, »MAN/SPONT« and »SIMV« ventilation modes

The alarm limits are switched over automatically with the change of ventilation mode.

In »IPPV« and »SIMV« modes, the »IPPV alarm limits« alarm mode is automatically activated on the monitor if the ventilation rate is equal to or greater than 6 breaths per minute.

The MAN/SPONT alarm limits are automatically activated for »MAN/SPONT« and the SIMV alarm limits for »SIMV« mode with a ventilation frequency of less than 6 breaths per minute.

The currently active alarm mode at any time is displayed in the status field and flashes for the first five seconds.

The full range of alarms (»IPPV alarm limits« alarm mode) can be reactivated by selecting IPPV mode or by setting the ventilation frequency to a value equal to or greater than 6 breaths per minute.

Alarm limits in ventilation mode			MAN/SPONT	SIMV with fIMV < 6	SIMV with fIMV ≥ 6 and IPPV	MAN/SPONT HLM	IPPV/SIMV HLM
Minute ventilation	AMV	<input checked="" type="checkbox"/>	off	on	on	off	on
			off	on	on	off	on
Insp. anaesthetic gas		<input checked="" type="checkbox"/>	on	on	on	on	on
			off	on	on	off	on
Airway pressure	Paw	<input checked="" type="checkbox"/>	on	on	on	on	on
			off	on	on	off	on
Exsp. CO ₂ concentration	et CO ₂	<input checked="" type="checkbox"/>	on	on	on	on	on
			on	on	on	on	on
Insp. CO ₂ concentration	in CO ₂	<input checked="" type="checkbox"/>	on	on	on	on	on
Insp. O ₂ concentration	FiO ₂	<input checked="" type="checkbox"/>	off	on	on	off	on
			on	on	on	on	on
Apnoea pressure	} alarm activated after		off	30 s	15 s	off	off
Apnoea flow			off	30 s	15 s	off	off
Apnoea CO ₂			1 min.	30 s	30 s	off	off
Functional oxygen saturation	SpO ₂	<input checked="" type="checkbox"/>	on	on	on	off	off
			on	on	on	off	off
	HR	<input checked="" type="checkbox"/>	on	on	on	off	off
			on	on	on	off	off

Heart/lung mode (HLM)

A special alarm mode can be activated during operation for the function »warning« in the »Configuration« menu for operation with a heart/lung machine (»HLM« mode).


All apnoea alarms are deactivated in this mode and the gas values are displayed continuously without connection to the respiratory phase.

SpO₂ alarms are deactivated in HLM mode. They are automatically reactivated when pulsations are detected again after ending HLM mode.

In this alarm mode, the parameters are monitored with the same limits as are set in the ventilation mode from which »HLM« was selected.

Heart/lung mode can be deactivated either

- via the »Configuration« menu or
- via the softkey »HLM off«.

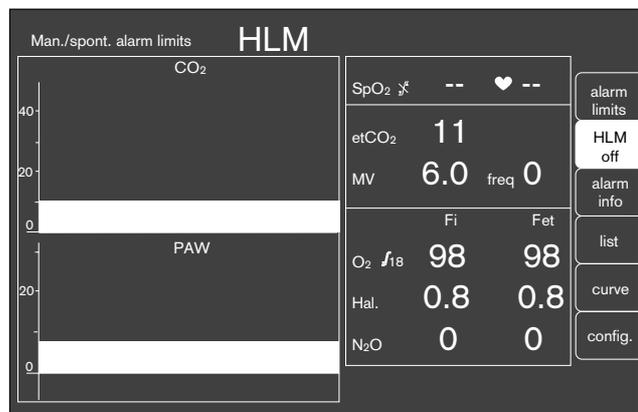

Messages, cause and remedy

Contents

	Page
Where messages occur	72
Location of valves and subsystems	73
Messages on the monitor	74
Messages on the ventilator	78


Where messages occur

Warning, caution and advisory messages

These are the three types of message output by the system and displayed in the alarm field. They are classified in order of priority and listed alphabetically in the following tables.

User prompts


appear in the bottom right-hand corner of the screen.

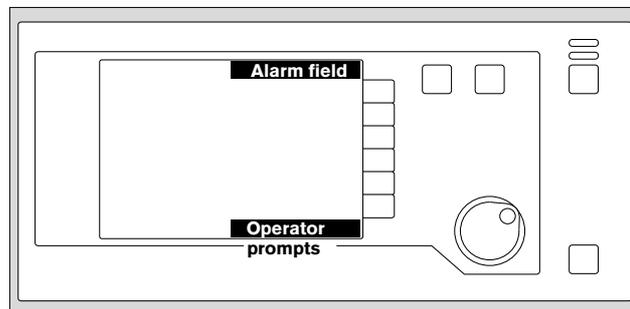

Messages on the ventilator

These are always output during operation. Refer to the chapter entitled »Anaesthesia ventilation«.

In exceptional cases, messages may also appear in the dialogue field during operation and the self-test. Refer to the table on page 78.

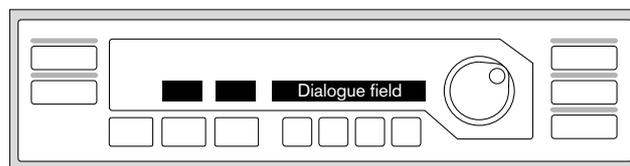

Malfunctions and system faults

Eliminate the cause where possible as indicated by the user prompt on the screen. Inform DrägerService if necessary, specifying the malfunction number and software number.


Call DrägerService if the malfunctions and/or advisory messages cannot be remedied in accordance with the descriptions or by repeatedly switching off and on again (with a delay of approx. 5 seconds in between) via the master power switch and repeating the self-test!

All malfunctions are signalled acoustically.


In the event of particularly serious problems, e.g. »Control pressure low«, it is possible that when manual ventilation is attempted no pressure will be generated in the breathing system. Simultaneous failure of the compressed air and compressed oxygen supply would be such a serious fault.

In these cases, immediately ventilate the patient with the separate emergency breathing bag.

Location of the subsystems:

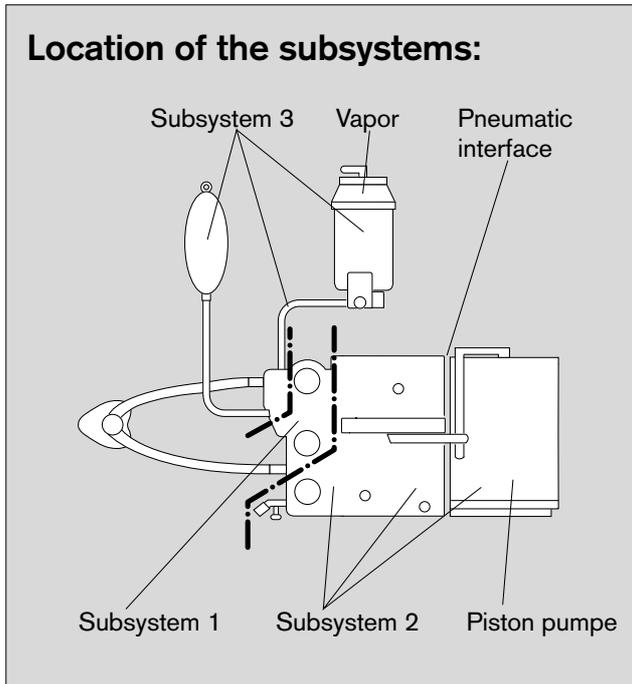


Location of valves in the breathing system:

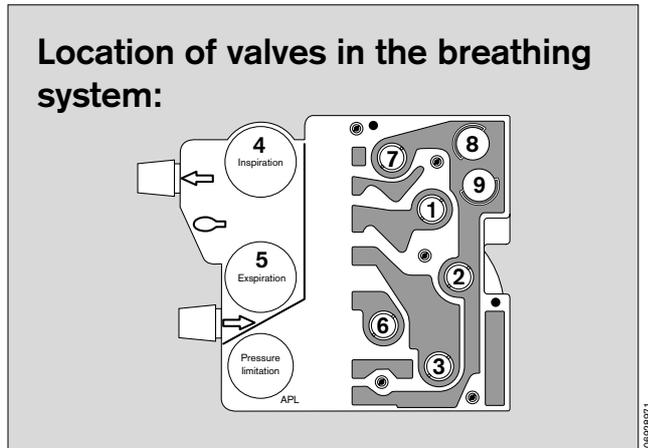

The valves are numbered as shown in the following schematic – V1, V2, ...

Simplified pneumatic schematic showing valves and subsystems:

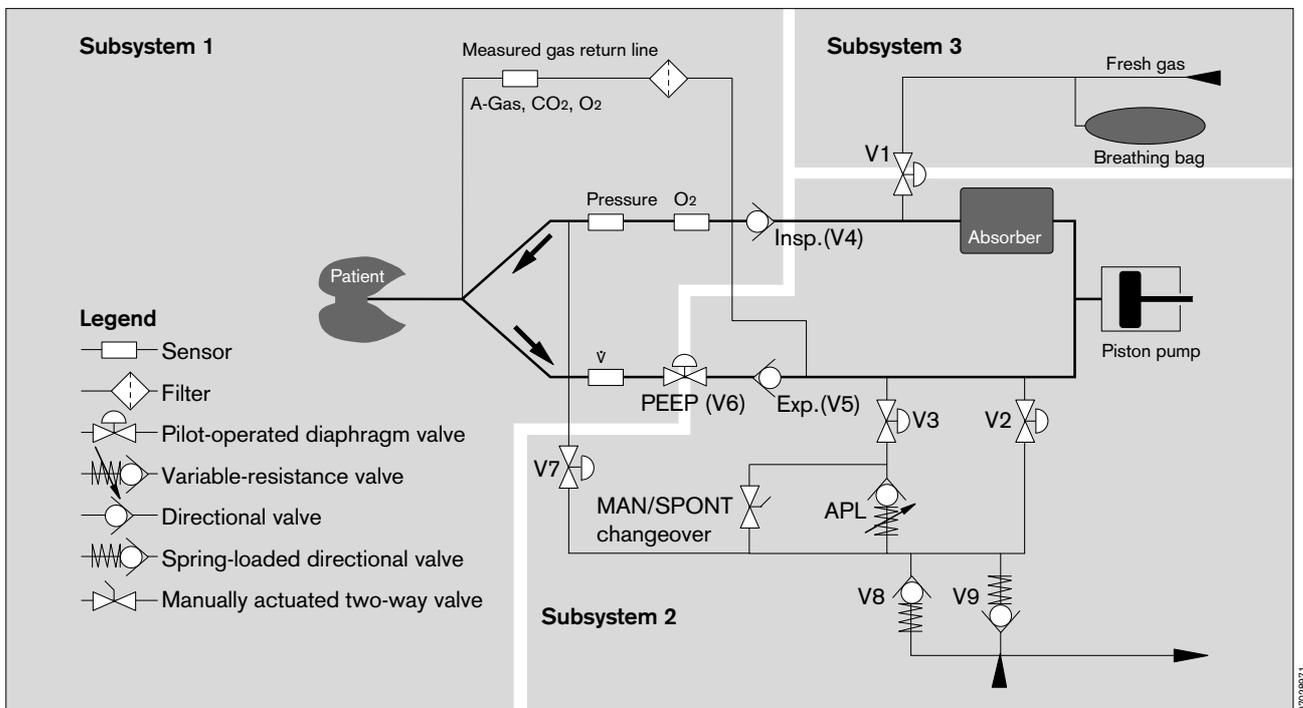


Messages on the monitor

Alarm messages are assigned to three priority classes (alarm priority) on the monitor of the Cato:


- Warning !!!
- Caution !!
- Advisory !


The patient's condition must be checked first, before the machine is examined on account of a possible measuring error!

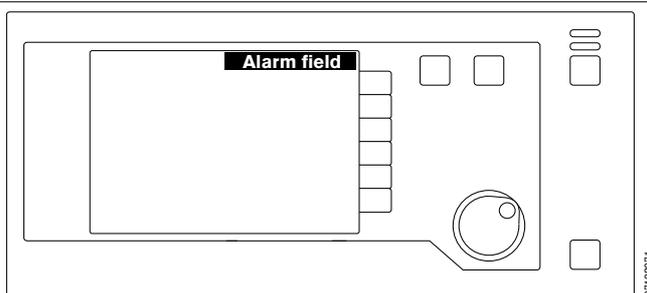


07128871

Messages	Cause	Remedy
% HAL HIGH % ISO HIGH % ENF HIGH % DES HIGH % SEV HIGH	!!! !!! !!! !!! !!! The inspiratory anaesthetic concentration exceeds the upper alarm limit in each case. The upper alarm limit has been exceeded for at least two breaths.	Check setting of anaesthetic vaporizer.
% HAL LOW % ISO LOW % ENF LOW % DES LOW % SEV LOW	!! !! !! !! !! Applicable inspiratory anaesthetic concentration below lower alarm limit. Value below lower alarm limit for at least two breaths.	Check setting of anaesthetic vaporizer.
% O2 ERR	! Sensor for inspiratory O2 measurement incorrectly calibrated? Sensor changed and/or not calibrated?	Calibrate sensor.
	Sensor exhausted? Sensor not plugged in? Sensor lead defective? Dräger sensor cell not used?	Replace and calibrate cell. Plug in sensor connector. Replace sensor housing. Use original sensor cell.
% O2 HIGH	!! Inspiratory O2 concentration exceeds upper alarm limit. O2 flush used?	Check O2 concentration in fresh gas flow.
% O2 LOW	!!! The inspiratory O2 concentration is below the lower alarm limit.	Check O2 supply. Check setting on O2 flow meter.
AGT ERR	! Anaesthetic gas measurement faulty.	Call DrägerService.
AGT NOT SEL	! Anaesthetic agent not selected or selection differs from that detected by Vapor identification.	Check and select anaesthetic agent.
APNOEA CO2	!!! Breathing/ventilation at a standstill. No breath detected by expiratory CO2 for 30 seconds. (No breath has been detected for 60 seconds in alarm mode »Man/Spont alarm limits«.)	Patient must immediately be ventilated by hand! Check patient's spontaneous breathing ability. Check hoses, tube and ventilator.

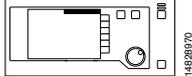Messages on the monitor			
Messages	Cause	Remedy	
APNOEA PRES	!!! Ventilation at a standstill. No change of pressure detected for 15 seconds; 30 seconds in the case of SIMV alarm limits. Inadequate supply of fresh gas. Leak in hose system.	Patient must immediately be ventilated by hand! Check fresh gas setting on anaesthetic unit. Check hoses and tube. Check ventilator.	
APNOEA VOL	!!! Ventilation at a standstill. No expiratory tidal volume for 15 seconds; 30 seconds in the case of SIMV alarm limits. Inadequate supply of fresh gas. Tube kinked. Leak in hose system.	Patient must immediately be ventilated by hand! Check ventilator. Check fresh gas setting on anaesthetic unit. Check tube and hose system.	
AW-TEMP HIGH	!!! Inspiratory breathing gas temperature over 40 °C.	Disconnect breathing gas humidifier. Set lower heating level when temperature has dropped to 37 °C.	
CO2 ERR	! Sidestream measurement is faulty.	Call DrägerService.	
CO2 LINE BLK	! Sample line blocked.	Replace sample line and water trap if necessary. Check sample line is not kinked. Examine measuring gas return line for stenosis, replace microbial filter if necessary.	
CO2 OFF ($\sqrt{\wedge}$)	! CO2 alarm deactivated.		
COM VENT ERR	!!! Communication between ventilator and monitor interrupted or faulty.	Alarm limits do not change over automatically. Set alarm limits by hand if necessary. Call DrägerService. Valve discs in the breathing system defective. Set the sampling rate to 200 mL/min due to the high frequency and time constant of the CO2 sensor.	
COOLING 8050 ?	! Temperature inside machine too high.	Clean filter at rear. Call DrägerService.	
ET CO2 HIGH	!! Upper alarm limit for end-expiratory CO2 concentration has been exceeded for at least two breaths.	Check ventilation.	

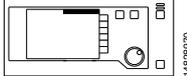Messages on the monitor			
Messages	Cause	Remedy	
ET CO2 LOW	!! End-expiratory CO ₂ concentration below lower alarm limit for at least two breaths.	Check ventilation.	
EXP-VALVE ?	!! Fault in expiration valve. Measured value for minute ventilation may be excessive.	Check expiration valve. This message can be deactivated until the next leak test by pressing the key for muting the audible alarmtone.	
FRESH GAS ?	!! Too little gas in manual ventilation bag.	Increase fresh gas flow. Repair any leaks in hose system.	
INSP CO2 HI	! Inspiratory CO ₂ concentration is above upper alarm limit of 5 mmHg.	Replace the soda lime in the circle system of the anaesthetic workstation.	
MIN VOL HIGH	!! Minute volume exceeds upper alarm limit.	Correct tidal volume or respiration rate on ventilator.	
MIN VOL LOW	!! Minute volume below lower alarm limit. Tube clogged / kinked? Leak in breathing system? Ventilation volume limited by pressure limitation?	Check tube and hoses. Seal breathing system. Correct ventilation pattern.	
N2O ERR	! N ₂ O gas measurement faulty.	Call DrägerService.	
NO SPO2 PULS	!!! No pulse signal detected by SpO ₂ measurement for approx. 10 seconds.	Check SpO ₂ sensor (dropped off? NiBP measurement on same arm?).	
O ₂ ✓ xx %	! The alarm limit appears after the parameter designation if the alarm limit for the inspiratory O ₂ concentration is below 21%.		
PAW HIGH	!!! Airway pressure exceeds the upper alarm limit. Ventilation hose kinked? Stenosis? Pulmonary problem? Cough?	Check patient's condition! Check hose system, tube and fresh gas flow.	
PAW NEGATIVE	!!! Negative airway pressure measured.	Check hose system and tube on ventilator.	
PRESS ERR	! Pressure sensor defective.	Call DrägerService.	
PRESS EXP HI	!!! End-expiratory pressure more than 10 mbar over PEEP.	Extend expiration period. Check hose system and microbial filter. Empty water traps.	

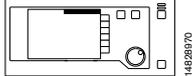Messages on the monitor			
Messages	Cause	Remedy	
PRESSURE LTD	!!, (!) Ventilator operating with pressure limitation. Change in lung compliance? Tube kinked? Microbial filter in inspiratory line soiled?	Check tube/filter. Increase Pmax or decrease VT if necessary. If pressure-limited ventilation is intentional: Press softkey »Config«, select »Alarms« and switch »Pmax« »off«.	
RS232COM ERR	! RS 232 communication interrupted.	Check plug connection.	
SPEAKER FAIL	! No alarm sound, loudspeaker defective.	Call DrägerService.	
SPO2 ERR	! SpO2 measurement faulty.	Call DrägerService.	
SPO2 HIGH	!! Oxygen saturation exceeds set upper alarm limit.	Check O2 concentration in fresh gas. Check ventilator.	
SPO2 LOW	!!! Oxygen saturation is below the set lower alarm limit.	Check O2 concentration of fresh gas. Check ventilation.	
SPO2 OFF (☒)	! SpO2 alarm deactivated.		
SPO2 PULS LO	!!! Pulse rate has dropped below the set alarm limit.	Check patient's condition!	
SPO2 PULSE HI	!! Pulse rate exceeds upper alarm limit.	Check patient's condition! Correct alarm limit if necessary.	
SPO2SEN DISC	! No pulse detected by pulsoxymeter although heart is clearly beating.	Check patient's condition (disturbed circulation?). Check SpO2 sensor positioned correctly.	
VENT ERR	!!! Fault in ventilator pressure sensor. Filter in pressure measuring hose blocked (water absorption!) Control pressure in ventilator too low. Hardware fault in ventilator. Machine not operational!	Patient must immediately be ventilated by hand! If no pressure build-up in the breathing system: immediately ventilate the patient with the separate emergency breathing bag. Check pressure gas supply. Replace filter in pressure measuring hose. If not possible: Switch off machine, call DrägerService.	
VOL ERR	! Flow sensor calibrated incorrectly? Flow sensor / flow sensor lead defective? Flow sensor not plugged in? Flow sensor soiled?	Calibrate sensor. Replace lead / sensor and calibrate sensor. Plug in sensor connector.	

Messages on the ventilator		
		
Messages	Cause	Remedy
????	Internal electronic fault.	Call DrägerService.
compliance test	The hose compliance is being tested during the leakage test in »standby« mode.	
Equipment fault	Attempt to cancel self-test when – a malfunction has been signalled beforehand or – self-test has already been broken off 10 times! (Mechanical ventilation is only possible when self-test has been completed successfully!) This message precedes the message »Malfunction No. XXX«.	Acknowledge to cancel alarm tone. Restart. For information. Note malfunction number and call DrägerService.
GB. Version	Message displayed at the beginning of the self-test.	Software version number is indicated when switching on. Inform DrägerService of version number when reporting malfunctions.
Leak test IPPV	The IPPV leakage rate is being determined during the leakage test in »standby« mode.	
Paed. hoses !	When changing the VT setting from values above 200 mL to less than 200 mL.	Use infant hoses. Run leakage test to measure new compliance, otherwise limited accuracy! Press rotary control to confirm!
piston test	The system occasionally runs a self-test to detect major leaks and faults in the inspiration valve.	
Pressure release	A static excess pressure continued for too long after AutoWakeUp. The system is relieved.	Switch ventilator to »standby«. Disconnect fresh gas.
Restart	Transient system fault. Power supply briefly interrupted.	Manual ventilation is not possible for approx. 10 seconds while the message »Warmstart« is displayed; wait for 10 seconds, the ventilator then continues with the former settings. Check parameter settings. Set up a substitute system if warmstart is repeated. Call DrägerService.
Self-test	Self-test has been started.	
Self-test ... x discount	Message displayed after cancelling the self-test. Only possible 10 times! A complete self-test must then be performed.	Operating mode can be selected. Skipping calibration functions and internal alignment procedures can impair the accuracy of measurements!
Standby	Machine on standby.	
Supply pressure?	Pressure supply inadequate.	Correct pressure supply. Use spare cylinders if necessary.

Care


Contents

	Page
Stripping down machine	80
Dismantling parts	81
Disinfection – cleaning – sterilization	83
Care methods	84
Care schematic	84
Care intervals	84
Disposing of consumable articles	85
Wipe cleaning	85
Disinfectant cleaning in machines	85
Disinfection with formaldehyde vapour	85
Sterilization with ethylene oxide	86
Steam sterilization (121 °C / 134 °C)	86
Tabular summary	87
Disposal of used batteries and O ₂ sensors	88
Disposal of apparatus	88
Reassembling system	89
SpO₂ measurement (optional)	95
Safety and precautions	95
Choice of sensor	95
C-Lock-ECG synchronization (optional)	96
Tips to avoid artefacts	97
Applying sensors	98
Durasensor DS-100 A	98
Oxisensor D-25 and D-20	98
Oxisensor I-20	99
Oxisensor R-15	100

Stripping down machine

Swing up table top to remove piston pump:
(only possible in »standby«)

- 1 Swing lever as far as possible to unlock.
- 2 Hold breathing system by handle and lift out.
Do not damage flat seals – avoid contact with sharp edges. It must be put down so that the sealing elements are not subject to constant pressure!

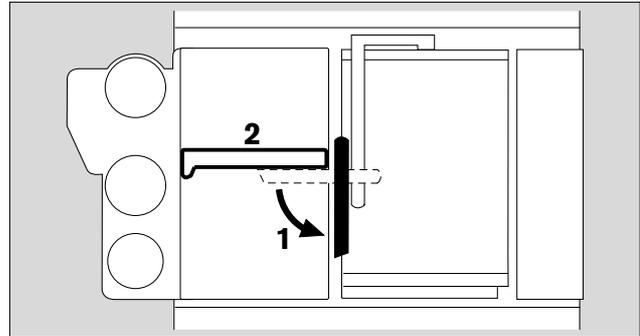


3 Caution!



Temperature of hotplate for breathing system equals approx. 60 °C.


- 4 Handle pointing up – lift out piston pump
– **do not use force** –
Do not damage flat seals – avoid contact with sharp edges. It must be put down so that the sealing elements are not subject to constant pressure!
- 1 Swing lever back to original position, otherwise table top cannot be reclosed.

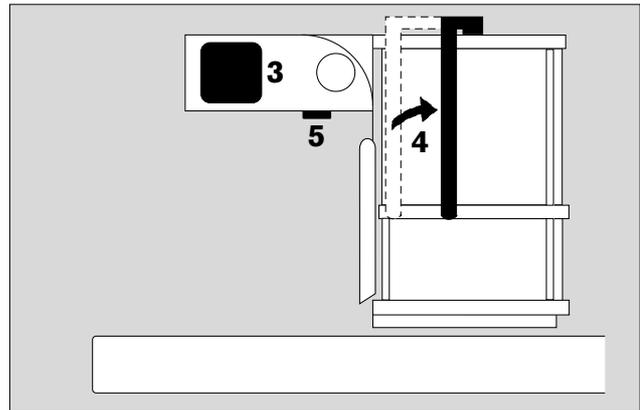

Remove anaesthetic gas scavenging system

Refer to separate Instructions for Use for anaesthetic gas scavenging system.


- Unplug anaesthetic gas scavenging hose from anaesthetic gas scavenging system.
- 5 Press button to unlock hose connector and pull downwards. Hose remains on connector.

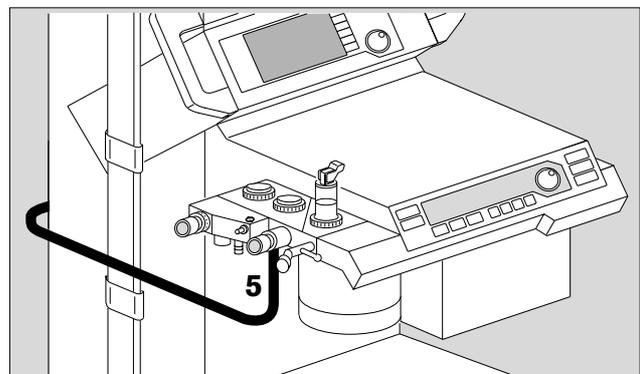

Dismantling parts

T-piece of measured gas hose (if used)


- 1 Remove filter.

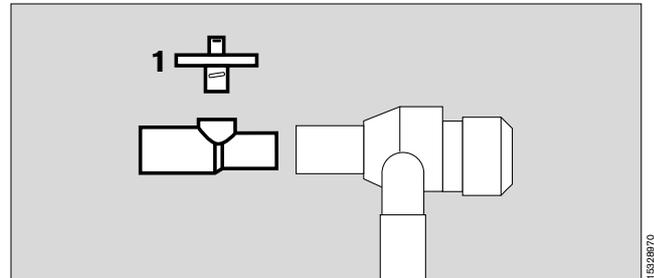

Inspiratory O₂ sensor (if used)


- **Must not be disinfected in liquid or autoclaved!**
- Wipe any dirt off housing or lead with damp disposable cloth.
- Unscrew O₂ sensor and remove O₂ cell. Do not touch wire mesh of O₂ cell, otherwise it loses its water-repellent property and measurement becomes inaccurate.

Dirt on wire mesh of sensor cell is simply wiped off with a disposable cloth moistened with distilled water.

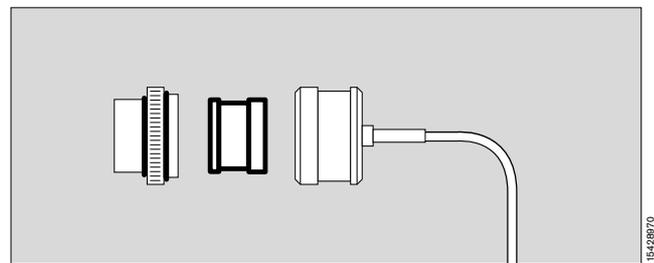

Temperature sensor (breathing gas) (optional)

- Draw temperature sensor out of Y-piece, remove Y-piece from ventilation hoses.
Prise lead out of hose clips.
- Pull connector out of socket.
- Wipe dirt off with damp disposable cloth.

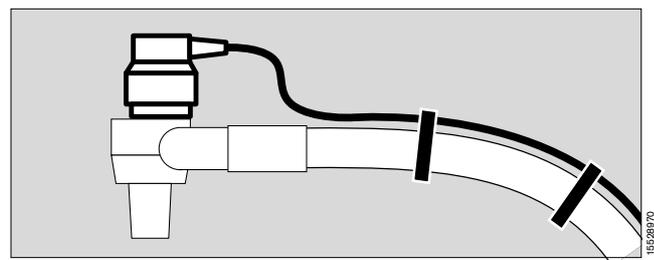


Flow sensor

The flow sensor cannot be disinfected or sterilized.

Contamination by infectious patients: dispose of as directed!


If contamination is unlikely, the flow sensor can be reused as long as it can be calibrated correctly.

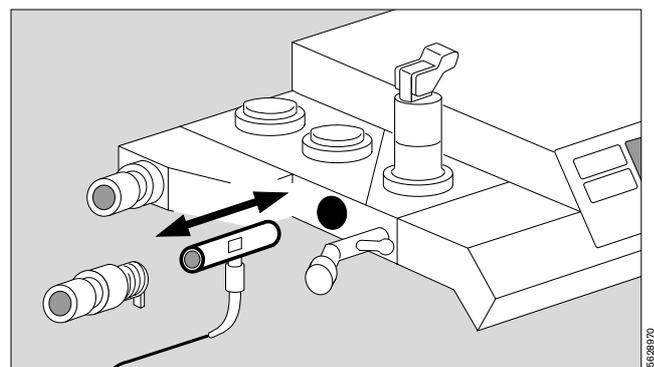


Breathing system

- Remove flow sensor.
- Unscrew pressure limitation valve APL.
- Dismantle breathing valves – valve discs must be handled with care.
- Replace measured gas return hose if contamination is possible.

- 1 Release the five quick-release screws by turning through a quarter-turn with a 6 mm hexagon key. Remove upper part.
- 2 The two valves are only unscrewed if soiled.

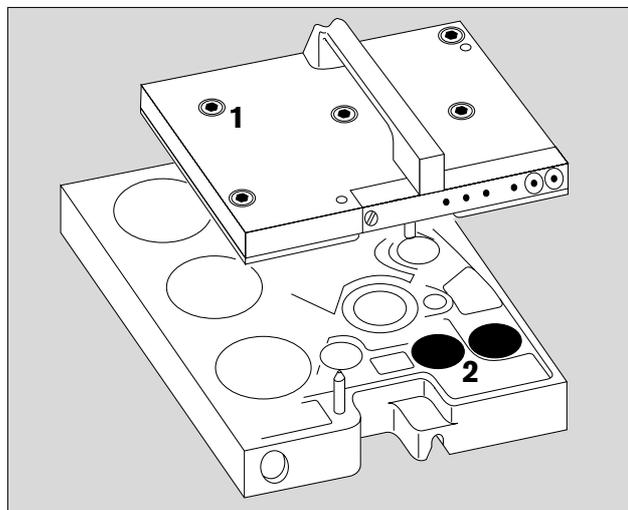


- Empty soda lime container (see page 45). Dispose of with household refuse (but not when treating infectious patients!).

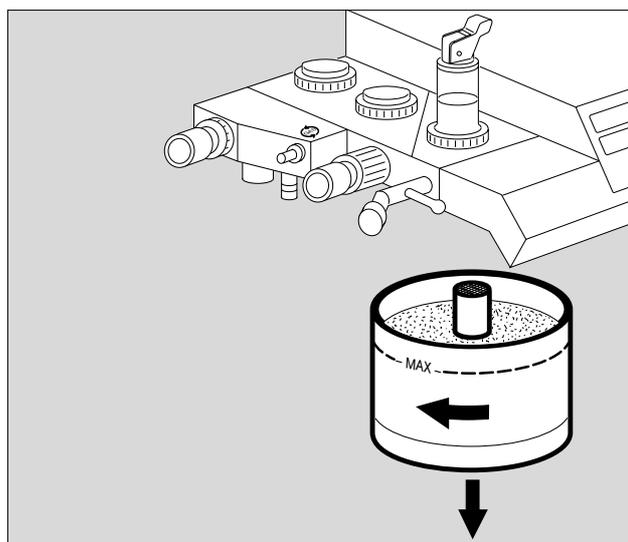

Piston pump

- 3 Release the two quick-release screws by turning through a quarter-turn with a 6 mm hexagon key.
- 4 Remove cylinder head.
- 5 Completely unscrew the central screw on the piston (no quick-release).
- 6 Remove piston and roller diaphragm.

- The remainder of the piston pump does not come in contact with breathing gas and does not require sterilization!

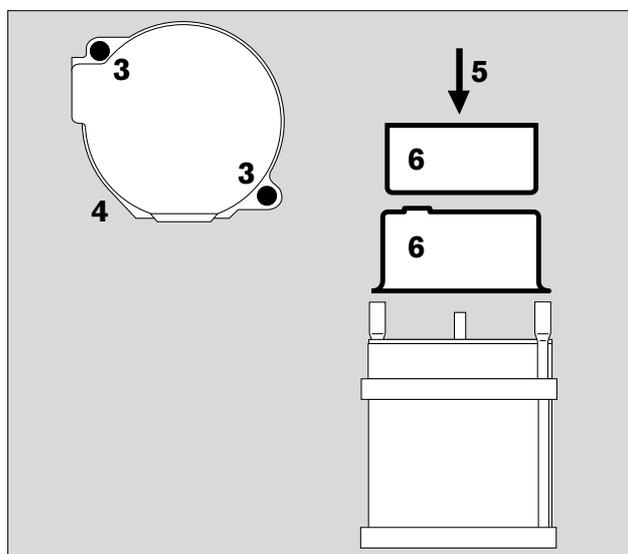

Disinfection – cleaning – sterilization

Agents and methods for disinfection, cleaning and sterilization

Only products from the list of surface disinfectants may be used for disinfection. Products based on

- aldehydes,
- alcohols and
- quaternary ammonium compounds
(e.g. **Buraton 10 F**[®] or **Terralin**[®] diluted as specified. Both are registered trademarks of Messrs. Schülke & Mayr, D-22846 Norderstedt.)

may be used to ensure material compatibility.

The following are not suitable:

- phenols,
- compounds liberating halogen,
- strong organic acids and
- compounds liberating oxygen.


Users in the Federal Republic of Germany are advised to use disinfectants specified in the current list published by the German Society for Hygiene and Microbiology (DGHM). The list also specifies the active agent in each disinfectant.

Users in countries in which the DGHM list is not available are advised to use the agents mentioned above.

Alcohol and agents containing alcohol must not be allowed to seep into the sample line!

Alcohol results in incorrect results when measuring the anaesthetic agent!

Care methods

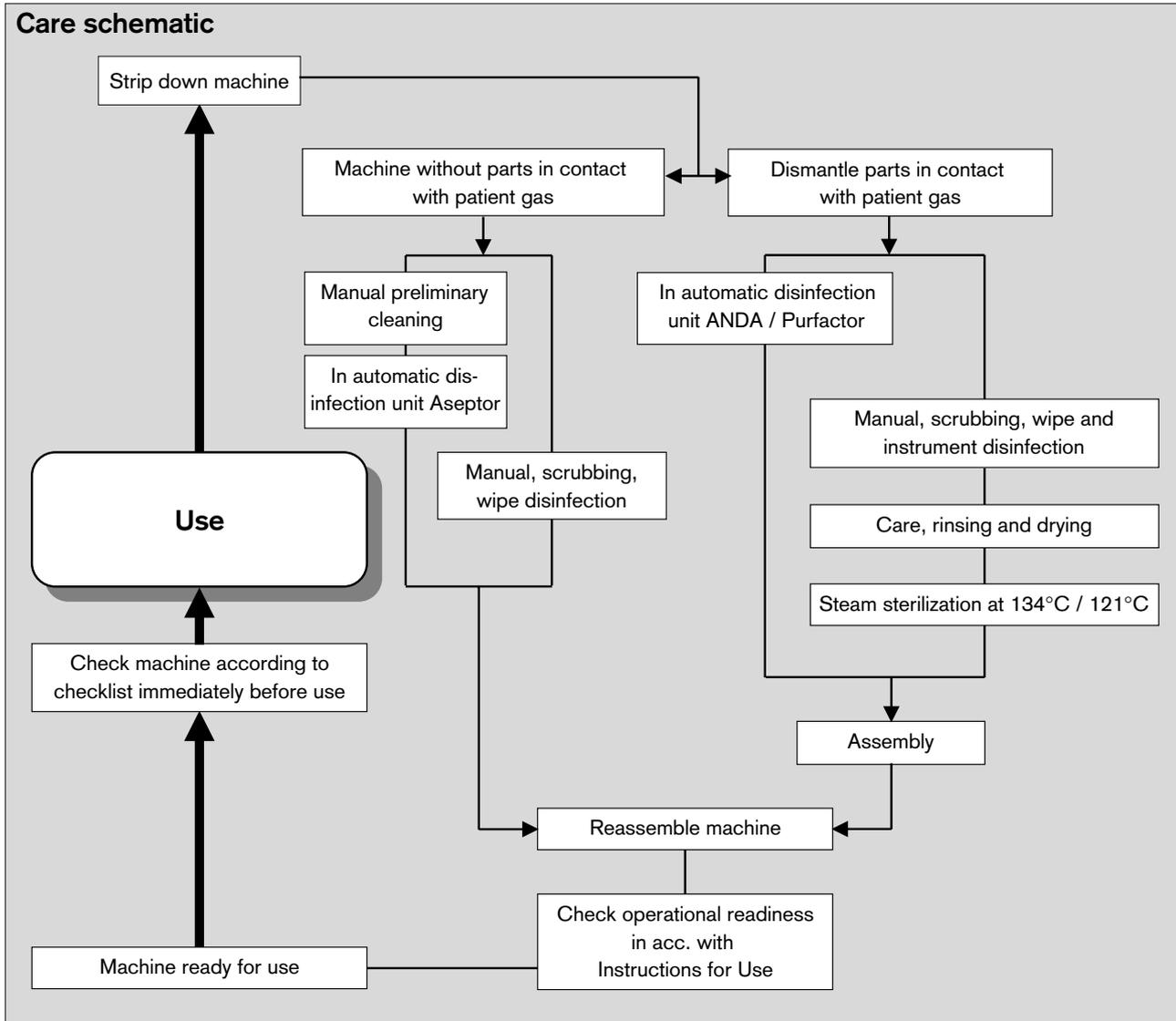

Care intervals

A number of steps can be omitted if microbial filters are used in the inspiratory and expiratory lines.
 The following table is intended as a guide; it does not replace the directions of the hospital's hygiene officer!

Component:	Non-infectious patients		Infectious patients
	with microbial filters:	without microbial filters:	with or without microbial filters:
Tube, mask	after every patient	after every patient	after every patient
Y-piece	after every patient	after every patient	after every patient
Both breathing hoses	after every patient	after every patient	after every patient
Temperature sensor with lead	after every patient	after every patient	after every patient
T-piece and filter of measured gas sampling system	after every patient	after every patient	after every patient
Flow sensor downstream of expiration nozzle	daily	after every patient	after every patient
Breathing bag with hose	daily	after every patient	after every patient
Breathing system	weekly	after every patient	after every patient
Soda lime container	daily	after every patient	after every patient
Anaesthetic gas scavenging hose	weekly	weekly	after every patient
Piston pump	weekly	after every patient	after every patient

Disposal of consumable articles

Consumable articles are listed at the bottom of page 87. Consumable articles and articles with limited service life (e.g. microbial filters, soda lime) can be disposed of as household refuse if used for treating non-infectious patients. **Infectious waste must be disposed in accordance with regulations!**

Wipe cleaning

Cato parts which can be wipe-cleaned are listed in the table on page 87.

Use a disposable cloth moistened with disinfectant cleaner.

Note effective time –
parts must only be wiped, not immersed –
do not allow liquid to seep inside the machine!

Cleaning and disinfection in automatic cleaning and disinfection machines

Cato parts which can be cleaned and disinfected in this way are listed in the table on page 87.

Set »Disinfection with moist heat«
(at least 93 °C, at least 10 minutes).

Alkaline or chlorinated cleaning and disinfectant agents must not be added due to the risk of corrosion!

Disinfection with formaldehyde vapour

Cato parts which can be treated in this way are listed in the table on page 87.

Parts in contact with breathing gas (breathing hoses, breathing bag, breathing system, soda lime container, piston pump) must **not** be disinfected with formaldehyde vapour!

Parts containing water, such as the humidifier or water traps, should be removed since formaldehyde accumulates here to a particularly large extent.

The machine can be disinfected with the supplementary Dräger units in place when using formaldehyde vapour. Electrical devices not made by Dräger and which may not be resistant to formaldehyde must be removed first.

After disinfection in formaldehyde vapour, the machine must be rinsed in fresh gas as specified in the Instructions for Use of the disinfection equipment in order to eliminate all formaldehyde residues (two rinses: O₂ / AIR and then O₂ / N₂O)! The rinsing effect can be verified by measuring the formaldehyde concentration at the fresh gas outlet (e.g. using Dräger formaldehyde test tubes 0.2/a).

Care

Care methods

Sterilization with ethylene oxide

Steam sterilization at 121 or 134 °C

Sterilization with ethylene oxide

Only the sensor cell of the O₂ sensor can be sterilized in this way – **no other parts!** Maximum temperature: 50 °C.

Flash-off time: at least 24 hours at 50 °C with 60-fold exchange of air. This impairs the service life of the O₂ cell!

Steam sterilization at 121 or 134 °C

Cato parts which can be sterilized in this way are listed in the table on page 87.

Steam sterilization must be preceded by sterilization with moist heat and/or wipe cleaning. All parts must be dry! Steam sterilization is only required after treating an infectious patient!

Minimum exposure times: 20 minutes at 121 °C and 10 minutes at 134 °C. Parts should preferably be sterilized at 134 °C since less time is required for the procedure.

Tabular summary

Machine components	Method				
	Disinfection by wiping or immersion	Cleaning with automatic cleaners	Formaldehyde vapour	Steam sterilization at 121 °C	Steam sterilization at 134 °C
Stripped down basic machine Parameter box (optional)	Wipe Wipe		•		
Rubber parts: Tube / mask Y-piece Breathing hoses Breathing bag Breathing bag hose Roller diaphragm of piston pump Anaesthetic gas scavenging system Pressure measuring hoses Connecting hoses for measured gas return system	• • • • • • • • • •	• • • • • • • • • •		• • • • • • • • • •	• • • • • • • • • •
Metal parts: Cylinder head of piston pump Piston of piston pump Parts of breathing system Valves of breathing system		• • • •		• • • •	• • • •
Plastic parts: Housing of piston pump Soda line container with screen T-piece for sidestream gas measurement Water trap for measured gas Water traps for airway	• • • • •	• • • • •		• • • • •	• • • • •
Filters: Microbial filter					24 times
Sensors: Temperature sensor (airway temperature) Temperature sensor (skin/rectal/oesophagus) O ₂ sensor (housing) Flow sensor	• • • externally				
<p>The following parts cannot be processed and prepared: (must be replaced if damaged or possibly contaminated)</p> <p>Filter of pressure measuring hose: Change every 6 months or when resistance increases (e.g. water absorption). The warning message P-Sensor fault resp. VENT ERR !!! is output.</p> <p>Filter of measured gas return hose: Replace every 6 months.</p> <p>for sidestream gas measurement: Water separator: Replace, depending on degree of contamination Connecting hoses: Replace after each patient Filter in T-piece: Replace after each patient</p>					

Care

Disposal of used batteries and O₂ sensors

Disposal of apparatus

Disposal of used batteries and O₂ sensors

Batteries and O₂ sensors:

- must not be thrown in the fire. Danger of explosion.
- must not be forced open. Risk of chemical burns.
- batteries must not be recharged.

Used batteries must be treated as special waste:

- Dispose of used batteries in accordance with local waste disposal regulations.

Used O₂ sensors can be returned to Dräger.

Disposal of apparatus

– at the end of its useful life.

Cato can be returned to Dräger for disposal in accordance with environmental regulations.

Reassembling system

Reassemble cleaned, disinfected and sterilized parts

Assemble the piston pump


- Check individual parts:

Gaskets: pressure marks? cracks?

Diaphragm: cracks? holes? deformation?

Defective parts must be replaced!

- 1 Fit roller diaphragm – production mark visible at the outside (arrow).
- 2 Insert piston – carefully place the rolled edge round the edge of the piston.
- 3 Screw piston into position (no quick-release catch).
- 4 Fit cylinder head and secure in position (two quick-release catches).

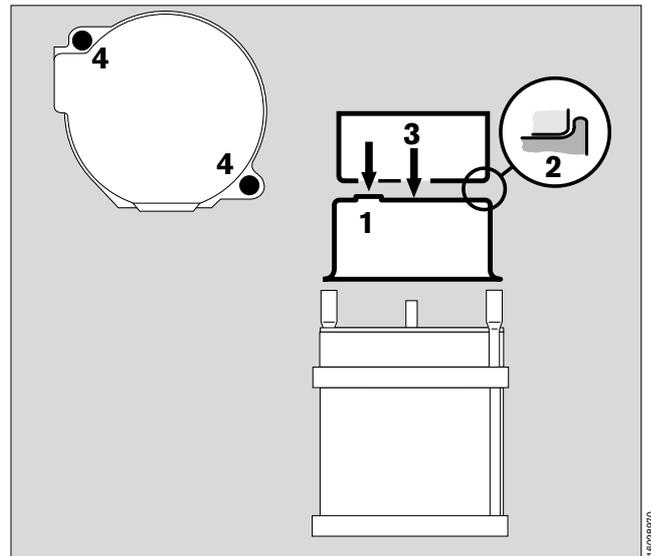

Assemble the breathing system

- Check individual parts:



Gaskets: pressure marks? cracks?

Diaphragms: cracks? holes? deformation?

Valve discs: chipped?

Defective parts must be replaced!


- 5 The six holes must be clear!
- 6 Fit valves if necessary.
- Place upper and lower parts of breathing system on top of one another.
- 7 Secure the five quick-release catches. (6 mm hexagon key).

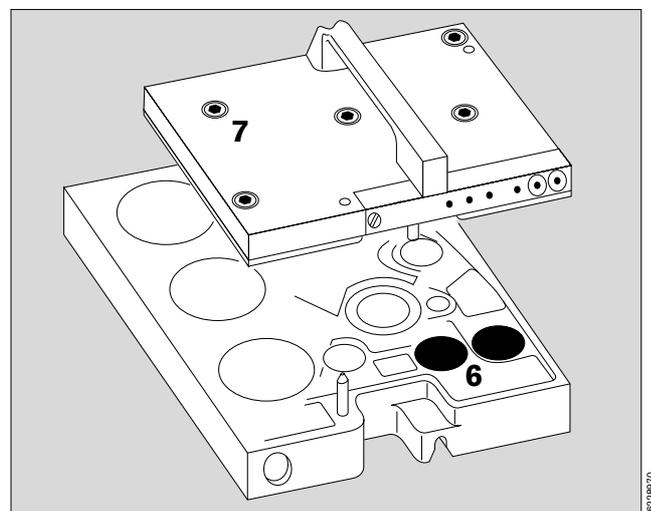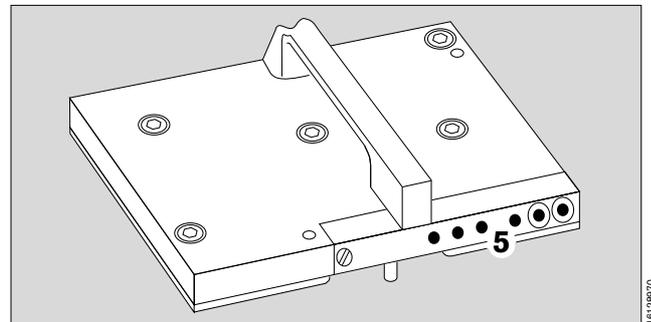

Care
 Reassembling system
 Fit piston pump and breathing system
 Replace O₂ sensor for gas sampling system
 Fit valves
 Assemble and fit the inspiratory O₂ sensor


Fit the piston pump

- Swing up table top
- 1 Handle in starting position.
Do not damage gaskets when fitting pump!
- Hold piston pump by its handle and insert in machine – **do not use force** –
- 2 Swing handle as far as possible to the left (black).

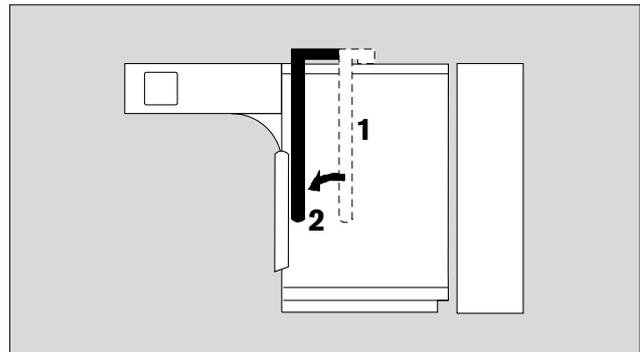

Fit the breathing system

- Hold breathing system by its
- 3 handle and insert in machine –
 - 4 lock in direction of arrow – swing as far as possible (black).
- Do not damage gaskets when fitting breathing system!**

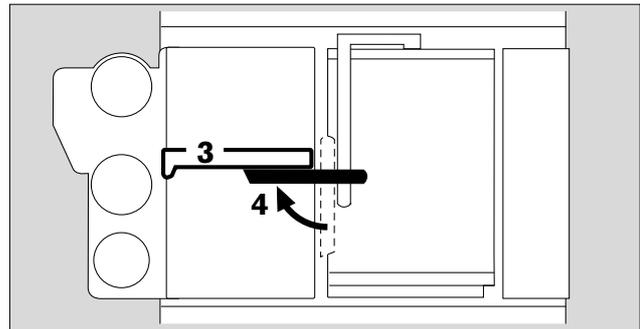


Replace O₂ sensor for gas sampling system (if the measuring cell is worn)

- 5 Unscrew the knurled screw on the back of the machine (behind the monitor) by turning anticlockwise.
- Draw used sensor or blank out of the knurled screw and insert new sensor.
- Screw knurled screw back into place.


When operating **without** sidestream O₂ sensor:

- Insert blank instead of O₂ sensor to seal the measuring system.

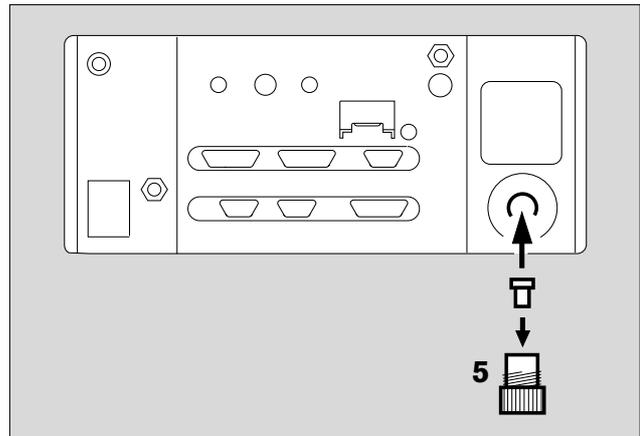


Fit valves

- 6 Fit valve discs in expiration and inspiration valves – screw on valve caps.
- 7 Screw on pressure limitation valve APL – scale facing the front.

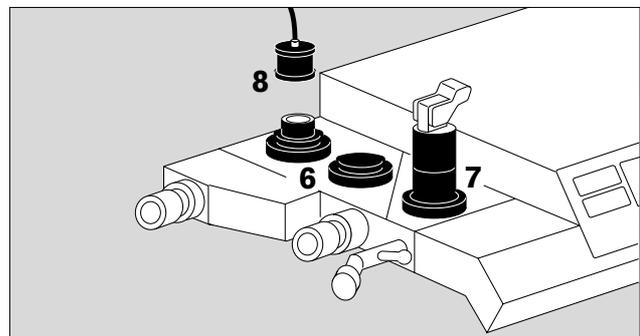


Assemble and fit the inspiratory O₂ sensor (if used)

- Do not touch wire mesh on cell!
 Insert cell with conductors facing wiring –
- screw sensor housing together –
- 8 plug sensor onto inspiration valve.

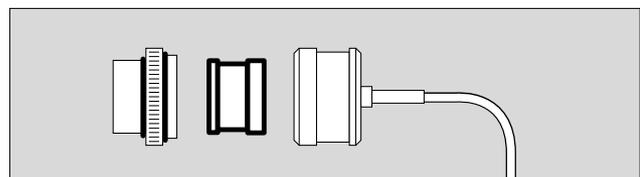


Connect fresh gas hose

- 1 to breathing system from below with plug connector.

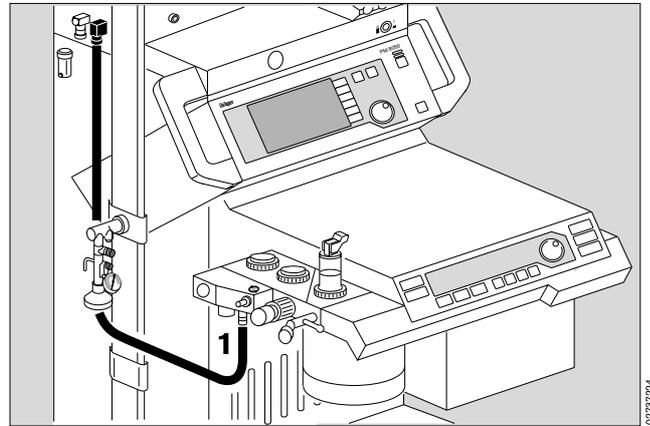

Connect pressure measuring line with filter

- 2 Plug pressure measuring line into coupling – until it engages.

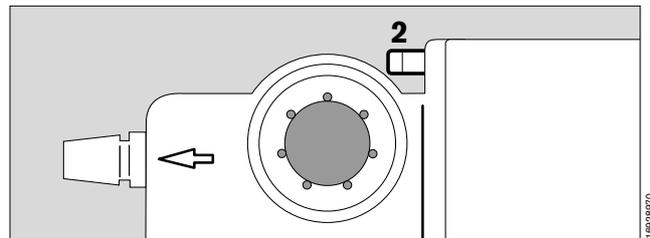


Fill soda lime container

The container can hold approx. 1.5 L soda lime »Drägersorb 800« (see separate Instructions for Use). This is sufficient to bind approx. 150 L CO₂ or for a period of approx. 6 hours.

The soda lime should be replaced daily, at the latest when two-thirds of the charge has changed colour.

The indicator colour **changes back as it dries** although the absorption capacity is exhausted!


- Place the chain on the bottom of the container.
 - Insert mesh – **must not be forgotten!**
Absolutely essential for absorption!
 - Uniformly fill the container with fresh soda lime all round, until the
– MAX –
mark is reached (approx. 3 cm below top edge).
 - Gently shake the container or knock against the table to settle the contents.
 - Remove any dust and granulated material on the edge of the container.
- 3 Fit the soda lime container into the breathing system from below and turn clockwise as far as it will go.

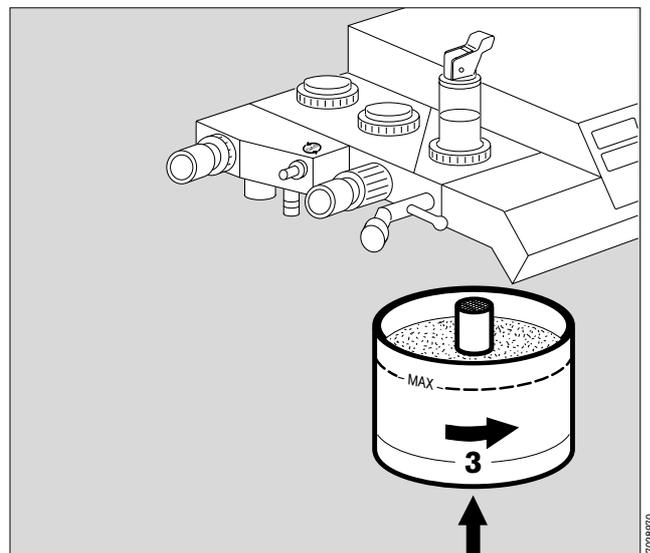


Care
 Reassembling system
 Connecting anaesthetic gas scavenging system (AGS)
 Connect sample line

Connecting anaesthetic gas scavenging system (AGS)

- 1 Insert the transfer hose, complete with scavenging adapter, into the breathing system from underneath until it engages, or connect the hose to the scavenging adapter of the semi open system.
 - 2 Route the extractor hose round the equipment to the rear and
 - 3 connect it to the connector on the collecting system.
- Refer to the separate Instructions for Use of the AGS anaesthetic gas scavenging system.

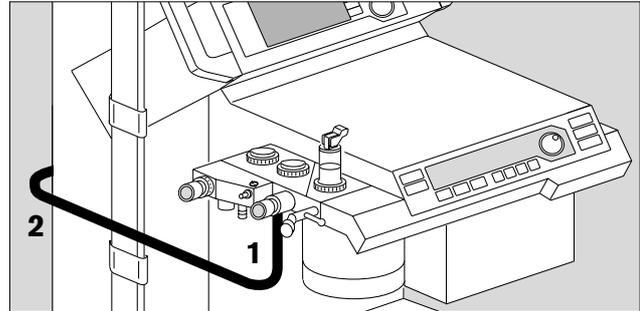

171288970

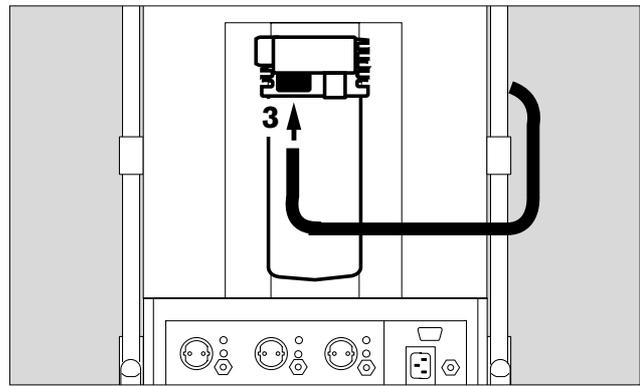


171288970

Connect sample line


For sampling CO₂, anaesthetic agent and O₂.

- 4 Screw filter into T-piece.
- 5 Insert T-piece into the patient connection of the Y-piece – filter facing upwards to prevent congestion by droplets of liquid.
- 6 Securely screw sample line onto filter and water trap.
Only use original sample lines, others may lead to changes in the machine's technical specifications!
- 7 The Y-piece with direct connection for the sample line can also be used instead of the T-piece and filter.

Alcohol or agents containing alcohol must not be allowed to enter the sample line!
 Alcohol will result in incorrect measurement of the concentration.

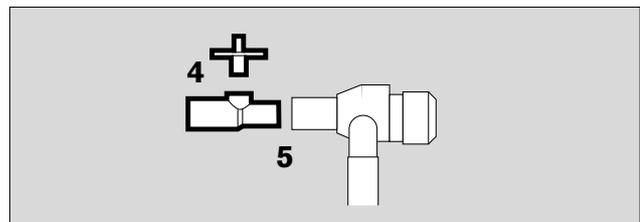


02837224

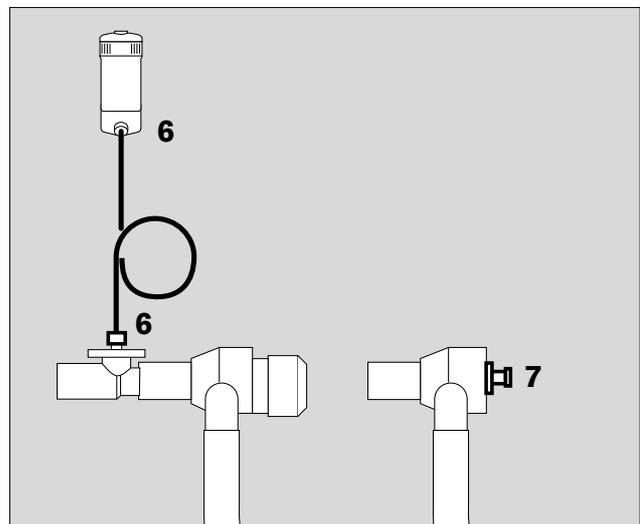

02837224

Fit flow sensor

- 1 Unscrew expiration nozzle.
- 2 Slide flow sensor into housing.
- 1 Screw expiration nozzle back into place.

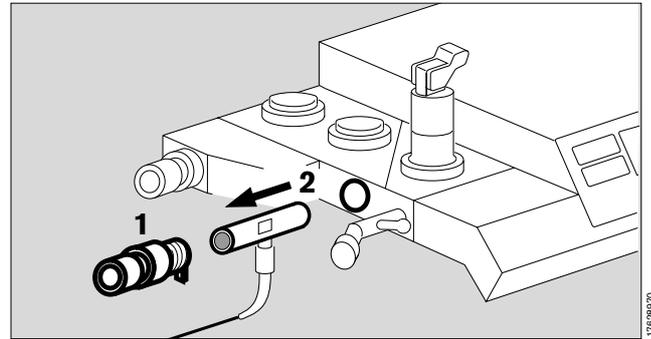

Fit microbial filter


- Fit a microbial filter with ISO nozzle on both the inspiration and expiration legs.

Fit temperature sensor

(Optional) with: Y-piece M 30 543 and hose clips 84 04 047.

The T-piece must be used for anaesthetic gas measurement in this case!

- 3 Insert temperature sensor 84 05 371 as far as possible into the hole in the Y-piece.
 After securing the Y-piece, align the sensor to face upwards so that it remains free from condensation.
- 4 Route the sensor lead back to the machine with hose clips along the inspiration hose of the anaesthetic system.

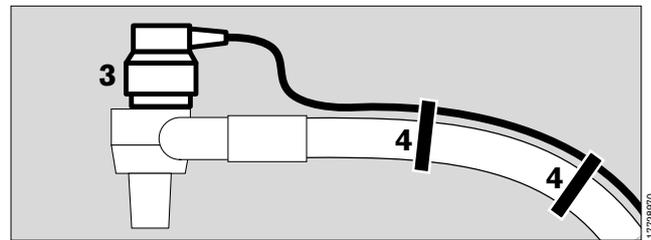


Install secretion aspirator

- Position the secretion aspirator on the mount on the frame with two secretion collecting vessels.
- 5 Install hose between negative pressure outlet and secretion collecting vessels.
 - 6 Connect the gas outlet on the Cato to the ejector.
 - 7 Plug the aspiration hose with handpiece onto the inlet.
- Secure the aspiration hose in the clamp.



Secretion aspirator may only be used in »MAN/SPONT« mode or with disconnected Y-piece.

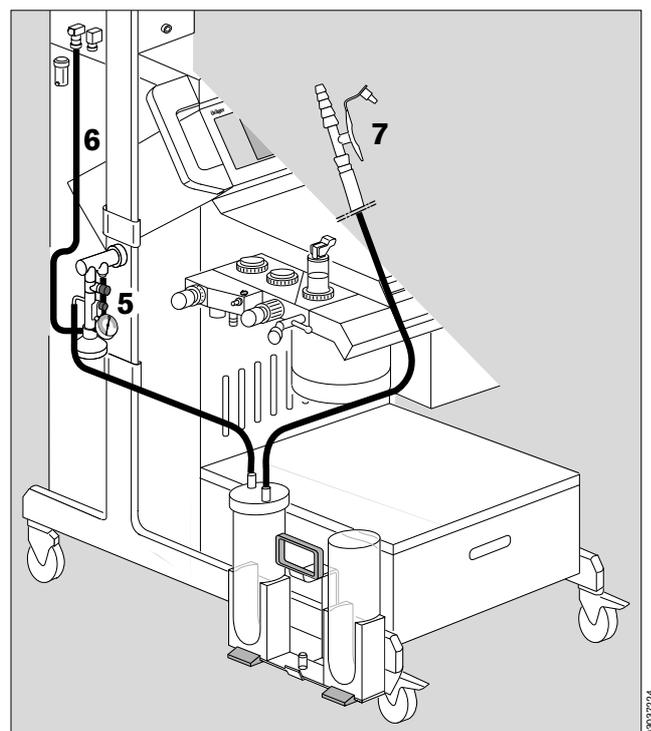

Care
Reassembling system
Connecting external equipment
Connecting equipotential bonding
Connecting to electricity supply

Connecting external equipment

Configuration of the interfaces is described on page 53.


Connection via the protocol interface

- 1 with data cable for printers with **serial interface**,
e.g. Desk-Jet (Hewlett-Packard)
or
for e.g. the PM 8060 Vitara Patient Monitor with the
MEDIBUS protocol.
See page 53 for configuration.

Connection via the Dräger RS 232 C MEDIBUS interface

e.g. to connect the PM 8060 Vitara Patient Monitor.

- 2 Connect with data cable.
- The equipment plugs must be secured with the screws provided.

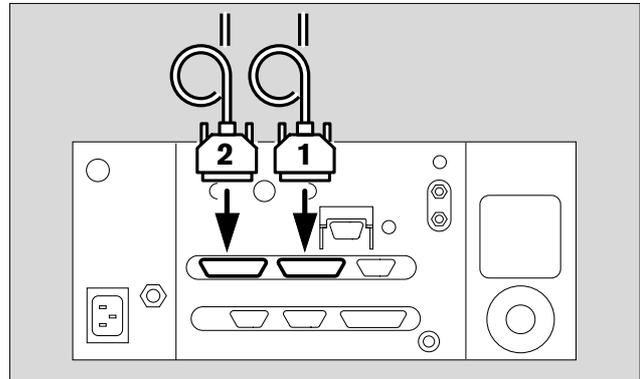

Connecting equipotential bonding


e.g. for intracardiac or intracranial operations.

- 3 Connect one end of the grounding cable to the pin at the rear.
- Connect the other end to the equipotential bonding pin on the operating table or ceiling pendant.

Connecting to electricity supply

The mains voltage must correspond to that specified on the rating plate on the rear of the machine.

- 4 Plug the mains plug into the wall socket.
- 5 The auxiliary power sockets are always switched on.

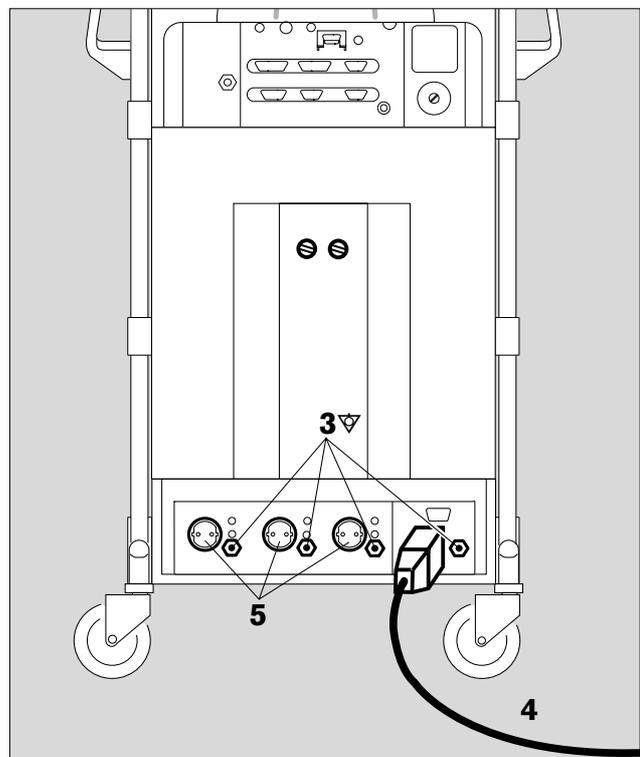

SpO₂ measurement (optional)

Safety and precautions

- The sensor must not be applied to limbs together with an arterial catheter, infusion or sphygmomanometer cuff.
- Blood circulation must not be impeded during application of the sensor. If possible, the site should be changed from time to time in order to avoid pressure necroses at the measuring point.
- The sensor should be protected against bright light (covered).
- Nellcor sensors should be used exclusively and applied as described below.
- Damaged sensors must not be used.
- The adhesive strip of the Oxiband sensor must be discarded after use. It should not be stretched unduly. Never use two strips together.

Choice of sensor

Nellcor sensors must be used exclusively. Note the Instructions for Use of the sensors. Tissue damage may result if they are positioned or used incorrectly.

The choice of sensor should be based on the following criteria:

- Patient's weight
- Mobility of the patient
- Possible application site
- Perfusion of the patient
- Period of use

The following table listing the various sensors available and their specific characteristics may prove useful here.


Type of sensor	OXISENSOR I-20	OXISENSOR D-20	DURASENSOR DS-100A	OXISENSOR D-25	OXISENSOR R-15
Age group	Infants	Children	Adults	Adults	Adults
Weight of patient	1 to 20 kg	10 to 50 kg	> 40 kg	> 30 kg	> 50 kg
Period of use	Short and long-term monitoring	Short and long-term monitoring	Short-term monitoring	Short and long-term monitoring	Short and long-term monitoring
Mobility of patient	Limited activity	Limited activity	Inactive patients only	Limited activity	Inactive patients only
Preferred site	Toe	Finger	Finger	Finger	Nose
Sterility ¹⁾	In sterile packaging	In sterile packaging	—	In sterile packaging	In sterile packaging

OXISENSOR™ I-20, OXISENSOR™ D-20, DURASENSOR™ DS-100A, OXISENSOR™ D-25 and OXISENSOR™ R-15 are registered trademarks.

¹⁾ in undamaged, unopened packaging.

- 1 Choose the appropriate sensor.
Plug the sensor connector into the rear of the monitor.

Do not let the sensor lead hang over the monitor, as it may impair SpO₂ measurement!

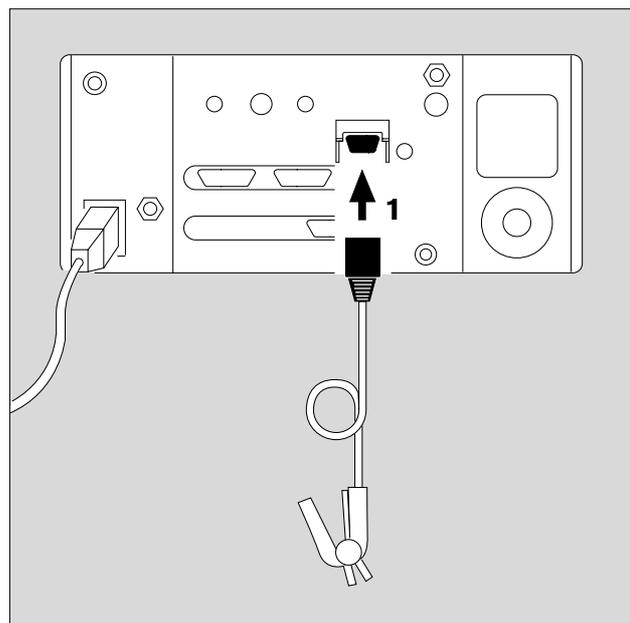

C-Lock ECG synchronization (optional)

If the patient is extremely mobile or has a very low arterial flow, the signals from the SpO₂ measurement can be amplified with the aid of C-Lock ECG synchronization. In this case, the monitor receives two separate signals reflecting the cardiac activity:


an optical signal from the SpO₂ sensor

and

an electrical signal from the ECG monitor.

The monitor uses the R-wave of the ECG signal to detect the pulse and for synchronization with the SpO₂ measurement.

- Connect the ECG signal from the ECG monitor to the rear of the monitor by means of a lead and pawl-type plug. Prerequisites for the electrical signal and pin assignment can be found in the »Technical Data« on page 121.

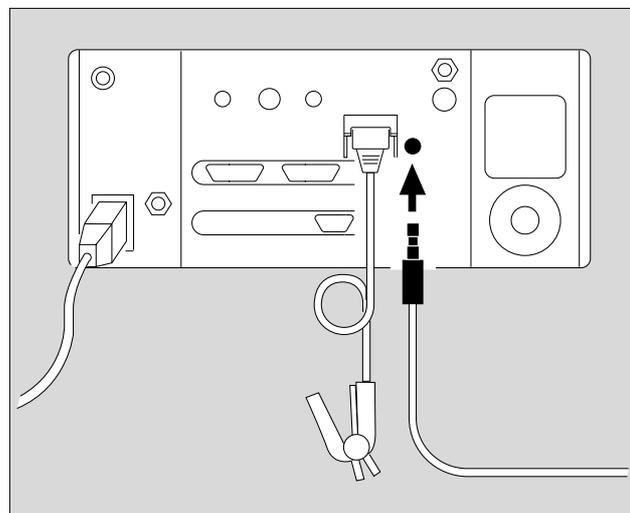

In the event of a delayed ECG signal

Synchronization may be impaired if the ECG signal is delayed by more than 40 milliseconds relative to the QRS complex.

The monitor must be operated without C-Lock ECG synchronization whenever there is any suspicion of such a fault existing.

Tips to avoid artefacts

Nellcor sensors must be used exclusively and positioned correctly in order to avoid the risk of measuring errors and tissue damage.

Damaged sensors with exposed electric wires must not be used – risk of electric shock.

Adhesive strips of the Oxiband OXI-A/N and OXI-P/I sensors must not be reused: they may not adhere properly.

The strips must not be stretched unduly.

Never use two strips together, as this may lead to venous pulsation and failure of the pulse signal.

High intrathoracic pressure, pressure on the thorax and other consecutive impairments of the venous flow can lead to venous pulsation with failure of the pulse signal.

The pulse signal may fail in the presence of shock, low blood pressure, severe vasoconstriction, major anaemia, hypothermia, arterial occlusion proximal to the sensor and asystolia.

The sensor must be covered in bright light (e.g. surgical lamps and direct sunlight), otherwise the pulse signal may fail or inaccurate results may be obtained.

The sensor should not be positioned on limbs together with an arterial catheter, sphygmomanometer cuff or intravascular venous infusion: pulse signal may fail and measurement becomes inaccurate.

Major concentrations of dyshaemoglobins, such as carboxyhaemoglobin or methaemoglobin, can make the measurement inaccurate.

Intravascular dyes, such as methylene blue, may similarly result in inaccurate measurement.

Electrocautery can impair the measuring accuracy; the leads and sensor should therefore be positioned as far away as possible from the electrocautery and its neutral electrode.

Sensor performance may be impaired if the patient moves violently, thus leading to inaccurate results. The sensor should be applied to a different site in such cases in order to reduce the risk of artefacts due to movement.


Applying the sensors

Durasensor DS-100 A

Reusable sensor for short-term monitoring of relatively quiet patients weighing over 40 kg.

The sensor is preferably positioned on the index finger, although other fingers can also be used. The fifth finger should be used if the patient is particularly large or obese.

- Open the clip slightly and slide the sensor onto the finger. The tip of the finger must touch the end, the soft padding resting on the nail and tip of the finger. The lead should be on top of the finger.
- Ensure that the finger is not compressed or hurt by the clip.


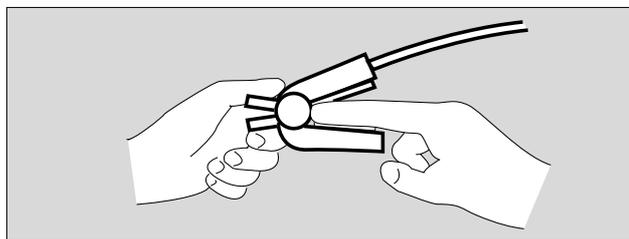


Oxisensor D-25 und D-20

Adhesive sensors for short and long-term monitoring of patients with limited mobility and weighing from 15 to more than 50 kg.

Long nails make application of the sensor more difficult and coloured nail varnish impairs the accuracy of measurement.



- Cut the nail if necessary.
 - Remove nail varnish if necessary.
 - Draw the protective film off the adhesive strip.
 - Place the sensor on a flat surface with the adhesive side facing upwards.
 - Place the tip of the index finger onto the middle of the optical element on the side of the sensor opposite the lead and wrap the adhesive strips round the finger.
 - Fold the other side of the sensor over the tip of the finger and align it on the underside so that the markings are lined up correctly. Press the sensor into place and wrap the adhesive strips round the finger.
- A thinner finger should be used instead of the index finger if the patient is very obese.

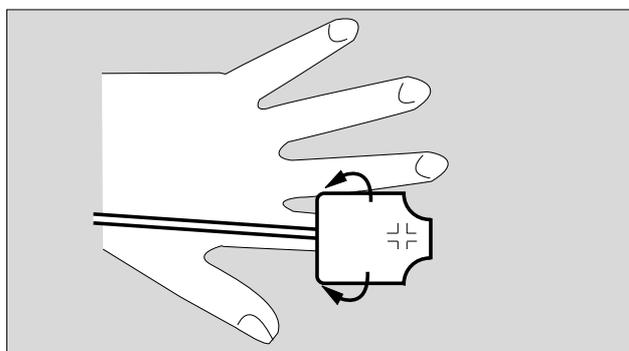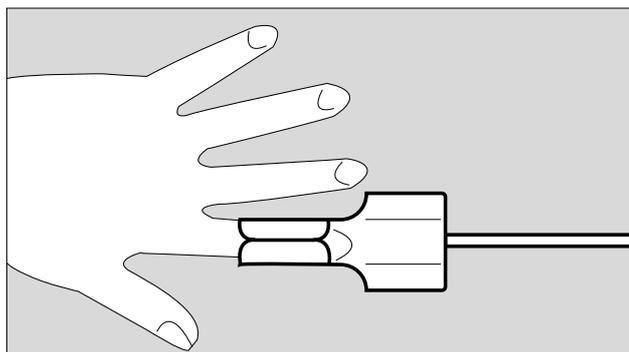

Oxysensor I-20

Adhesive sensor for short and long-term monitoring of patients with limited mobility and weighing between 3 and 15 kg.


- Peel the protective film off the adhesive strip.
- Place the sensor underneath the big toe so that the dotted line corresponds to the inner edge of the toe and the marking is in the middle of the toe.
- Wrap the sensor round the toe so that the other marking rests exactly on top of the toenail.
- Secure the sensor lead to the foot with the additional adhesive tape.

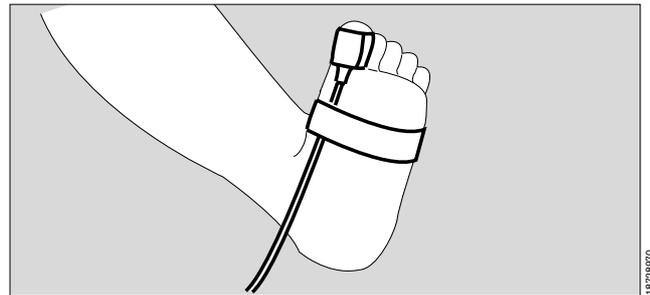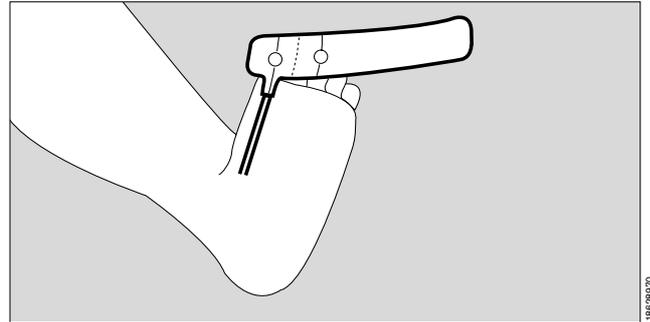

Reusing the sensor:

The sensor can be reused if the tape is still tacky. Adhesion is improved by small additional adhesive spots.

- Hold the adhesive spots by the blue tags, peel them off the backing paper and remove the protective film.
- Affix one spot to the middle of each optical element.
- Position the sensor as described above.

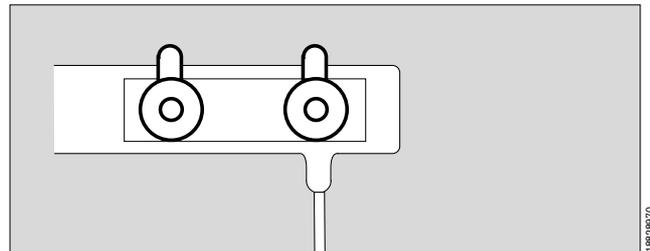

Other measuring point:

The sensor is preferably applied to the big toe because it moves less than the patient's hand. If the big toe is not available, however, the sensor can also be applied to the thumb.


- Peel off the protective film on the adhesive strip.
- Position the sensor under the thumb so that the dotted line corresponds with the edge of the thumb and the marking is in the middle of the thumb.
- Wrap the sensor round the thumb so that the other marking lies exactly on top of the thumb nail.
- Secure the sensor lead to the hand with the additional adhesive tape.



Oxisensor R-15

Disposable adhesive sensor for short and long-term monitoring of **inactive** patients weighing more than 50 kg. Preferably used for patients possibly suffering from severe vasoconstriction or poor circulation.

- Clean the bridge of the nose with the liquid in the enclosed ampoule – keep away from the eyes.
- Peel off the protective film on the sensor.
- Align the sensor symmetrically on the bridge of the nose: the two symbols should lie on the bone/cartilage boundary.
- Press the sensor firmly into place and hold for 10 seconds to ensure good adhesion.
- The R-15 sensor must not be used on patients with nasal intubation or a mask.

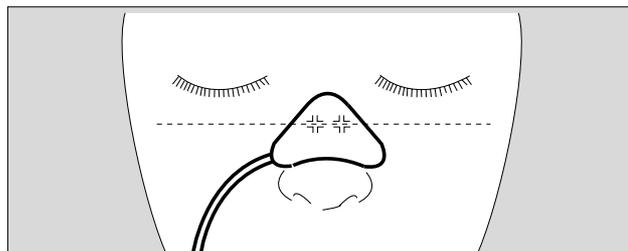

Check operational readiness

Contents

	Page
O ₂ shortage signal	102
Supply of anaesthetic agent	102
Temperature measurement (optional)	102
SpO ₂ measurement (optional)	102
Power failure alarm	104
Self-test	103
Calibrate flow sensor	103
Calibrate inspiratory O ₂ sensor with 100% O ₂	103
Calibrate O ₂ sensor for sidestream measurement with 100% O ₂	104
Linearity	104
Manual ventilation function	105
Automatic ventilation function	105
Maintenance intervals	106

Check operational readiness
 O₂ shortage signal, Supply of anaesthetic agent
 Temperature measurement, SpO₂ measurement
 Power failure alarm


Operational readiness of the machine must be restored and checked whenever it has been cleaned, disinfected, sterilized, repaired or serviced!

Supplementary equipment must be checked in accordance with the respective Instructions for Use. Note the applicable time limits for filters, maintenance and calibration, e.g. in conjunction with equipment for measuring blood pressure or body temperature.

Checking operational readiness includes the check based on the checklist (page 21 onwards), the self-test (page 31 onwards) and the following:


O₂ shortage signal

- 1 Open O₂ delivery valve.
- Interrupt O₂ supply.
- O₂ shortage signal must be given after approx. 3 seconds: continuous tone for at least 7 seconds.

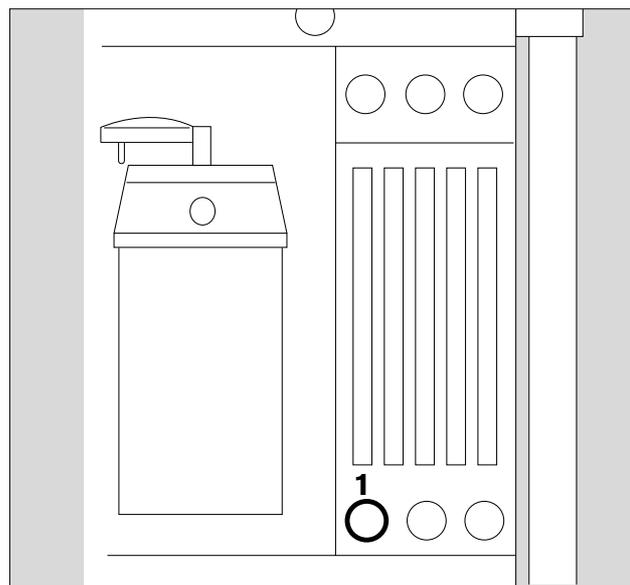

Supply of anaesthetic agent

- 2 Check filling level – top up anaesthetic agent if necessary – in accordance with separate Instructions for Use of the anaesthetic agent vaporiser.
 - If the last inspection was more than 6 months ago (sticker on Vapor):
 - Change Vapor!
- 3 Lock control dial in »0« position.

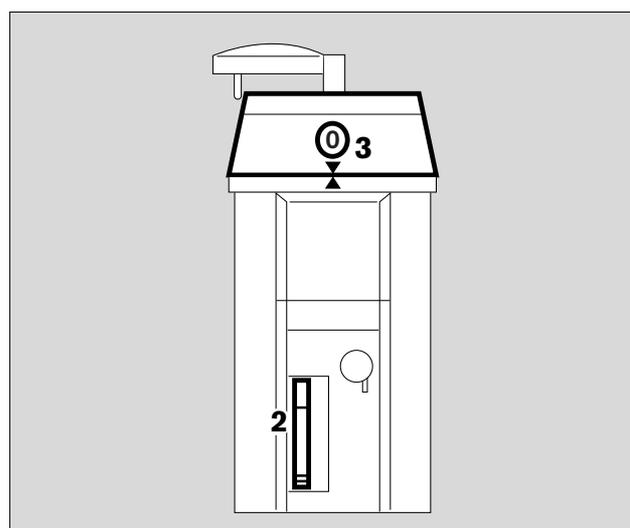


Temperature measurement (optional)

- Remove temperature sensor from Y-piece and place in a water bath (21 to 49 °C) – compare measured value on screen with that of a second thermometer of known accuracy.


SpO₂ measurement (optional)

- Apply Durasensor DS 100 A to own finger.
- Indication must be plausible.

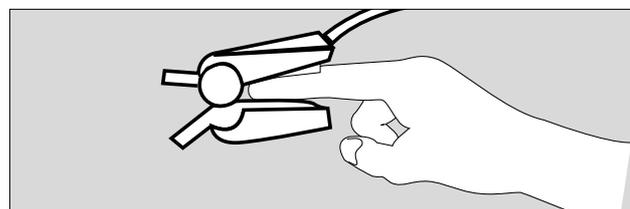

Power failure alarm (only if no UPS fitted)

- Interrupt power supply, e.g. disconnect plug from socket.
- 4 Switch on machine – press power switch, turn to »I«. Power failure alarm is given.

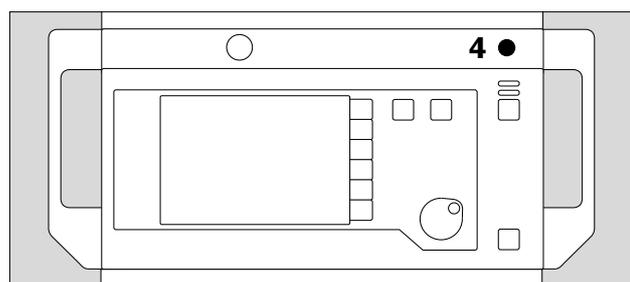

Continuous tone – volume must remain constant for 30 seconds.

If not, reconnect to mains and leave machine on for 24 hours so that battery can recharge. Repeat test!

- Switch off machine. Press power switch again and turn to **O**. Alarm goes out.
- Reconnect to power supply.

Self-test

The self-test must be completed successfully in order to establish readiness for operation!

Supplementary equipment must not be switched on until after the self-test.

Start self-test:

- Switch on machine: press power switch.
 Test proceeds as described from page 31 onwards.

Calibrate flow sensor, see page 30

Calibrate inspiratory O₂ sensor with 100% O₂

This is only necessary if sidestream O₂ measurement is not used.



- Remove O₂ sensor and place test adapter 68 01 349 on sensor.
- Allow an O₂ flow of approx. 1 L/min to flow over the O₂ sensor for approx. 2 minutes.
- Select »more« with the rotary control.
- Select »O₂ sensor 100 Vol.%« with the rotary control.
- Display:

When the O₂ sensor has been flushed with O₂ for approx. 2 minutes:

- Press rotary control to confirm; calibration is then started and continues automatically.

The clock icon is replaced by a tick (✓) when calibration is complete.

- Replace the O₂ sensor in its mount.

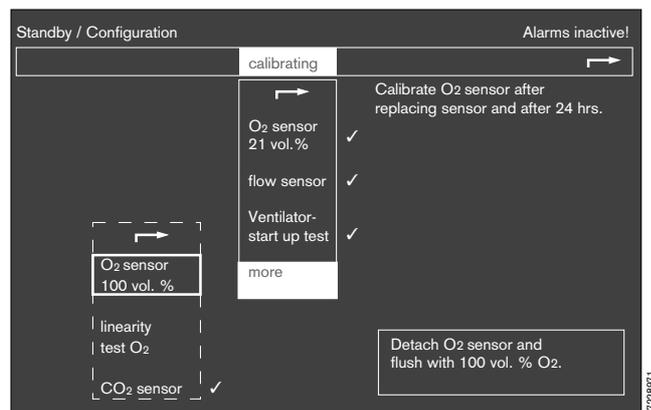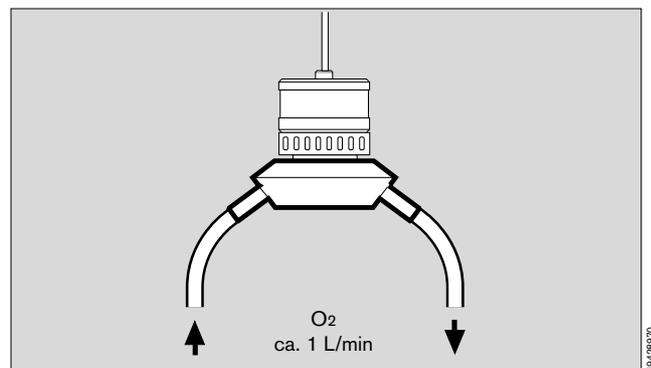

Check operational readiness
 Calibrate O₂ sensor for sidestream measurement
 Check linearity

Calibrate O₂ sensor for sidestream measurement with 100% O₂


This is necessary for the monthly linearity check.

Prepare a substitute sampling line:

- Cut the sampling hose through the middle.
- Unscrew the original sampling hose from the water trap and screw on the slit sampling hose.

On Cato:

- Disconnect fresh gas hose from breathing system.
- Set an O₂ flow of 1 L/min O₂ at the O₂ flow tube and slide the sampling hose into the fresh gas hose.
- Use the rotary control to select »**calibrating**« and then »**more**«.
- Select »**O₂ sensor 100 Vol.%**« with the rotary control.
- Display:
- Allow O₂ to flow for approx. 2 minutes.
- Press rotary control to confirm; calibration is then started and continues automatically. The clock icon (✓) is then replaced by a tick (⊖).
- Screw the original sampling hose back onto the water trap.
- Plug the fresh gas hose back into the breathing system.

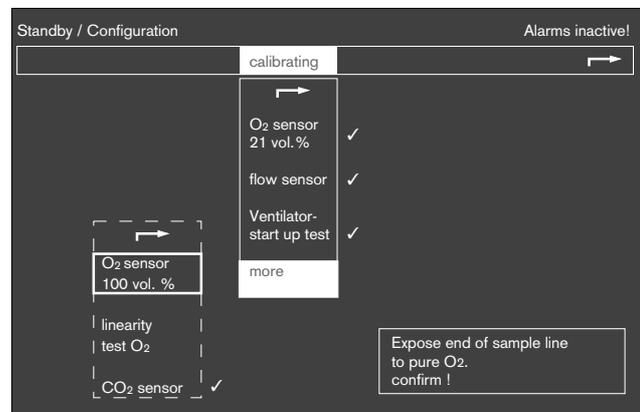

Check linearity

– Must be checked every month.


First calibrate the O₂ measurement used (inspiratory or sidestream) with 100% O₂ (see above).

Then:

- Expose the inspiratory O₂ sensor to ambient air for approx. 2 minutes or,
- in the case of the O₂ sensor for sidestream measurement:
- unscrew the sampling hose from the water trap and allow it to take in air for approx. 2 minutes.

Display on screen should show between 18 and 24 vol.% O₂.

If value displayed is outside 18 to 24 vol.% O₂ sensor capsule is faulty.

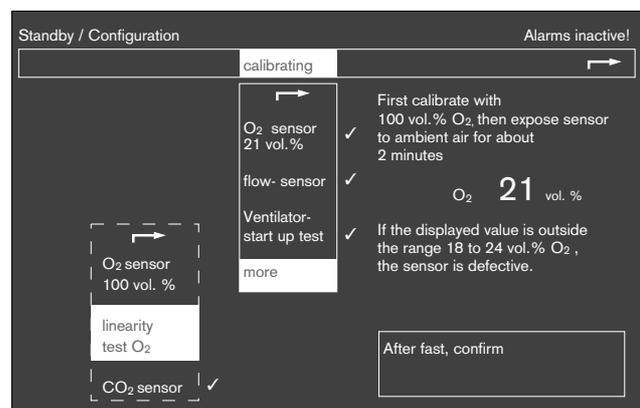


Then:

- Put inspiratory O₂ sensor back and calibrate
 or
- Put O₂ sensor for sidestream O₂ measurement back
 and calibrate.
- Put O₂ sensor back,
 and
- screw sampling hose back onto water trap.

Manual ventilation function


Machine in Manual/Spontaneous mode:

- 1 Press »MAN/SPONT«.
- 2 Pressure limiting valve (APL) set to MAN.
 - Connect lung simulator, exercise thorax or breathing bag to Y-piece.
- 3 Set fresh gas flow.
- 2 Set maximum ventilation pressure between 5 and 70 mbar on APL valve – turn valve head for this purpose.
 - Compress breathing bag.
 - Compare pressure indication on monitor with setting on pressure limiting valve.
 If excess pressure has to be relieved quickly:
 - 2 Press vertically down on the lever of the pressure limiting valve.

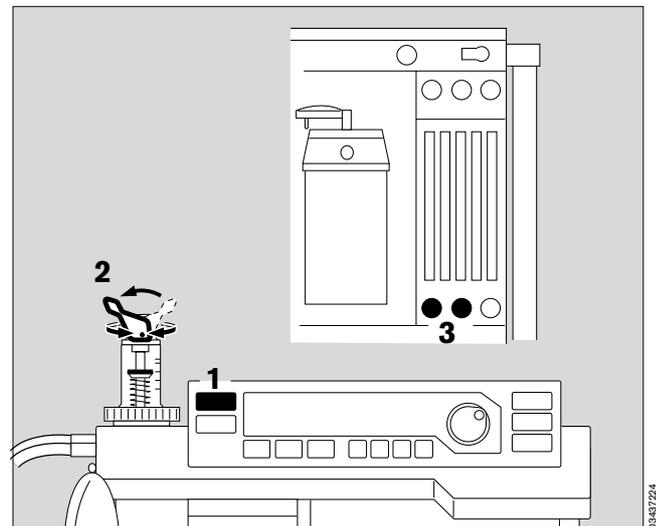

Automatic ventilation function

- Connect lung simulator, exercise thorax or breathing bag to Y-piece.
- 4 Set fresh gas.
 - 5 Press »IPPV«.
 - 6 Press rotary control.
 - 7 Piston movement is indicated on the bar graph.
- The machine starts with the ventilation parameters set upon delivery (or programmed to customer's requirements by DrägerService) (see page 39).
- Lung simulator inflates regularly.
 - Pressure profile is displayed on the monitor.
 - Volume measurement yields plausible values.

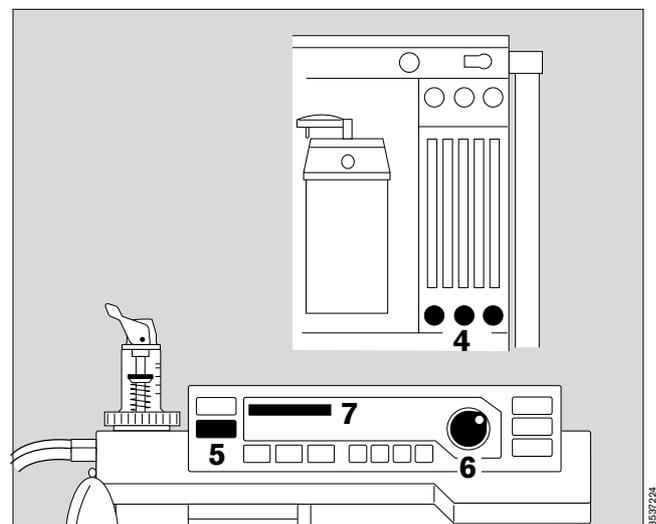

Maintenance intervals

The machine and machine parts must be cleaned and disinfected before starting any maintenance work and before returning to the manufacturer for repair.


"WaterLock" water trap	See separate Instructions for Use.
O ₂ sensor	Replace if calibration is no longer possible or if the message FIO₂ INOP! is displayed. Disposal: see page 88.
Flow sensor	Replace if calibration is no longer possible or if the message FLOW INOP! is displayed. Can be incinerated at temperatures above 800 °C with low pollutant emissions.
Bacterial filter in the secretion aspiration line	Replace after 14 days. Disposal: as infectious special waste. Can be incinerated at temperatures above 800 °C with low pollutant emissions.
Bacterial filter in measured gas return line	Replace every 6 months. Can be incinerated at temperatures above 800 °C with low pollutant emissions.
Bacterial filter in pressure measuring line	Replace every 6 months. Can be incinerated at temperatures above 800 °C with low pollutant emissions.
Cooling air filter	Clean monthly and dry well, or replace filter. Replace after 1 year at the latest. Disposal: as domestic waste.
Lithium battery for data storage	Must be replaced by qualified personnel after 2 years. Disposal: see page 88.
Optical test bench for measuring anaesthetic gas concentration	Must be inspected every 6 months by qualified personnel.
Time Keeper RAM	Must be replaced after 3 years by qualified personnel. Disposal: in conformity with local waste disposal regulations.
Inspection and servicing	Every 6 months by qualified personnel.

What's what


Contents

	Page
Overall view of the machine	108
Operating elements and displays on ventilator	109
Operating elements and displays on monitor	110
Operating elements and displays on the gas and anaesthetic agent measuring tube bank	111
Elements on breathing system	112
Connections on the rear of the monitor	113
Dimensions / Weight	114


Overall view of the machine

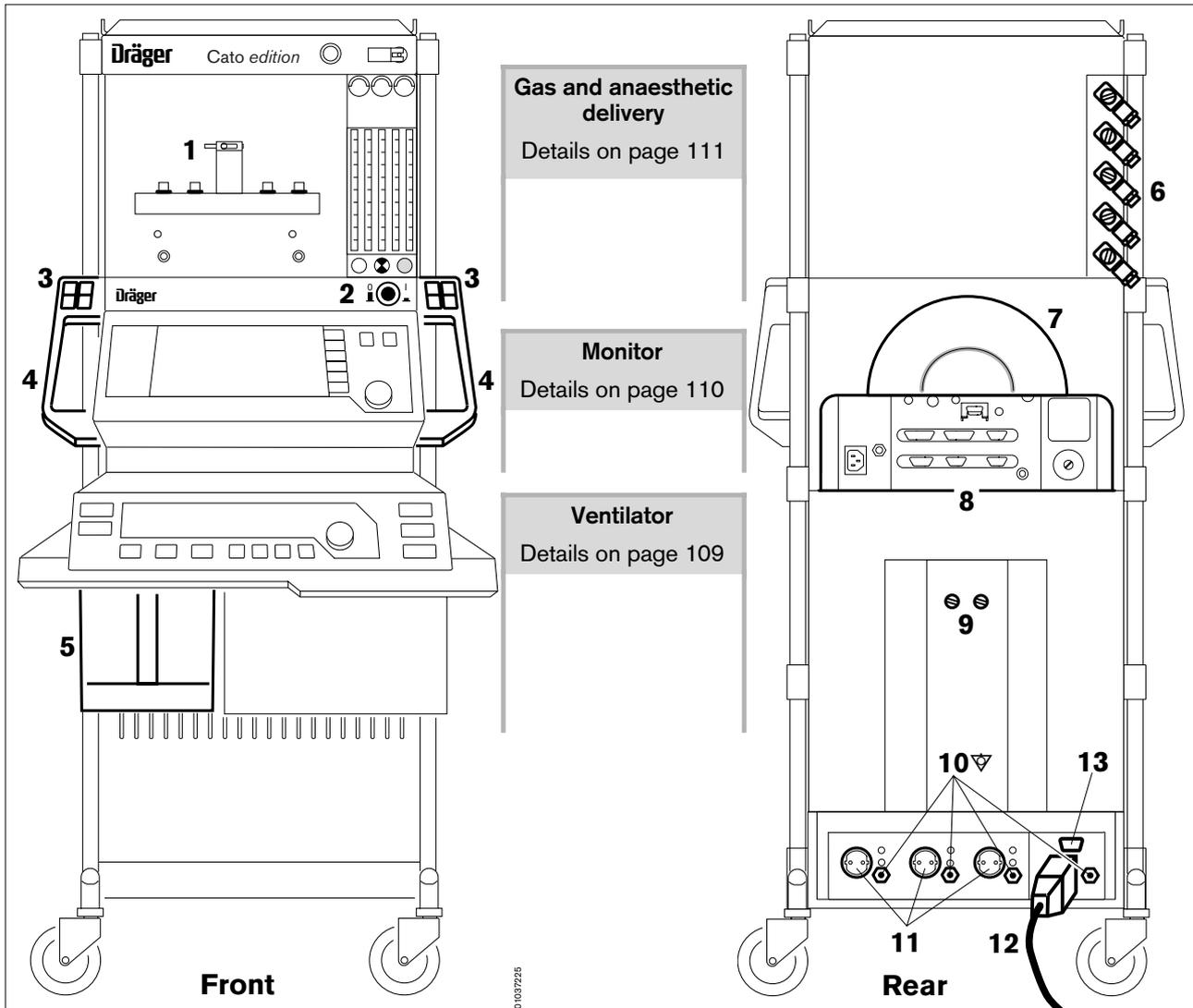

- 1 Vapor connection
- 2 Master switch
- 3 Push-buttons for ceiling pendant (optional)
- 4 Transport handles
- 5 Soda lime container
- 6 Gas supply ports for O₂, AIR, N₂O and gas cylinder ports for O₂ and N₂O (from top to bottom)
- 7 Holders for cables and hoses
- 8 Rear of monitor
- 9 Fixing bolts of the anaesthetic gas scavenging system
- 10 Grounding pin (equipotential bonding)
- 11 Sockets for supplementary equipment
- 12 Power cable
- 13 Sub-D socket for connecting the uninterruptible power supply (UPS)
- 14 AIR connector for secretion aspiration and fresh gas port for the ventilator
- 15 Water separator with water trap in the gas sampling line


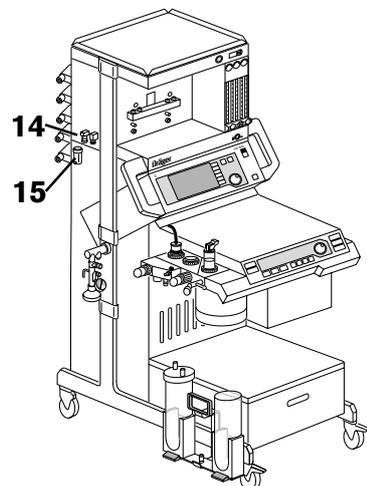


Operating elements and displays on ventilator

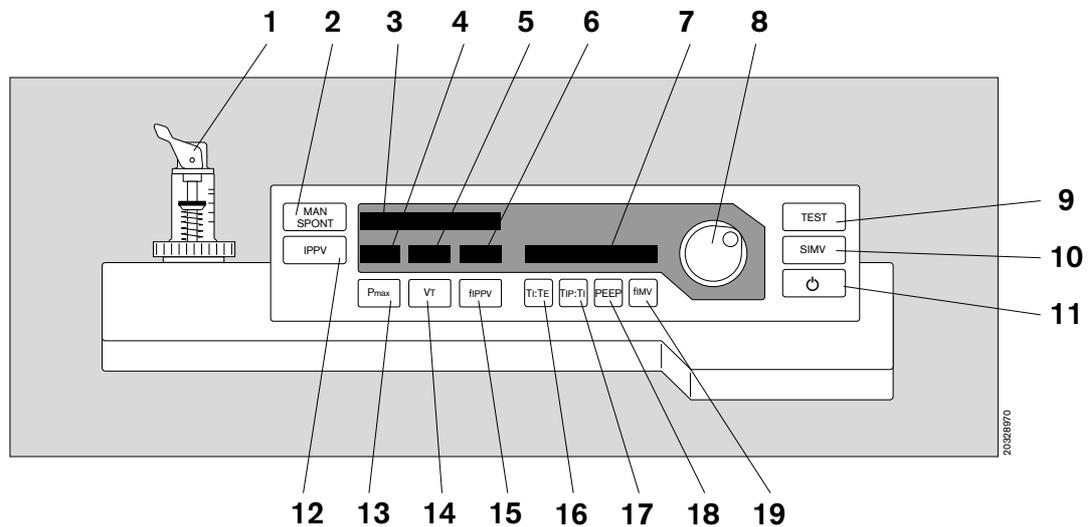

- 1 Pressure limiting valve (APL) with Manual/Spontaneous changeover
- 2 MAN/SPONT key for manual ventilation/spontaneous breathing
- 3 Bar graph of relative piston movement [%]
- 4 Indication of the set maximum pressure (P_{max})
- 5 Indication of the set tidal volume (V_T)
- 6 Indication of the ventilation frequency (f_{IPPV})
- 7 Window for interactive settings and selection
- 8 Rotary control for settings and selection (confirmation)
- 9 Key for leakage test and compliance measurement
- 10 Key for mechanically-aided spontaneous breathing (SIMV)
- 11 Key for invoking »standby« mode
- 12 Key for automatic ventilation (IPPV)
- 13 Key for setting the maximum pressure (P_{max})
- 14 Key for setting the tidal volume (V_T)
- 15 Key for setting the ventilation frequency (f_{IPPV})
- 16 Key for setting the ratio of inspiration time to expiration time (TI:TE)
- 17 Key for setting the ratio of inspiration pause time to inspiration time (TIP:TI)
- 18 Key for setting the positive end-expiratory pressure (PEEP). Only possible with IPPV!
- 19 Key for setting the frequency for mechanically-aided spontaneous breathing (f_{IMV})


Operating elements and displays on monitor

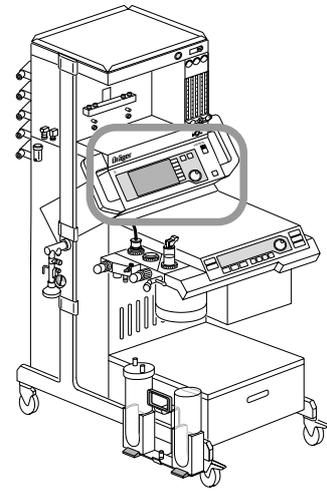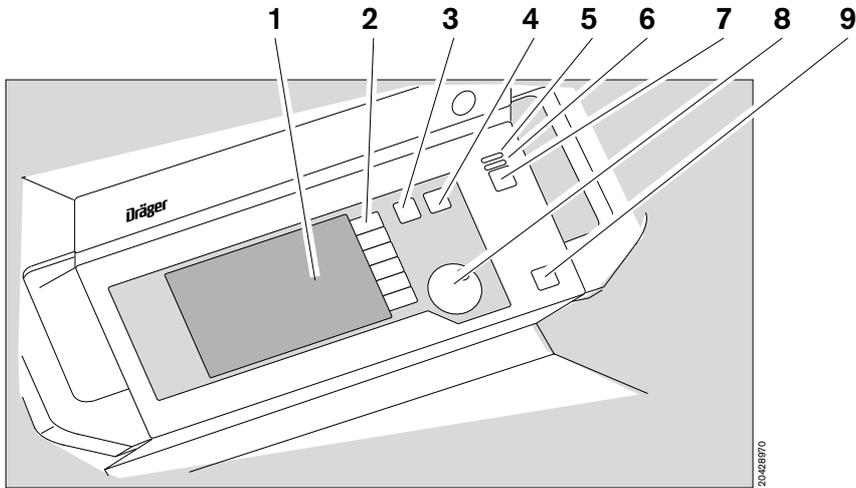


Front panel of monitor:

- 1 Screen
- 2 Softkeys
- 3 Key for successively calling up the standard screen, data screen and trend screen
- 4 Key for calling up the standard screen
- 5 Red warning LED
- 6 Yellow caution/advisory LED
- 7 Keys for deactivating the alarm tone for two minutes and for reactivating it
- 8 Rotary control for selecting = turn and confirming = press menu items
- 9 Key for standby or measuring mode. The yellow LED lights up in standby mode

Screen layout:

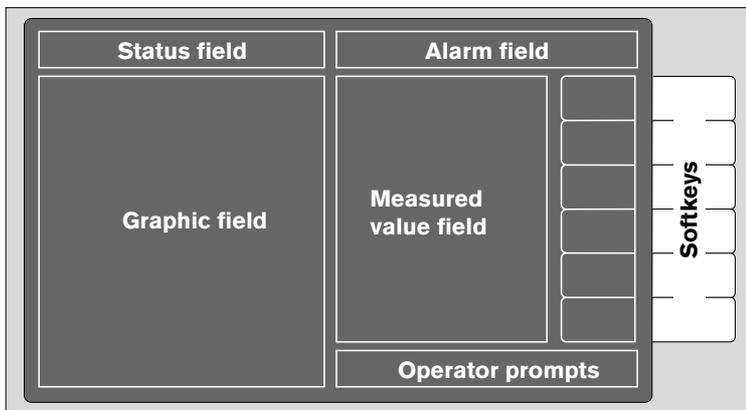


Operating elements and displays on the gas and anaesthetic agent measuring tube bank

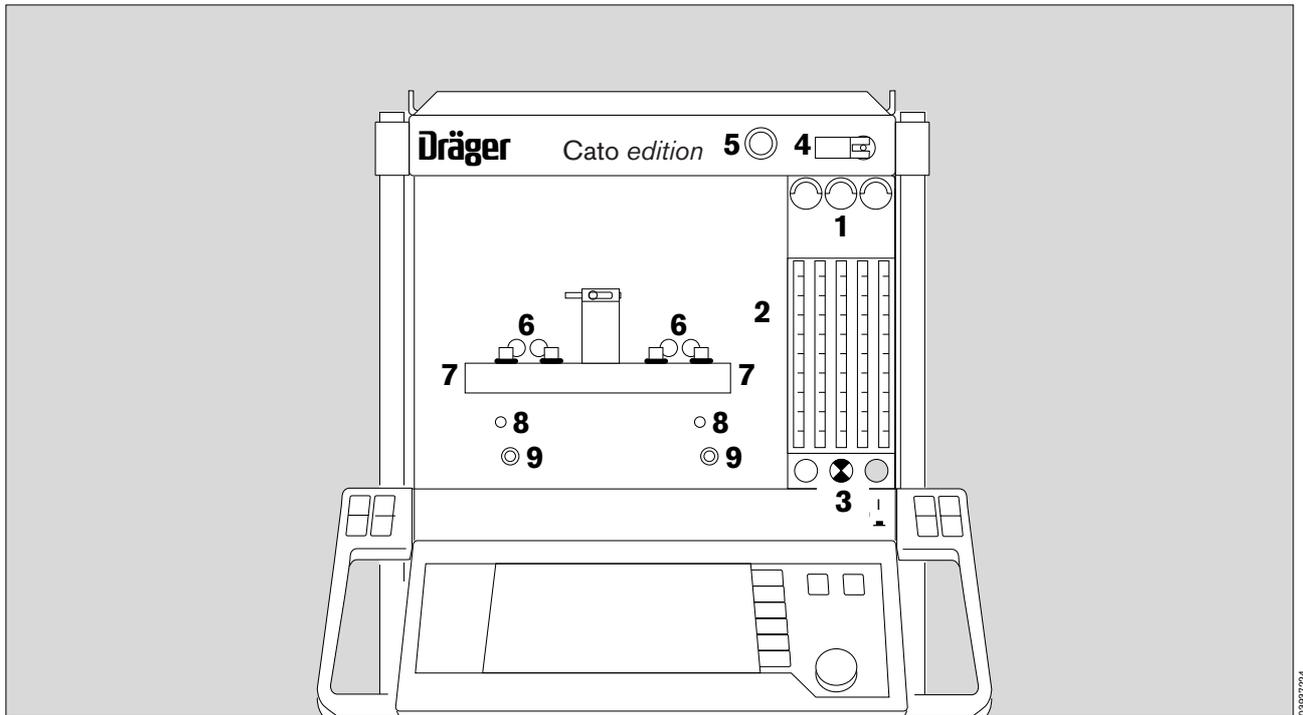

- 1 Pressure gauge for O₂, AIR, N₂O
- 2 Flow measuring tubes for O₂, AIR, N₂O
- 3 Delivery valves for O₂, AIR, N₂O
- 4 N₂O / AIR changeover
- 5 Key for O₂
- 6 Optical code reader for Vapor recognition
- 7 Vapor connection
- 8 Supporting pin
- 9 Whistle for O₂ shortage signal

Elements on breathing system

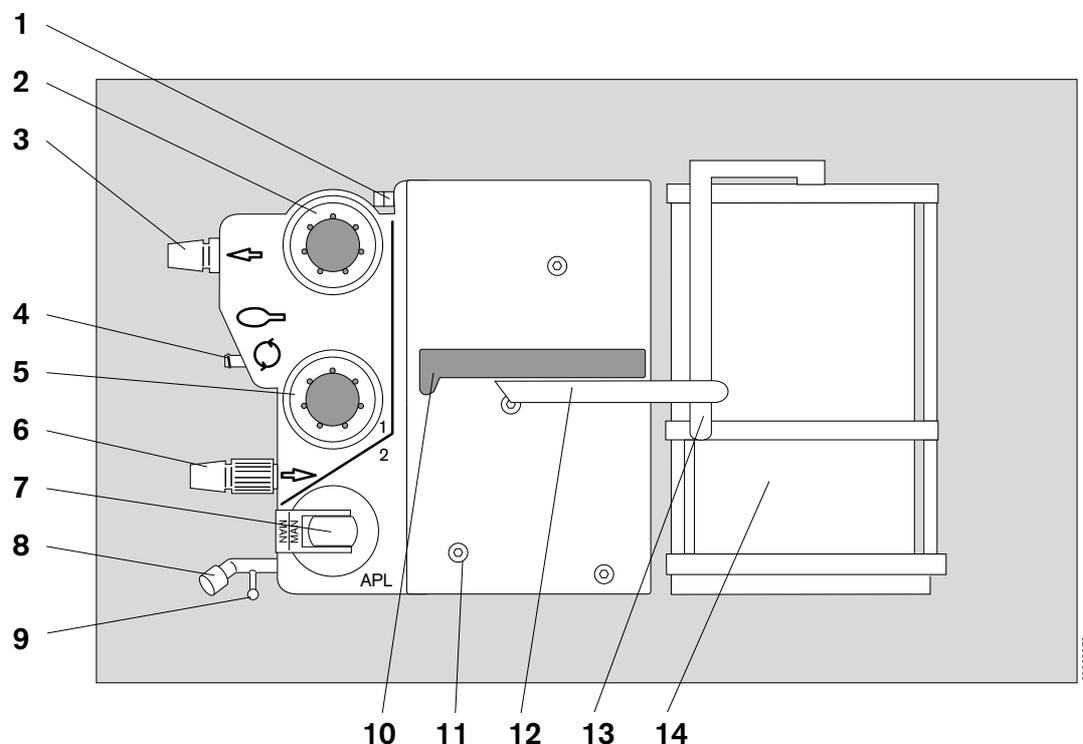


- 1 Plug-in connection for pressure measurement (P_{aw})
- 2 Inspiration valve (with mount for O_2 sensor, optional)
- 3 Inspiration port (ISO cone 22)
- 4 Quick-release coupling for measured gas return line
- 5 Expiration valve
- 6 Expiration port (ISO cone 22) with mount for flow sensor (screwed in)
- 7 Pressure limiting valve (APL) with pressure setting (turn), MAN/SPONT changeover (two-way switch) and pressure relief (press)
- 8 Plug to seal Y-piece for leakage test
- 9 Support for manual ventilation bag
- 10 Handle of breathing system
- 11 Quick-release catches for dismantling breathing system (5 units)
- 12 Release lever for breathing system and piston pump
- 13 Release lever and handle of piston pump
- 14 Piston pump

See also page 73 for


- position of subsystems
- gas flow chart and
- position of valves in breathing system.

Connections on the rear of the monitor

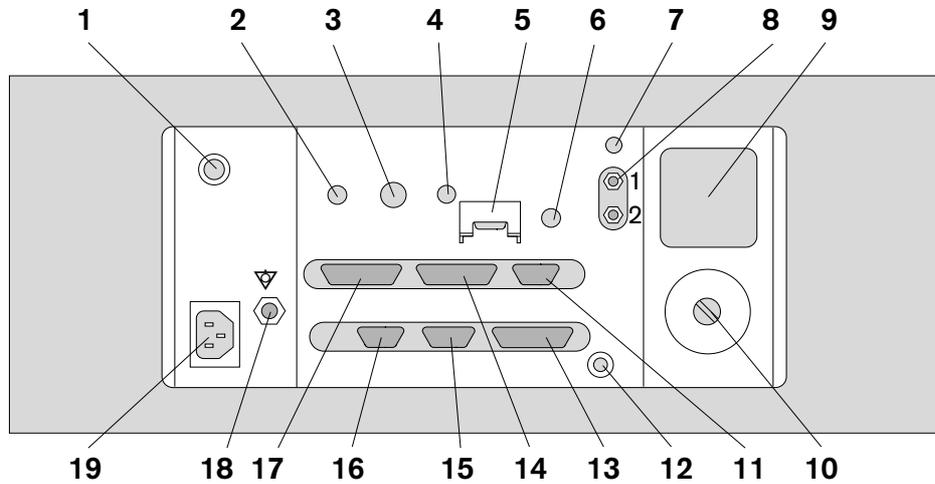

- 1 Master switch
- 2 Connection for O₂ sensor
- 3 Connection for flow sensor
- 4 Connection for temperature sensor
- 5 Connection for SpO₂ sensor lead
- 6 Input for ECG/SpO₂ synchronization
- 7 Connection for anaesthetic gas return or scavenging
- 8 Connection for sidestream CO₂ / O₂ / anaesthetic gas measurement (leads to water trap on the left-hand side of the Cato as seen from the front. The sampling line is connected to the water trap.)
- 9 Cooling air filter
- 10 Sensor for sidestream O₂ measurement
- 11 Analog output
- 12 Connection for pressure measuring line
- 13 Connection for ventilator
- 14 RS 232 interface (printer); can also be configured as RS 232 C interface (MEDIBUS)
- 15 Connection for identification of the anaesthetic vaporizer
- 16 Not used
- 17 RS 232 C interface (MEDIBUS; only type-tested equipment and cables may be connected via this interface)
- 18 Equipotential bonding pin
- 19 Mains power socket

Dimensions

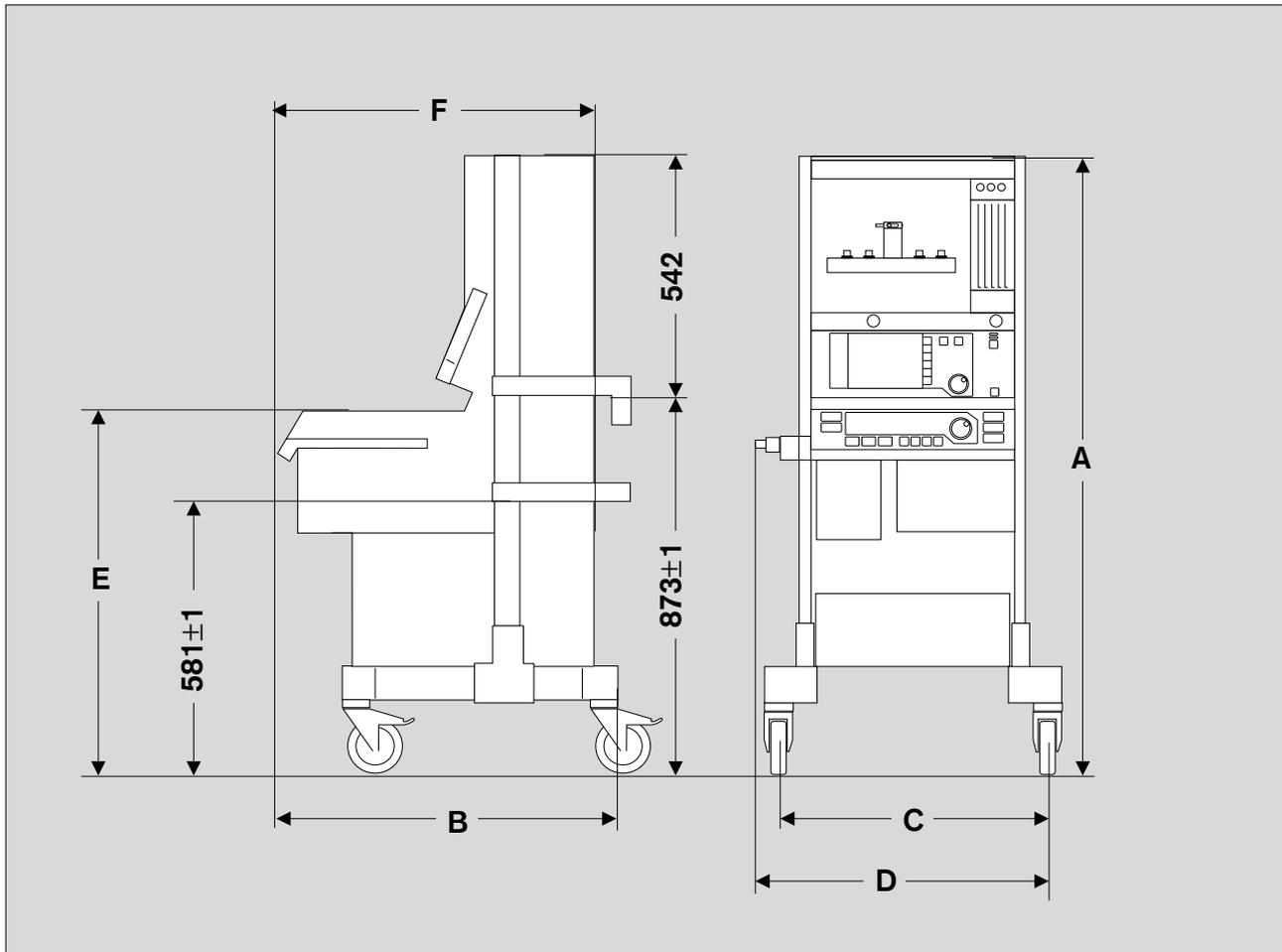

Standalone machine:

A = 1480 mm
B = 785 mm
C = 530 mm
D = 730 mm
E = 830 mm

Ceiling version:

A = 1415 mm
B = 785 mm
C = 530 mm
D = 730 mm
E = 760 mm
F = 900 mm

Weight

Cato basic machine approx. 140 kg
Cato with two Vapors and monitor approx. 180 kg

Technical data

Contents

	Page
General	116
Power supply	116
Ambient conditions	116
Medical gas supply	117
Breathing system	117
Ventilator	117
Gas delivery	118
Monitor	118
Pressure measurement	118
Flow measurement	118
O ₂ measurement in main stream (inspiration line)	118
O ₂ , CO ₂ and anaesthetic agent measurement in sidestream (sampling)	119
SpO ₂ measurement	120
Data communication	120
C-Lock™ ECG synchronization	121
Operating parameters	121


Where tolerances are given in percent and as absolute values,
 the higher value applies in each case!

General

- Identification: The Serial No. and Article No. are located on the rating plate (nameplate) on the back of the machine. The Article No. also specifies the equipment level of the machine.
- Weight: Cato basic machine approx. 140 kg
 Cato with two Vapors and monitor approx. 180 kg
- Dimensions.....: See dimensional drawings on page 114

Power supply

- Rated operating voltages: depending on power adapter
 - 100 V_{AC} +10 %, -15 %, 50/60 Hz
 - 120 V_{AC} ±10 %, 50/60 Hz
 - 127 V_{AC} ±10 %, 50/60 Hz
 - 230 V_{AC} ±10 %, 50/60 Hz
 - 240 V_{AC} ±10 %, 50/60 Hz
 Conversion only by specially trained personnel.
- Power input: Max. 280 W (without auxiliary power sockets and without accessories)
- Fuses: The fuses in the power adapter and master switch on contactor are only accessible to qualified personnel with special tools.
 Auxiliary power sockets (3x): 2 fuses installed per socket (2A DIN 41 622, 230/240 V_{AC})
- Auxiliary mains sockets: Do not connect any HF surgery devices to the auxiliary mains sockets!
 The connection of devices to the auxiliary mains socket may result in increased risk of electric shock in the event of earthing failure!

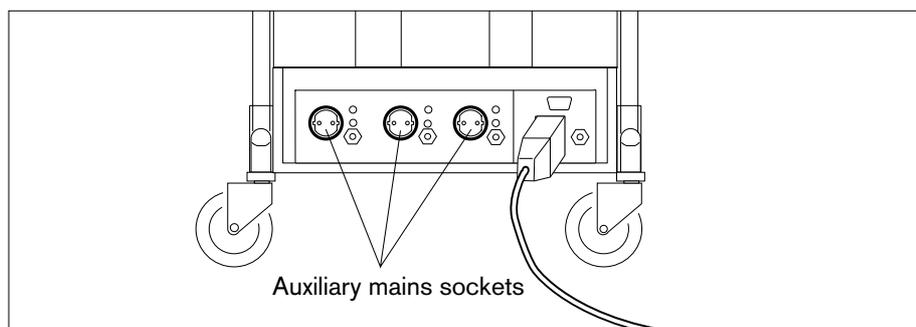

Ambient conditions

- Temperature: + 15 to +35 °C (operation)
 -20 to +50 °C (storage / transport)
- Humidity: 20 to 80 %, no condensation (operation)
 0 to 98 %, no condensation (storage / transport)
- Air pressure.....: 80 to 106 kPa (operation)
 50 to 110 kPa (storage / transport)

Medical gas supply

Pneumatic connections.....:	Piped medical gas supply – O ₂ , N ₂ O, AIR Spare gas cylinders 11 L or 3 L for O ₂ and N ₂ O (optional) Anaesthetic gas scavenging
Required pressures (piped supply):	O ₂ : 2.7 to 5.5 bar N ₂ O : 2.7 to 5.5 bar AIR : 2.7 to 5.5 bar
Maximum gas consumption.....:	O ₂ : 100 L/min at 5 bar ± 10%, including 50 L/min at 5 bar ± 10% for O ₂ flush N ₂ O : 20 L/min at 5 bar ± 10% AIR : 50 L/min at 5 bar ± 10%
O ₂ shortage signal	Whistling signal tone. Triggered: at latest when O ₂ pressure drops below 170 kPa (1.7 bar) Stops: at latest when O ₂ pressure rises above 260 kPa (2.6 bar)

Breathing system

Gas volume	Approx. 3 L total enclosed gas volume
Compliance	Approx. 3 mL/mbar Approx. 4 mL/mbar with absorber
Absorber volume	1.5 L soda lime Dräger sorb 800 Binds approx. 150 L CO ₂ , sufficient for approx. 6 hours of operation
Pressure relief	90 ± 10 mbar
Pressure limiting valve	Variable between 5 and 70 mbar, ± 15% of set value
Patient connections	ISO cone 22 mm ⌀
Anaesthetic gas scavenging	Nominal socket diameter 27 mm or ISO cone 30 mm ⌀ (≅ EN 1281-1)
Leakage ventilation system	< 120 mL/min at 30 mbar
Maximum pressure in breathing system	10 to 80 mbar ± 20% or +3 mbar, but at least 10 mbar over PEEP
Expiratory and inspiratory resistance.....:	≤ 4 mbar with a flow of 60 L/min (according to EN 740)

Ventilator

Drive gas consumption	2 L/min O ₂ or AIR in operation 0 L/min in »standby« mode
Tidal volume (V _T)	< 20 mL ± 30 % or ± 6 mL 20 to 100 mL ± 10 % or ± 10 mL 100 to 1400 mL ± 5 % or ± 15 mL
Frequency (f _{IPPV})	6 to 80 per minute
Minute volume (V̇ _E).....:	to 25 L/min
T _I :T _E	1:3 to 2:1 ± 5 %
T _I P:T _I	0 to 60 % ± 5 %
Inspiratory flow V̇ _E max.....:	max. 75 L/min
Frequency (f _{IMV})	3 to 80 per minute ± 5 % (f _{IMV} < f _{IPPV})
PEEP	0; 2 to 20 mbar ± 10 % or ± 2 mbar (P _{max} – PEEP > 10 mbar)
Trigger pressure.....:	-1 mbar ± 0.5 mbar (invariable)

Gas delivery

Low-flow measuring tubes: (Calibrated for 20 °C, 1013 mbar)

O ₂ : 0.02 to 0.5 L/min	±10 %
At minimum scale:	+20 %, -10 %
At minimum scale:	±5 %
O ₂ : 0.55 to 10.0 L/min	±10 %
At minimum scale:	+20 %, -5 %
N ₂ O: 0.02 to 0.5 L/min	±10 %
At minimum scale:	-20 %, +10 %
At minimum scale:	±5 %
N ₂ O: 0.55 to 10.0 L/min	±10 %
At minimum scale:	+20 %, -5 %
AIR: 0.2 to 14.0 L/min	±10 %
At minimum scale:	+15 %, -5 %

Delivery output.....: O₂, N₂O, AIR: at least 9 L/min each

Monitor

Pressure measurement

Airway pressure.....: -10 to 80 mbar

Resolution: 1 mbar

Accuracy: at least 1 mbar or better than ±4% of the measured value

Flow-Messung

Tidal volume VT.....: Range: 0.01 to 9.99 L
Resolution: 0.01 L
Accuracy: better than 0.01 L or ±8%¹⁾ of the measured value
(under calibration conditions and 1013 hPa)

Minute ventilation AMV: Range: 0 to 99.9 L/min
Resolution: 0.1 L/min
Accuracy: better than 0.2 L/min or ±8%¹⁾ of the measured value under
calibration conditions and BTPS (BTPS = Body Temperature,
present atmospheric Pressure, 100% humidity Saturated)

Respiration rate: Range: 0 to 80 per minute
Resolution: ±1 per minute
Accuracy: ±1 per minute
1) When using an microbial filter on the expiratory side, the tolerance
increases to ±15%.

O₂ measurement in main stream (inspiration line)

Range.....: 5% to 100% by volume

Resolution: 1% by volume

Accuracy: calibrated with air: ±3 % by vol. in 5 to 50% by vol. measuring range
±5 % by vol. in 50 to 100% by vol. measuring range
calibrated with 100 % O₂: ±3% in the 5 to 100% by vol. measuring range

Response time t_{10...90}.....: better than 15 seconds

**Technical data
Monitor**

Display range for sevoflurane: 0% to 11% by volume
 Accuracy: 0 to 5% better than $\pm 0.2\%$ by volume absolute
 5 to 9% better than $\pm 0.4\%$ by volume absolute
 Resolution: 0.1 % by volume absolute

Response time $t_{10 \dots 90}$: At 200 mL/min: 450 ms
 At 60 mL/min: 1.2 s

Warm-up phase: 8 minutes

Zero drift: Within the specified accuracy, without limitation in time

SpO2 measurement

Display range: 0 to 100% SpO2
 Accuracy (adults): 70 to 100% SpO2, better than $\pm 2\%$ SpO2
 50 to 70% SpO2, better than $\pm 3\%$ SpO2
 0 to 50% SpO2 not specified
 Accuracy (neonates): 70 to 95% SpO2, better than $\pm 3\%$ SpO2
 0 to 70% SpO2 not specified
 95 to 100% SpO2 not specified
 Resolution: 1 % SpO2


Pulse rate: 20 to 250 per minute
 Accuracy: ± 2 per minute
 Resolution: 1 per minute

Sensors: Type compatible with Nellcor sensors Oxisensor, Oxiband and Durasensor
 Wavelengths: 660 nm (red), 920 nm (infrared)


Acoustic pulse signal: A tone is generated for each pulse beat detected.
 Pitch proportional to oxygen saturation.

Data communication


Data interface: RS-232-C (MEDIBUS)
 Connector: 25-pin sub-D, electrical isolation 1.5 kV
 Pin assignment: 1 \cong shield, 2 \cong TxD, 3 \cong RxD, 7 \cong GND

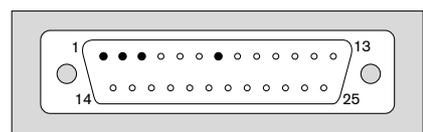

Record (printer):
 (can also be configured as RS 232 C (MEDIBUS))
 Connector: 25-pin sub-D, electrical isolation 1.5 kV
 Pin assignment: 1 \cong shield, 2 \cong TxD, 3 \cong RxD, 7 \cong GND

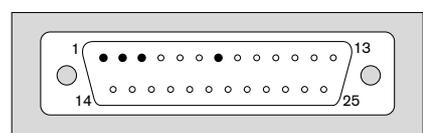

Analog outputs:
 CO2: 0 to 10 kPa \cong 0 to 10 V
 Connector: 9-pin sub-D, electrical isolation 1.5 kV
 Pin assignment: 1 \cong shield, 3 \cong positive (+), 4 \cong negative (-)

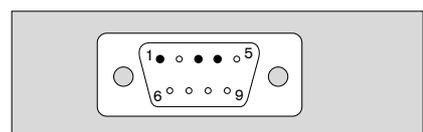


C-Lock™ ECG synchronization (optional)

Prerequisite for the
ECG synchronization signal: Pos. pulse with > 4.5 V and > 10 ms duration to drive 2 mA.

Max. permitted signal delay
in relation to the current
QRS complex: 40 ms

Signal isolation from
remaining electronics: Dielectric strength > 4 kV

Socket for 2-pin pawl-type
connector, dia. 3.5: Pin assignment:

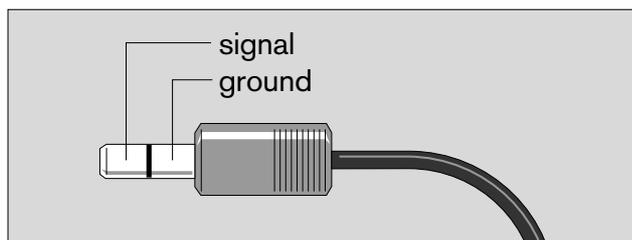

Operating parameters

Noise emission : Max. 56 dB(A) (as determined in a free field over a reflecting surface)

Protection classes:

SpO2 sensor : Type BF  electrically isolated from protective earth conductor

Electromagnetic compatibility : Tested to EN 60601-1-2

Classification : Class II b
as per Directive 93/42/EEC
Annex IX

UMDNS-Code : 10-134
Universal Medical Device
Nomenclature System

Technical documentation available on request

Descriptions

Contents

	Page
Flow chart of self-test	124
Ventilation with automatic adjustment of rebreathing to match fresh gas flow	126
Automatic leakage tests IPPV / MAN	127
Automatic compliance correction	127
Notes on setting fresh gas flow	128
Relationship between fresh gas and gas concentration in breathing system	128
Relationship between fresh gas and time constants in breathing system	128
Principles of measurement	129
O ₂ measurement	129
Flow measurement	129
CO ₂ and anaesthetic agent measurement	129
SpO ₂ measurement	131
Temperature measurement	132
Pressure measurement	132
Definition of low-flow and minimal-flow anaesthesia	132
SIMV, Synchronised Intermittent Mandatory Ventilation	133
S-ORC, O ₂ Ratio Control	135
Safety features of gas supply	136
Notes for reducing condensation	136
Operation of the ceiling version	137
Cato with ceiling suspension unit DVE 808X	137
General requirements	137
Connecting the Cato ceiling version	138
Disconnecting the Cato ceiling version	139
What to do if problems arise	140
Additional maintenance and care instructions	140
Abbreviations used	141
Symbols used	142
Index	143



Flow chart of self-test

Flow chart of self-test in ventilator

Legend:

- A Display in dialogue window of ventilator
- F Indicated faults must be remedied, instructions and advisories observed.
- I Information in dialogue window of ventilator
- Press rotary control
- # Correspondingly numbered test blocks in VT window

Test sequence *):

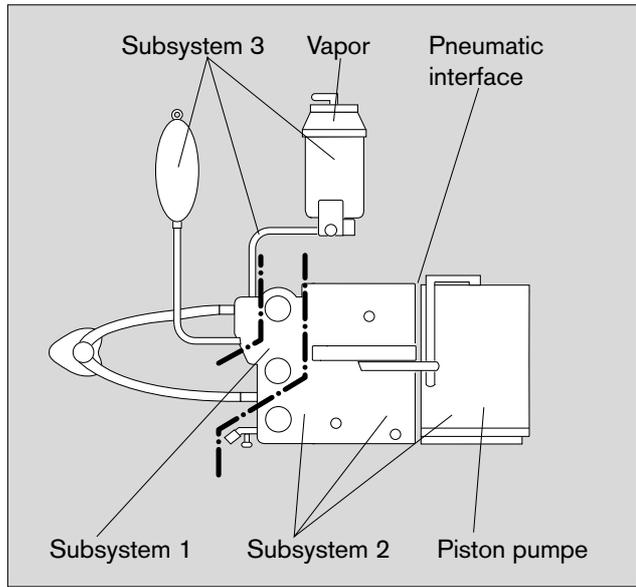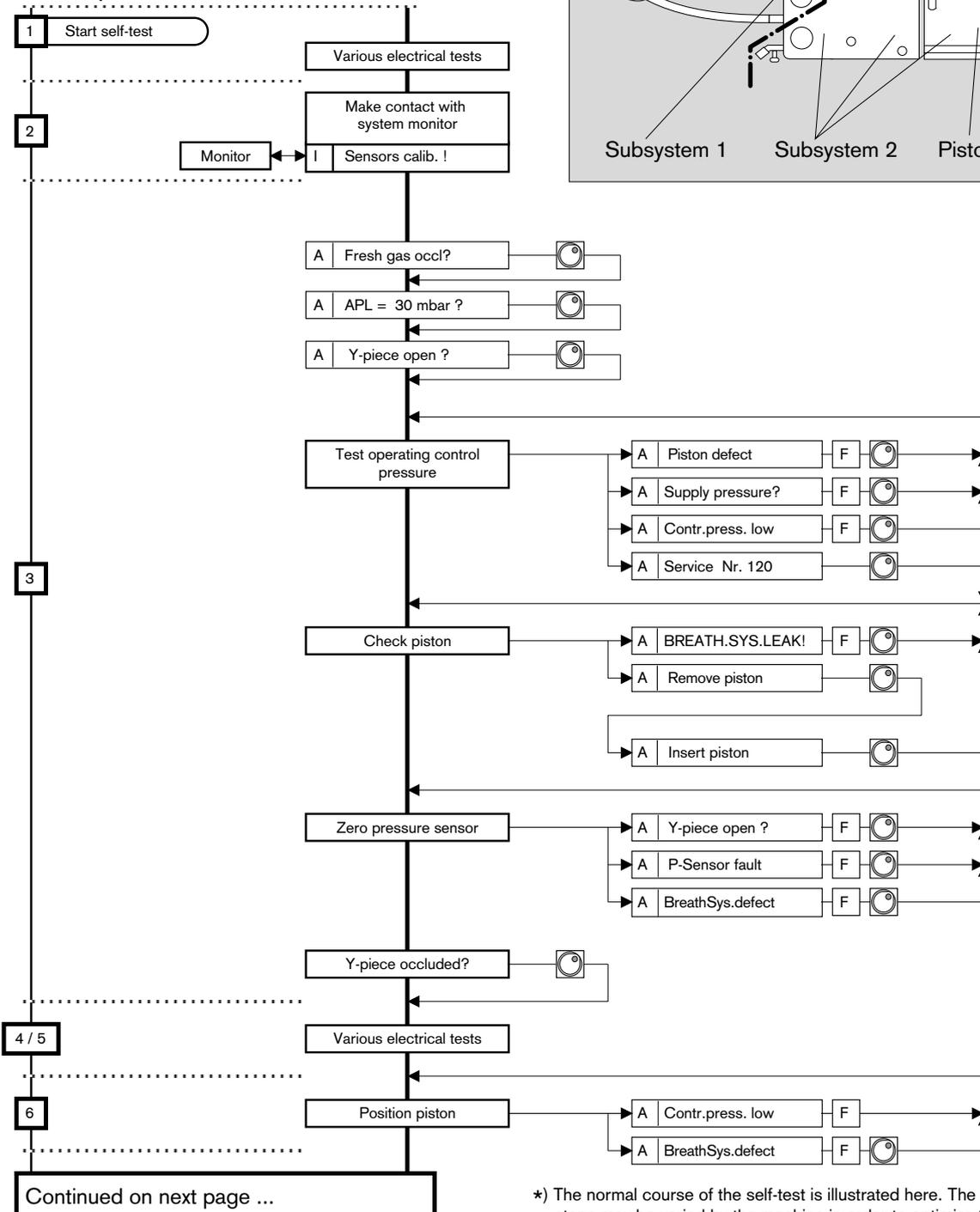


07262871

*) The normal course of the self-test is illustrated here. The order of test steps may be varied by the machine in order to optimize the test time.

Flow chart of self-test in ventilator (cont.)

Test blocks:

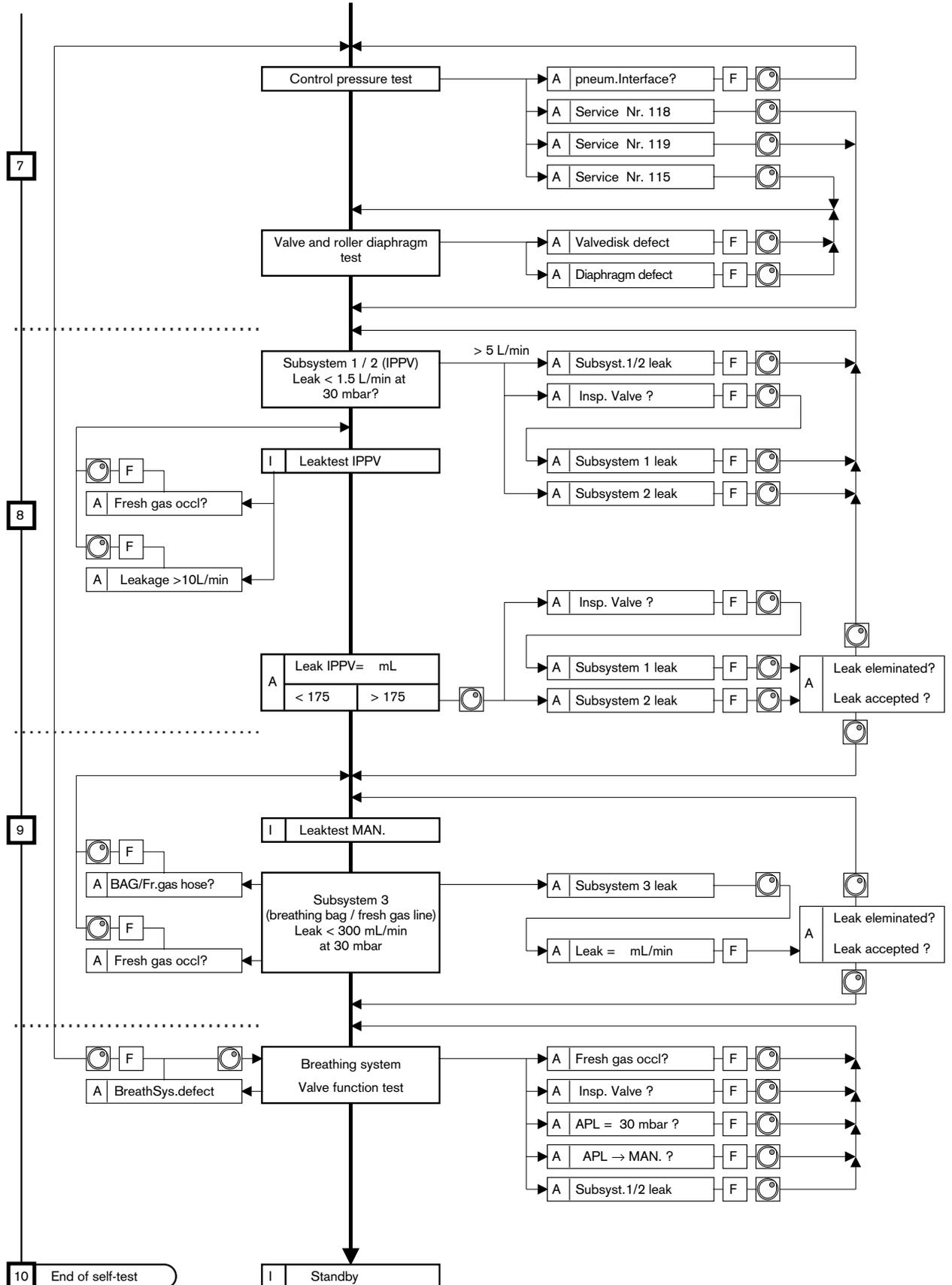

Descriptions

Ventilation with automatic adjustment of rebreathing to match fresh gas flow

Ventilation with automatic adjustment of rebreathing to match fresh gas flow

Most of the breathing systems used for anaesthesia today are based on the rebreathing principle. Part of the expired gas is redelivered to the patient after absorbing CO₂ and enriching with anaesthetic gases and anaesthetic agent. The excess anaesthetic is scavenged, the amount of scavenged anaesthetic gas essentially depending on the set fresh gas flow.

Administration of anaesthetic gas with reduced fresh gas flow (low-flow technique) yields a number of major advantages: lower consumption of anaesthetic gases and agents, more effective humidification and heating of the inspiratory gas, lower environmental burden and good manual ventilation properties.

The design of the breathing system is an aspect of essential importance for low-flow anaesthesia. The high degree of fresh gas utilization is a major prerequisite. Systems suitable for low-flow techniques should be designed so that it is impossible, firstly, for too much expiration gas to disappear in the anaesthetic scavenging line without building up a constant pressure and, secondly, for fresh gas to escape without first having been administered to the patient.

In closed anaesthesia systems, anaesthetic gas cannot escape from the breathing system and no more fresh gas is delivered than is actually required by the patient. However, closed systems must also meet additional requirements:

the breathing system must be absolutely tight and must feature additional monitoring and control elements.

The breathing system implemented in the Cato automatically matches its degree of openness to the fresh gas flow.


During inspiration, breathing gas streams from the piston pump to the patient. Valve V2 of the excess gas outlet and the fresh gas shutoff valve V1 are closed. Expiration is initiated when the fresh gas shutoff valve V1 is opened.

Expired gas from the patient's lungs streams into the breathing bag which serves as a reservoir and also into the retracting piston pump. The excess gas outlet valve V2 is closed.

Unlike the case with conventional semi-closed breathing systems, the valve opening time for discharging excess gas is controlled as required. The system remains open longer for anaesthesia with high fresh gas flow.

If the fresh gas flow is inadequate in closed anaesthesia systems, the pressure measuring function will detect that the patient's expiratory pressure has dropped below approx. -3 mbar.

This shortage of fresh gas is signalled by the Cato and the piston pump stops in order to avoid a negative pressure in the patient.

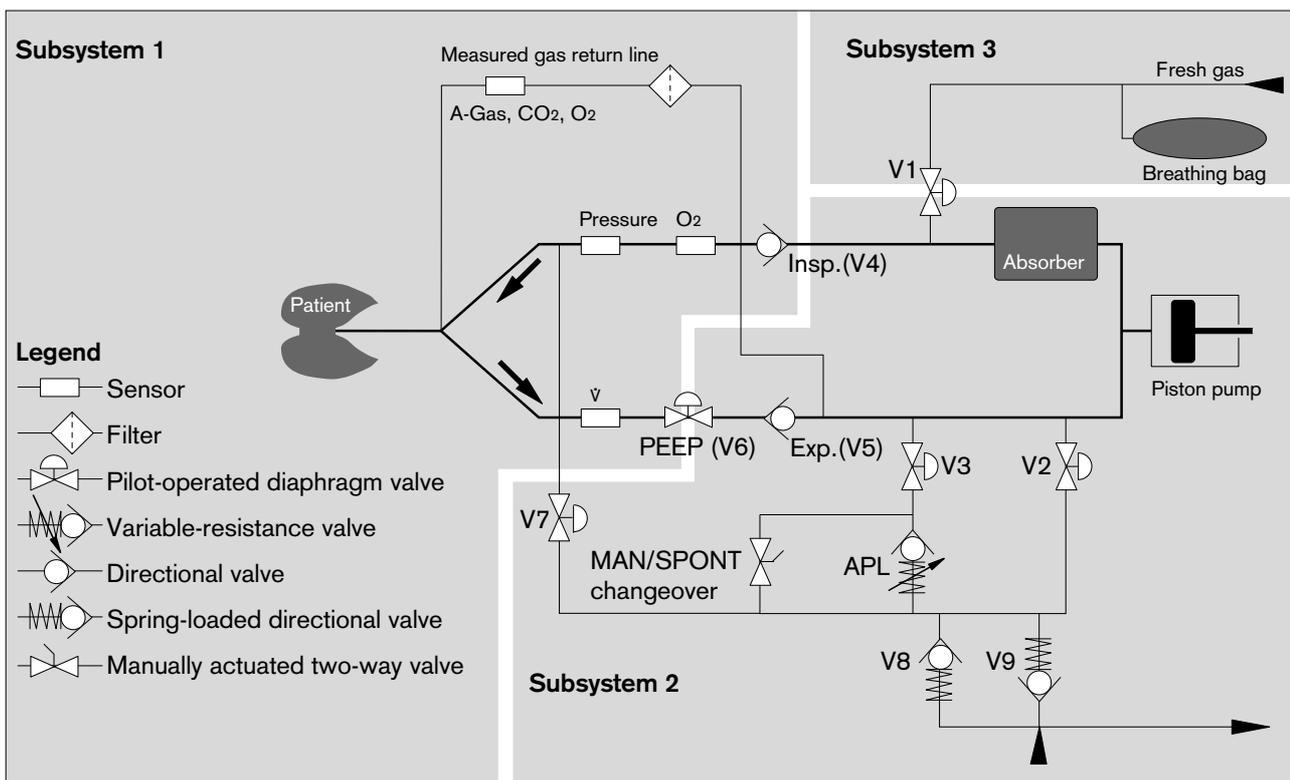

Automatic leakage test IPPV

This leakage test identifies any leaks of relevance for automatic ventilation in subsystems 1 and 2 of the Cato ventilation system. It also encompasses the breathing hoses up to the Y-piece, as well as the measured gas sampling and return lines if installed. The overall system compliance is determined at the same time.

The IPPV leakage test and the leakage test started in »standby« mode is carried out by building up a constant pressure of 30 mbar. The piston movement necessary to compensate the gas escaping through leaks is measured and calculated and indicated as a volume per unit time.

The effective leakage over the complete ventilation cycle is lower than the value indicated, since the effective mean pressure P_{mean} in IPPV ventilation mode is considerably lower than the test pressure.

The relationship depends on the rate of pressure increase, the plateau time and the ratio $T_i:T_E$. The effective leakage value varies with the value measured in the leakage test as $P_{mean} : P_{test}$.

Example:

Test leakage = 30 mL/min
 P_{test} = 30 mbar
 P_{mean} = 6 mbar

Effective leakage = Test leakage x P_{mean} / P_{test}
Effective leakage = 6 mL/min

Automatic MAN leakage test

This leakage test is also part of the self-test and locates any leaks of relevance for manual ventilation in subsystem 3 of the Cato ventilation system. The breathing bag, fresh gas hose, Vapor and internal connections up to the bank of measuring tubes are tested for leaks.

The test is normally carried out at a pressure of 30 mbar. If the leakage value remains below 300 mL/min, this is not indicated and the self-test continues. This subsystem contributes only marginally to the overall leakage, since the mean pressure is normally below 5 mbar.

Automatic compliance correction

The stroke volume applied by a ventilator not only ventilates the patient's lungs, but also the hose system connecting the patient to the ventilator.

This means that only part of the stroke volume is effectively used to ventilate the lungs, the rest remaining in the compressible hose volume. This compressible volume must be known for ventilation to be effective.

Example:

A patient with a lung compliance of 3 mL/mbar requires a tidal volume of $V_T = 60$ mL.

Disregarding the compressible hose volume, this would yield an effective airway pressure of

$$P = \frac{V_T}{C_1} = \frac{60 \text{ mL}}{3 \text{ mL / mbar}} = 20 \text{ mbar.}$$

The actual compressible hose volume (hose system) equals 3 L, however, which corresponds to a compliance of $C_2 = 3$ mL/mbar.

The actual airway pressure is therefore:

$$P = \frac{V_T}{C_1 + C_2} = \frac{60 \text{ mL}}{(3 + 3) \text{ mL / mbar}} = 10 \text{ mbar}$$

In other words, 30 mL stroke volume ventilate the lungs, the other 30 mL remaining in the hoses.

The compliance of the breathing system (breathing gas block, soda lime container, hoses, etc.) is determined during the leakage test and saved by the Cato.

This calculated compliance value is used to calculate the volume stored in the breathing system and hoses for each ventilation pressure. In order to correct it, the Cato starts with the set tidal volume and reaches the correct volume after three to six breaths. The corrected volume is constantly verified automatically.

The measured value must be limited to plausible ranges for safety reasons. This limit is set at 3.9 mL/mbar when using adult hoses (tidal volumes greater than 200 mL) and at 0.8 mL/mbar when using infant hoses (tidal volumes less than 200 mL). The maximum length of the breathing hoses should therefore not be exceeded (see table below).

Table of maximum hose lengths:

Hose	Maximum length	
	with filter	without filter
Adults; black	3.0 m	3.5 m
Adults; blue	6.5 m	7.0 m
Infants; black	2.2 m	4.4 m
Infants; blue	2.2 m	4.4 m

Descriptions

Notes on setting the fresh gas flow

Relationship between fresh gas flow and gas concentration

Relationship between fresh gas flow and time constants

Notes on setting the fresh gas flow

The fresh gas delivered must

- cover the patient's consumption (uptake) and
- any leaks in the system.
- It must also compensate the gas sampled by the system monitor if the measured gas return line is not connected.

This is particularly important when setting a low fresh gas flow.

The patient's uptake of gas depends on the anaesthesia and primarily comprises the consumption of O₂ and nitrous oxide. The consumption of O₂ can be calculated approximately using Brody's equation:

$$\text{O}_2 \text{ flow} = 10 \times \text{BW}^{0.75} \quad \text{in mL/min}$$

$$\text{BW} = \text{Body weight} \quad \text{in kg}$$

This corresponds to an O₂ uptake of roughly 3.5 mL/min per kg body weight.

Higher O₂ consumption results in a lower inspiratory O₂ concentration due to rebreathing when a low fresh gas flow is set.

- The uptake of nitrous oxide varies with time and can be approximated by the following rule of thumb:

$$\frac{1}{\sqrt{t}}$$

Approximate steady state value: 1.5 mL/min N₂O per kg body weight.

- The leakage from the ventilation system depends on the airway pressure (mean value) and can be determined with the aid of the automatic leakage test.
- The measuring gas sampled to measure the CO₂, N₂O and anaesthetic agent can be set on the system monitor (60 or 200 mL/min).
- The gas is returned to the breathing circulation via the measured gas return line. This would reduce the required fresh gas by 200 mL/min in the following example.

Example

for estimating the minimum fresh gas flow required in the steady state:

Body weight	=	70 [kg]:	
<hr/>			
\dot{V} O ₂	=	240 [mL/min]	
\dot{V} N ₂ O	=	100 [mL/min]	+
Leakage	=	10 [mL/min]	+
Sampled gas	=	200 [mL/min]	+
<hr/>			
Fresh gas required	≥	550 [mL/min]	

This example shows that the fresh gas flow (O₂ and N₂O) must be set to at least 550 mL/min.

Relationship between fresh gas flow and gas concentration in breathing system

The inspiratory concentration differs from the set fresh gas concentration due to rebreathing and the O₂, N₂O and anaesthetic agent uptake by the patient. The lower the fresh gas flow, the larger the concentration gradient between fresh gas, inspiratory and expiratory gas concentration becomes.

Since the concentration of the fresh gas flow in this flow range bears little resemblance to the concentration at the patient, it is important to measure the anaesthetic agent concentration as close to the inspiration tube as possible in this mode!

The measuring system is integrated into the system monitor.

Relationship between fresh gas flow and time constants in breathing system

The response time following a change of concentration of O₂, N₂O or anaesthetic agent in the fresh gas depends on the set fresh gas flow.

The inspiratory concentration in the breathing system corresponds more and more accurately with the fresh gas concentration as the fresh gas flow increases. At a low fresh gas flow, a change of concentration takes effect in the breathing system very slowly. This process can be speeded up by increasing the fresh gas flow abruptly.

Rule of thumb for estimating the system response over time:

$$T = VC / \dot{V}FG$$

where:

T = Time constant of the system in minutes

VC = System volume in litres
 (breathing system, ventilation hoses,
 residual volume of the lungs)

$\dot{V}FG$ = Fresh gas flow in L/min

Example

for estimating the time constant of the system (comprising breathing system and hoses):

$$VC = 6 \text{ [L]}$$

$$\dot{V}FG = 3 \text{ [L/min]}$$

$$T = 2 \text{ [min]}$$

In this example, the change of concentration in the breathing system would have reached roughly 60% of the change in fresh gas concentration after approx. 2 minutes.

The current flowing through the cell is proportional to the partial oxygen pressure in the gas mixture to be measured.

At constant pressure and constant temperature of the gas mixture to be measured, the measured value is directly proportional to the partial oxygen pressure.

Flow measurement

Measuring principle and signal processing:

The sensor is based on the principle of a constant-temperature hot-wire anemometer. The breathing gas flows round a very thin, electrically heated platinum wire in a measuring tube. The wire is heated to a constant temperature of 180 °C which is controlled by a control circuit. Heat is dissipated when gas flows past this wire. The larger the volume of gas flowing past per unit time, the more heat will be dissipated.

The heating current required to maintain a constant wire temperature is taken as an indicator for the gas stream.

Gas type compensation:

The effect of the various types of gas contained in the breathing gas is compensated by a second heated platinum wire. The heat dissipated by the second wire in the stationary gas column in the measuring tube is determined during a period in which there is no gas flow (i.e. during inspiration when the sensor is positioned on the expiration side). The gas composition is determined on the basis of the specifically different thermal conductivity of the types of gas present in the breathing gas.


Linearization is performed with the aid of internal calibration tables for the gas mixtures O₂/N₂O, air and 100% O₂.

Principles of measurement

O₂ measurement (Measuring principle of the galvanic cell)

The O₂ sensor is based on the principle of a galvanic cell.

Oxygen molecules from the gas mixture to be measured diffuse through a plastic membrane into the electrochemical cell and are reduced on precious metal electrodes.

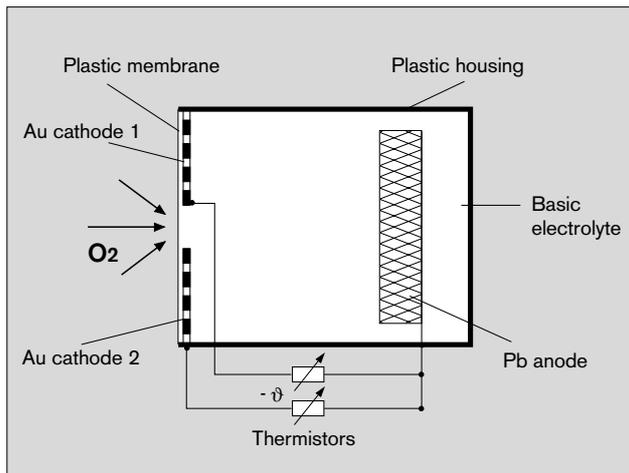

A base electrode is oxidized at the same time. It is depleted by the oxidation process and essentially determines the service life of the sensor.

Measurement of CO₂ and anaesthetic agent

CO₂ and anaesthetic agent absorb infrared light. A pump entrains a small amount of breathing gas through a measuring cuvette irradiated with infrared light. Different filters make it possible to select a frequency band in which only one of the gases to be identified is absorbed. All gases can be measured quasi continuously by changing filters rapidly.

The absorption reflects the gas concentration in the cuvette. The gas concentrations in the breathing gas can be calculated by simultaneously measuring the temperature and absolute pressure in the cuvette.

Descriptions
Principles of measurement

Cross-sensitivity of the anaesthetic gas measurement: Measurement of the anaesthetic agent can be falsified by vapours of organic substances (such as those contained in cleaning agents or disinfectants) in the air round about, the sampling hose or the T-piece. Elevated values for anaesthetic agent will be displayed, particularly when using halothane, if the patient's breathing air contains alcohol.

Mixtures of different anaesthetic vapours may considerably impair the accuracy with which the concentration is measured!

Disturbance variables in sampled gas measurement

When analysing the measured values, the temperature, humidity and pressure conditions during measurement must be taken into account.

Whereas calibration takes place with dry gas under NTPD conditions (Normal Temperature 20 °C, Pressure 1013 hPa, Dry - relative humidity 0 %), the measurements during patient monitoring are taken by sampling the gas under BTPS conditions (Body Temperature 37 °C, ambient Pressure, Saturated - relative humidity 100 %).

The gas sampling and measurement process creates a negative pressure of about 100 to 200 mbar compared to ambient pressure at the concentration measurement site (depending on sample flow, condensation and water separator). The partial pressure measured at the sensor is corrected to the current ambient pressure with the aid of the pressure measured in the measuring cuvette.

Effect of temperature:

The gas temperature at the sensor is measured and its effect on concentration measurement is compensated.

Effect of humidity:

The gas sampled during expiration has a temperature of 37 °C and a relative humidity of about 100 %.

It contains about 47 mmHg water vapour. Up to the water trap, the gas cools down to approximately ambient temperature. The water vapour content is reduced to e.g. 17 mmHg at approx. 20 °C. The difference condenses in the sampling hose and is separated in the water trap. Consequently, the volume at sea level is reduced by $30 : 760 = 4 \%$, thereby increasing the measured relative gas concentration by 4%. This error is not corrected in the PM 8050 because it is small compared to the specified accuracy of the sensors.

Example:

Gas	Concentration at the Y-piece	Displayed value
O ₂	30%	31%
N ₂ O	57%	59%
Isoflurane	2%	2%
CO ₂	5%	5%
Water vapour	6%	—


SpO₂ measurement

The light absorption properties of oxygenated arterial blood (oxyhaemoglobin HbO₂) differ from those of unsaturated venous blood (reduced haemoglobin Hb).

O₂ saturation is a logarithmic function of the irradiated light intensity (Lambert-Beer's law).


The effect of such dyshaemoglobins as carbon monoxide haemoglobin HbCO and methaemoglobin MetHb is normally negligible.

The sensor comprises two light-emitting diodes which alternately emit infrared and red light at typical wavelengths of 920 nm and 660 nm respectively. The radiation intensity is measured by a photodetector opposite the diodes. The sensor is positioned on a limb in which the arterial blood vessels can be irradiated, such as a finger, toe or the nose.

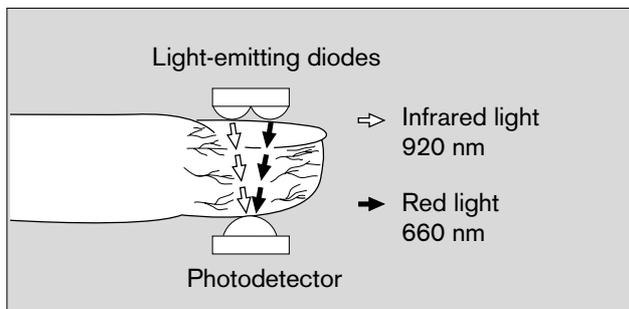


These two wavelengths – 920 nm and 660 nm – are used because meaningful absorption values are still obtained for oxygenated and reduced blood, even in the presence of slight perfusion, and because they differ significantly. The light alternately emitted by the diodes is completely absorbed by the pulsating arterial blood, the skin, finger nails, muscular tissue, bones and venous blood.

Except for the pulsating arterial blood, the amount of light absorbed by the other components remains constant as regards the quantity and optical density over a defined unit of time.

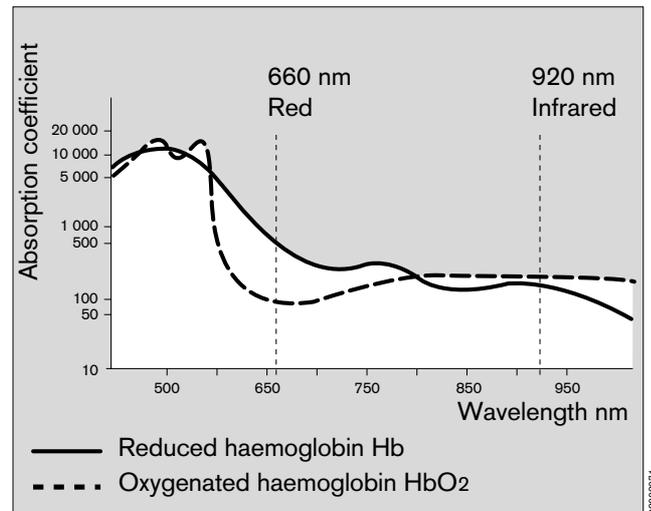

The arterial blood pulsating with every beat of the heart, however, produces a change of volume synchronous with the pulse in the irradiated tissue. In other words, absorption of the irradiated light also changes in time with the pulse.

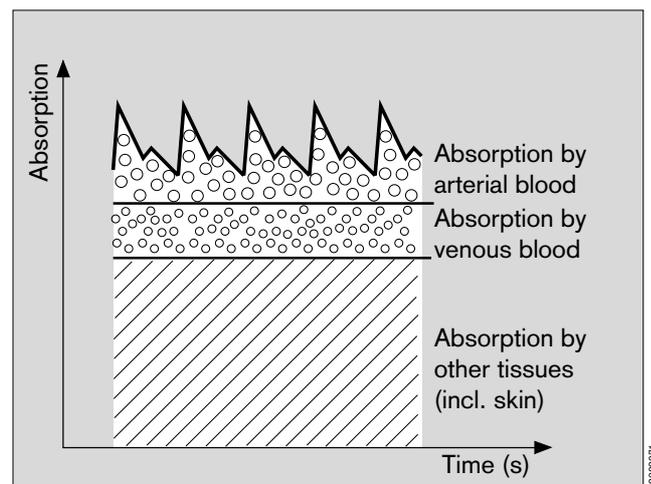

The light absorbed when there is no pulsating blood (during the diastole) is determined first. This yields the amount of light absorbed by tissue and non-pulsating blood.


The absorption value does not normally change during the pulse phase and provides a reference value for the pulsating part of the absorption.

Following the next beat of the heart, the absorption is measured again when the pulsating blood enters the tissue. The absorption of light changes for both wavelengths, due to the pulsating arterial blood.

Descriptions

Principles of measurement

Definitions for »low-flow« and »minimal flow« anaesthesia

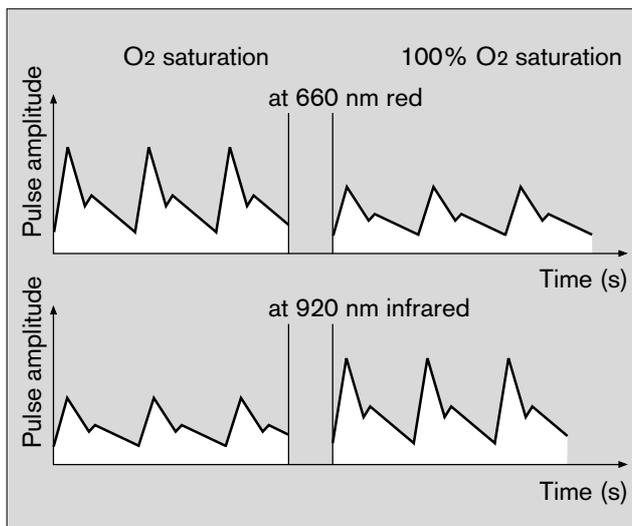

The diagram above shows an example of the light absorbed by the blood at 660 nm (red) and 920 nm (infrared).

At 660 nm, the absorption and corresponding pulse amplitude decrease with increasing O₂ saturation, but rise at 920 nm. Since the absorption coefficients of HbO₂ and Hb are known for both wavelengths, the system can calculate how much of these two haemoglobins is present. The quotient obtained by dividing the oxygenated haemoglobin (HbO₂) by the reduced and oxygenated haemoglobin (Hb + HbO₂) is known as the functional saturation:

$$\% \text{ SpO}_2 (\text{func}) = 100 \cdot \frac{\text{HbO}_2}{\text{HbO}_2 + \text{Hb}}$$

and refers to the haemoglobin capable of transporting oxygen.

Dyshaemoglobins, HbCO and MetHb are normally negligible, but may affect the accuracy of the measurement.

Temperature measurement

Temperature-dependent change in resistance of an NTC resistor (NTC = negative temperature coefficient) with linearization circuit.

Pressure measurement

Principles of measurement:

Piezoresistive change of resistance in a membrane.

Determination of PEEP and plateau pressure:

PEEP (positive end-expiratory pressure) is the airway pressure at the end of expiration.

P_{lat} (plateau pressure) is the airway pressure measured 16 milliseconds before expiration begins.

Definitions for »low-flow« and »minimal flow« anaesthesia

Low-flow anaesthesia is performed with a fresh gas flow considerably below the minute ventilation. When setting such low fresh gas volumes, the anaesthetic gases must be returned to the patient via a semi-closed or closed rebreathing system.

The rebreathing volume increases when the fresh gas flow is reduced and the excess gas volume decreases correspondingly.

Although the fresh gas flow can only be infinitely reduced to the gas volume taken up by the patient at a given moment of anaesthesia in a completely hermetic system, a distinction is nevertheless made between the following methods:

The fresh gas flow is reduced to 1 L/min for **low-flow anaesthesia** and to 0.5 L/min for **minimal-flow anaesthesia**.

In the case of **non-quantitative anaesthesia** in a closed system, the gas delivery settings are corrected frequently to adjust the fresh gas volume in line with the volume of gas taken up by the patient so that the internal pressure and charge of the breathing system do not decrease and the ventilation pattern remains unchanged.

In the case of **quantitative anaesthesia** in a closed system, the composition of the fresh gas corresponds exactly to the volumes of oxygen, nitrous oxide and inhalation anaesthetic taken up by the patient at a given moment in anaesthesia. This ensures that the composition of the anaesthetic gas also remains constant, in addition to the gas charge in the system and the ventilation pattern.

(Source: Baum, J.: »Die Inhalationsnarkose mit niedrigem Frischgasflow« (Inhalation anaesthesia with low fresh gas flow), published by Thieme, Stuttgart 1992)

SIMV Synchronized Intermittent Mandatory Ventilation

Mixture of mechanical ventilation and spontaneous breathing.


In SIMV mode, the patient can breathe spontaneously at specified regular intervals. Between these intervals, mandatory (i.e. automatically delivered) ventilation strokes ensure a minimum degree of ventilation.

The mandatory ventilation strokes are the same as those for IPPV ventilation. They are defined by the parameters V_T , IPPV frequency f_{IPPV} , $T_I:T_E$ and T_{IP} .

Each mandatory breath is followed by a pause in which the patient can breathe spontaneously.


In order to prevent the next mandatory breath being applied during the expiratory phase of spontaneous breathing, a trigger function ensures that the mandatory ventilation stroke is synchronized with the inspiratory spontaneous breathing phase during an expectation period.

The time between the end of each mandatory ventilation stroke and the beginning of the next is subdivided into a spontaneous breathing time T_{Spont} and a trigger time $T_{Trigger}$.

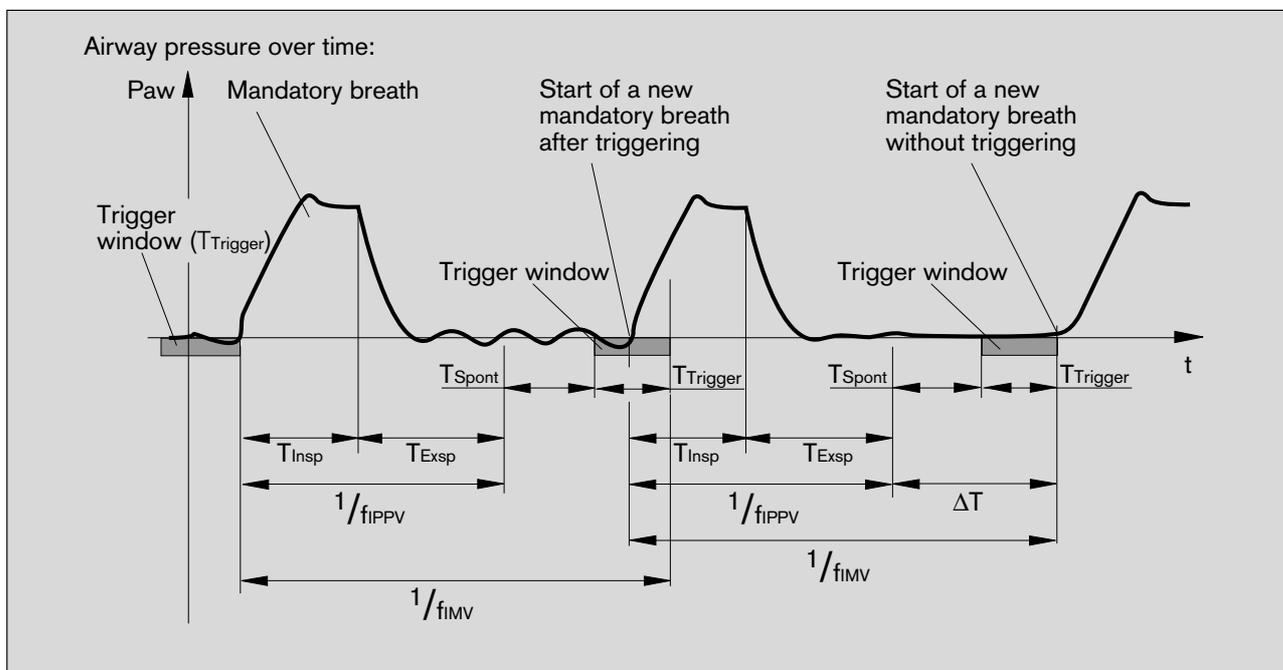

08528571

Descriptions
SIMV

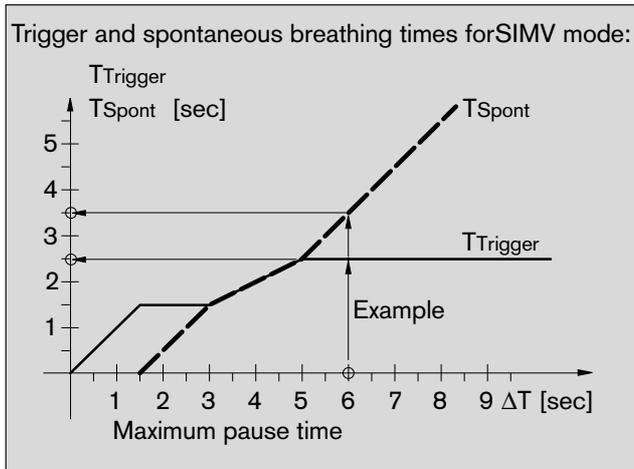

Example:

fIMV = 5 per minute

fIPPV = 10 per minute

$$\Delta T = \frac{1}{f_{IMV}} - \frac{1}{f_{IPPV}} = \frac{1}{5 \text{ per min.}} - \frac{1}{10 \text{ per min.}} = 6 \text{ s}$$

From the diagram we can see that:

T_{Spont} = 3.5 seconds and

T_{Trigger} = 2.5 seconds

During the trigger time, the system checks whether the airway pressure drops at least 1 mbar below the pressure measured at the end of the expiration phase.

The mandatorily applied minute volume may increase if an automatic ventilation stroke is applied at the beginning of each trigger period!

The duration of a mandatory stroke plus the spontaneous breathing time is calculated as follows:

$$\frac{1}{f_{IPPV}} + T_{Spont} = 6 \text{ s} + 3.5 \text{ s} = 9.5 \text{ s}$$

This corresponds to a frequency of approx. 6 per minute and the applied minute volume increases to 6 per minute · VT.

Oxygen ratio control - S-ORC

To ensure an O₂ concentration greater than 21% by volume in the fresh gas, the anaesthetic workstation has an O₂ ratio control system (S-ORC = Sensitive Oxygen Ratio Controller).



S-ORC limits the N₂O flow as a function of O₂ flow. Up to an O₂ flow of 200 mL/min, N₂O is cut off. N₂O is slowly released as from an O₂ flow of 200 mL/min. For O₂ flows greater than 300 mL/min, N₂O can be added, and the S-ORC prevents the O₂ concentration falling below 21% O₂ by volume.

The graph shows the range of adjustable O₂/N₂O ratios.

S-ORC is not an oxygen-specific monitoring system. By adding anaesthetic vapours (e.g. Desflurane up to 18% by volume), the O₂ concentration in the fresh gas is reduced.

Therefore:

Monitor the O₂ concentration with the breathing gas monitor.

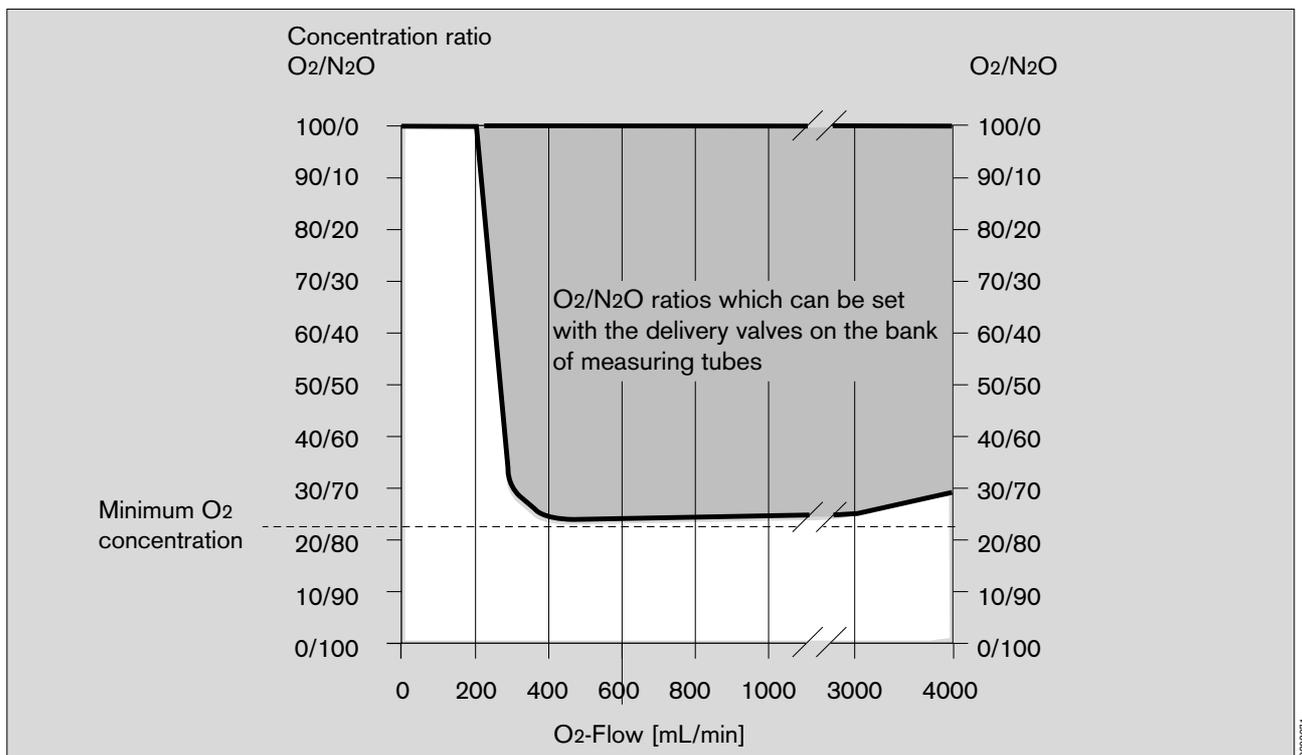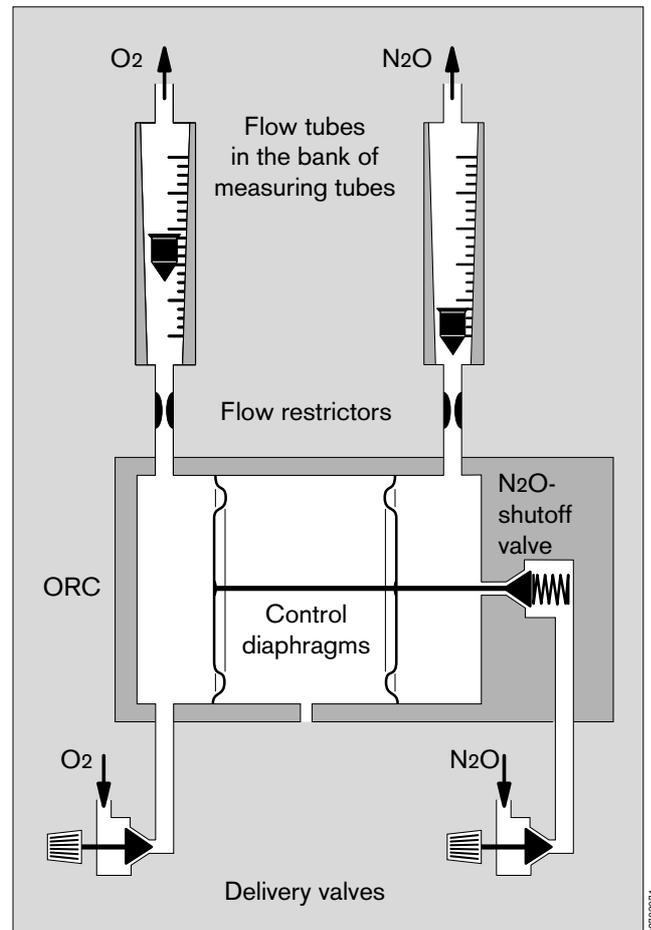

Descriptions

Safety features of gas supply

Notes for reducing condensation

Safety features of gas supply

N₂O lock and AIR changeover

If the O₂ supply fails during operation,

- the O₂ shortage signal sounds (pneumatic whistle),
- delivery of N₂O is stopped and
- «AIR» is automatically activated on the bank of measuring tubes, although the control lever is still set to »N₂O«.

Resumption of the O₂ supply pressure is detected by the machine and the former status is restored automatically.

Drive gas changeover

Approx. 2 L/min drive gas are required for the ventilator and approx. 12 L/min for the secretion aspirator. This gas is normally drawn from the compressed air supply.

If it is not available or has failed, the machine automatically switches over to the O₂ supply and draws its drive gas from there.

Notes for reducing condensation

Water vapour is liberated when CO₂ is absorbed by the soda lime.

The greater the proportion of rebreathed gas (low-flow technique), the more moisture is produced.

Cato has been optimized for this low-flow technique:

- The electric heating prevents condensation forming in the breathing system.
- A large water trough has been integrated into the base of the absorber pot to collect condensation.
- The large cross-sections of the gas lines and valves reduce the risk of condensation impairing their functioning.

The following points should be noted when carrying out low-flow anaesthesia in practice, particularly during extensive surgery and at low room temperatures.

- Hoses should be routed so that condensation can collect at the lowest point and cannot flow back to the patient or to the breathing system. Condensation must be drained from the hoses at regular intervals.
- Water traps should be used for surgery lasting more than 1.5 hours. They must be positioned at the lowest point in the system so that the condensation can flow into the water traps. Water traps must be drained regularly.

Pressure and flow measurement may be impaired if condensation flows into the breathing system.

- The breathing system must be replaced by a dry one in such cases.
- The microbial filter must be protected against condensation. Although insensitive to moisture, condensation will increase the resistance in the microbial filter! The microbial filter must then be replaced. If condensation collects in the pressure measuring line:
 - Replace the pressure measuring line and its filter.
 - Install the T-piece and filter for the sampling hose with the connection pointing upwards. A new T-piece must be fitted after every patient.
 - Install the Y-piece with Luer-Lock connection, measuring port pointing upwards. If microbial filters are used in the breathing system, the Y-piece and hoses must be replaced after every patient. If microbial filters are used in the Y-piece: the Y-piece and hoses must be replaced every day.
- The sampling hose must be replaced if there are droplets of condensation in the hose.

Recommendation

If condensation is to be expected:

- Replace the breathing system and piston cylinder unit every day.
- Replace the soda lime every day.

If the breathing system and piston cylinder unit cannot be replaced every day:

- Drain the condensation frequently with the aid of a disinfectant.
- Leave the breathing system and piston cylinder unit open for a few hours so that they can dry.

Operation of the ceiling version

Cato with ceiling suspension unit DVE 808X

The DVE 808X with corresponding lift and brake control, as well as the required mount, should be used as the ceiling suspension unit for the Cato.

The DVE is turned and pivoted manually after releasing the brakes.

The ceiling version of the Cato includes a mounting arm with which the machine is coupled to the mount on the DVE. Both the mounting arm and the mount are equipped with sensors ensuring that the Cato has been correctly fitted and coupled. This is signalled by green LEDs on the touch-sensitive keypad of the pendant control panel.

The main arm of the DVE 808X with Cato may have a maximum length of 100 cm plus 50 cm for the extension arm.

The worktop on the ventilation part of the Cato is normally 83 cm above the floor. The height of the DVE can be adjusted relatively by +53 / -7 cm.

The rate of lift equals 15 mm/s. Further technical details can be found in the Instructions for Use/Installation of the DVE 808X.

Unterbrechungsfreie Stromversorgung

Bei einem Deckengerät entfallen die Hilfs-Netzsteckdosen.

General requirements

DVE units equipped for connection of a Cato must be capable of carrying a load of 250 kg with the safety specified by EN 60601-1.

It should be noted that these requirements must also be met when the mounting arms are positioned at an adverse angle. The structural conditions must also be checked to ensure that they can safely bear the resultant loads.




Maximum permissible total weight = 200 kg. Depending on the DVE used and on the prevailing conditions, the Cato can be raised off the floor to a height of approx. 60 cm. It must not be moved across or positioned above people or life-support systems!

No spare gas cylinders

Spare gas cylinders are not installed due to lack of space and because they are too heavy.


Dimensions

The dimensions of the ceiling version can be seen in the dimensioned drawing on page 114.

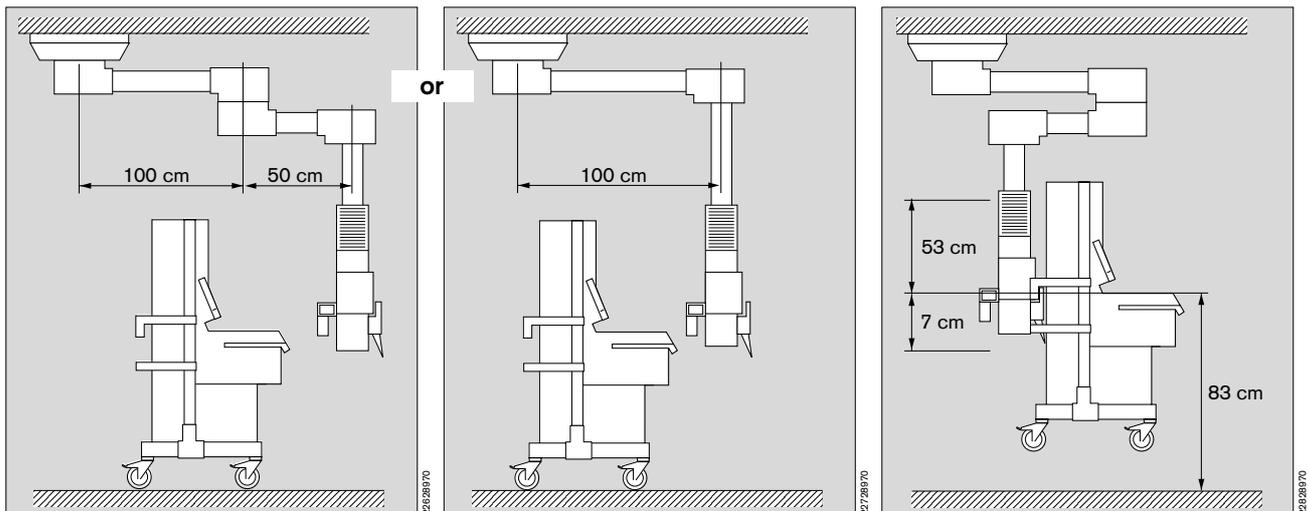

Connecting the Cato ceiling version

1 Swing supporting flap upwards!

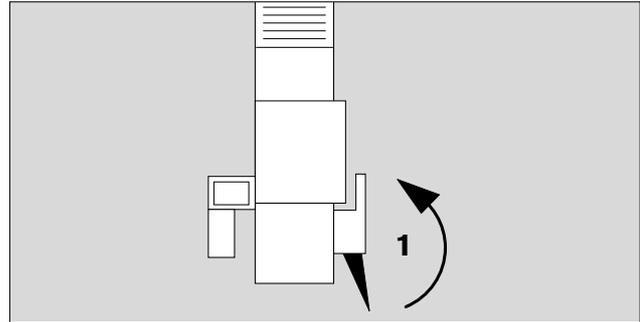


2 Press key until DVE reaches lower limit stop.

3 Press key to release the brake of the ceiling and/or intermediate bearing.

The DVE can then be swivelled.

4 Key for releasing the brake of the head bearing.

The mount for the Cato can now be swivelled.

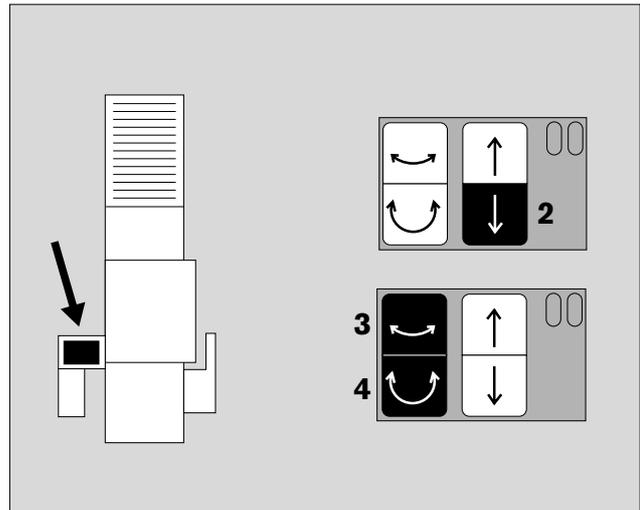


5 Swing supporting flap down again.


The DVE can no longer be operated via its own keypad.

- Plug the mains connector of the Cato into the socket on the DVE.


- Plug the gas withdrawal connectors into the corresponding withdrawal sockets.

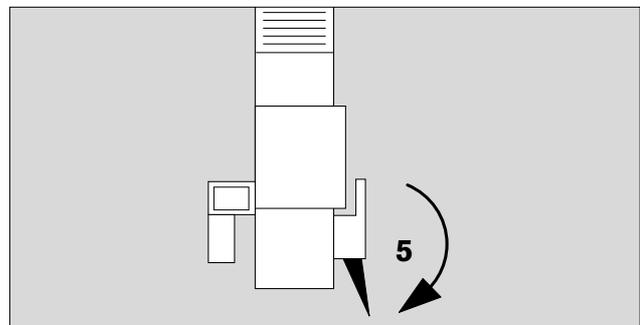


6 Move Cato right up to the mount on the DVE.

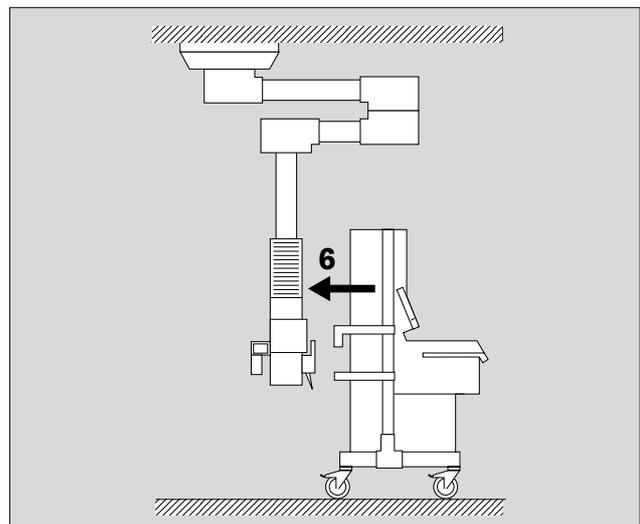


- 1 Both green LEDs on the touch-sensitive keypads on the Cato and DVE light up when correctly positioned.

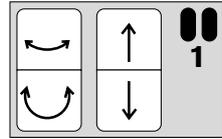

- The DVE can now be controlled (via the keypad in the handles of the Cato)!
 When Cato has been raised approx. 5 cm, the keypad on the DVE also becomes active in parallel to those on Cato's handles.

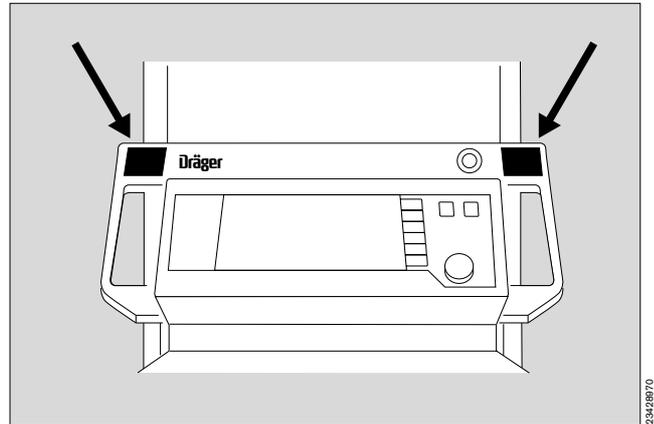

- 2 Press key until Cato reaches the required working height.

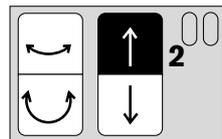


- 3 Press key to release brake for the ceiling and/or intermediate bearing.

Cato can now be swivelled.

- 4 Key to release the brake for the head bearing.

Cato can be turned.

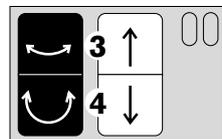

Disconnecting the Cato ceiling version


Any obstacles in Cato's path as it descends will be subject to Cato's full weight (approx. 200 kg) and may be crushed.



Before lowering Cato, always ensure that there is nothing to prevent it from being deposited directly on the floor!

- 5 Press key until lower limit stop is reached.

- Disconnect mains plug and gas supply lines of the Cato.

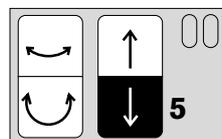


Descriptions

Disconnecting the Cato ceiling version

What to do if problems arise

Additional maintenance and care instructions


1 Move Cato away from the DVE.

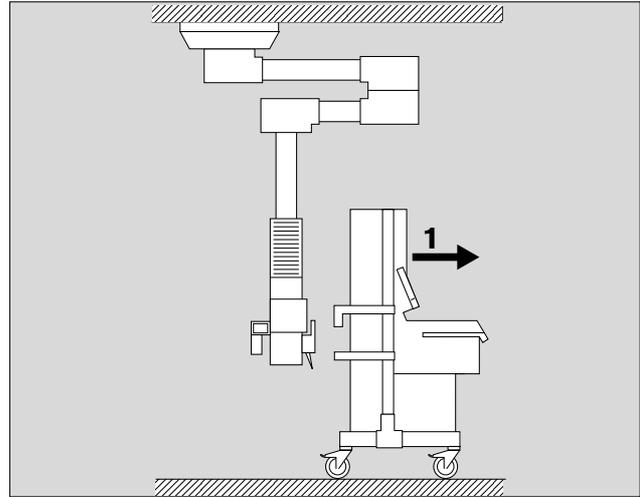

2 Swing supporting flap upwards.

- The DVE can now be controlled via its own keypad again!

As from approx. 5 cm above the lower limit position, the DVE can also be moved upwards even when the supporting flap is lowered.

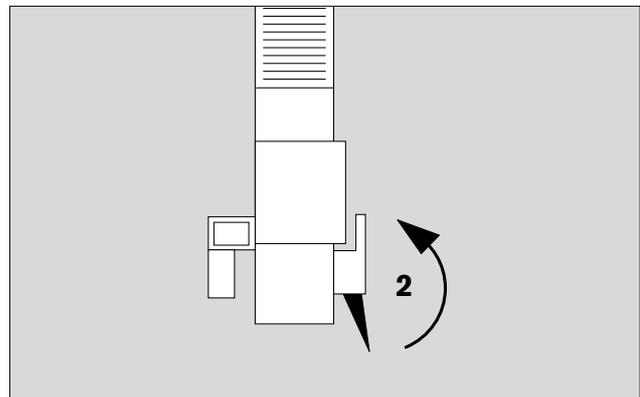

What to do if problems arise

- If the machine inadvertently strikes an obstacle before reaching the floor, the DVE will stop **before** it is forcibly disconnected and **cannot** be controlled via the keypad on the Cato.

The machine can then **only be raised** via the keypad on the DVE.

- If the machine subsequently **remains at an angle** on its mounting journal:

Briefly lower it onto the ground and pick it up again with the DVE.

- The mount may suffer damage in extreme cases!

Call DrägerService if damage is suspected. Depending on how serious the damage is, the Cato may crash to the ground when raised again!

Additional maintenance and care instructions

- The windows on the optical sensors must be cleaned!
Abrasives and scratching agents must not be used.

The sensors are resistant to all cleaning agents and disinfectants usually used in hospitals.

Abbreviations used

Abbreviation	Significance	Abbreviation	Significance
AIR	Compressed air for medical use	IPPV	Automatic ventilation mode: intermittent positive pressure ventilation
AMV	Minute ventilation	LED	Light-emitting diode
APL	Adjustable pressure limitation	LED display	7-segment display with light-emitting diodes
AW-Temp	Inspiratory breathing gas temperature	Man/Spont	Manual ventilation or spontaneous breathing
BAG	Connection for breathing bag	Mean	Mean pressure
BW	Body weight	NTC	Resistance sensor with negative temperature coefficient
CAL	Calibration has been performed	Off	Switched off, deactivated
C _{syst}	System compliance	ON	Switched on, activated
C _{pat}	Patient compliance	ORC	Oxygen Ratio Control
DGHM	German Society for Hygiene and Microbiology	P _{aw}	Airway pressure
et CO ₂	End-expiratory CO ₂ concentration	PC	Personal computer (IBM-compatible)
Fet Des.	End-expiratory desflurane concentration	PCV	Ventilation with pressure limitation
Fet Enf.	End-expiratory enflurane concentration	P _{peak}	Actual measured peak pressure
Fet Hal.	End-expiratory halothane concentration	P _{lat}	Plateau pressure
Fet Iso.	End-expiratory isoflurane concentration	Pleth.	Plethysmogram
Fet N ₂ O	End-expiratory N ₂ O concentration	P _{max}	Limit pressure
Fet Sev.	End-expiratory sevoflurane concentration	P _{mean}	Mean airway pressure
Fi Des.	Inspiratory desflurane concentration	Power	Electricity supply
Fi Enf.	Inspiratory enflurane concentration	PEEP	Positive end-expiratory pressure
Fi Hal.	Inspiratory halothane concentration	SIMV	Synchronized intermittent mandatory ventilation
Fi Iso.	Inspiratory isoflurane concentration	SpO ₂	Functional O ₂ saturation
Fi Sev.	Inspiratory sevoflurane concentration	Ti:TE	Ratio of inspiration time to expiration time
Fi N ₂ O	Inspiratory N ₂ O concentration	TiP:Ti	Ratio of inspiratory pause time to inspiration time
Fi O ₂	Inspiratory O ₂ concentration	V̇	Inspiratory and expiratory flow
fIPPV	IPPV frequency	V̇ _E	Minute ventilation
fIMV	SIMV frequency	V̇ _{FG}	Fresh gas flow
Flow	Expiration flow	V _{max}	Maximum inspiration flow
Freq	Respiration rate	V _T	Tidal volume, stroke volume
Hb	Haemoglobin	V _C	System volume
HbCO	Carbon monoxide haemoglobin	ZV	Piped medical gas supply (for compressed air, vacuum, N ₂ O and O ₂)
HbO ₂	Oxyhaemoglobin		
HLM	Heart-lung machine, mode for ...		
in CO ₂	Inspiratory CO ₂ concentration		
INOP	Malfunctioning		

Symbols used

Symbol	Significance
	Pulse rate
	Defibrillator strength
	Lower alarm limit
	Upper alarm limit
	Alarm monitoring inactive
	Cursor frame in menus
	Close menu, return to preceding menu level
	Changeover switch for standby and measuring modes
	Cyclic activation of basic pages
	Display standard page
	Suppress alarm tone for 2 minutes
	Important note!
	Connection for equipotential bonding
	Protection class type BF (EN 60601-1)
	Request for calibration
	Action has been completed successfully
	Action is being carried out
	Warning message
	Caution message
	Advisory message
	Alarm limit inactive

Index

Abbreviations	141	Bacterial filter, pressure measuring hose	106
Acids, disinfectants	83	Bacterial filter, secretion aspiration	106
Adult mode	52	Basic configuration, monitor	54
Advisory messages (Messages, Cause and Remedy)	74	Batteries, disposal	88
Advisory, concept / display	65	Baud rate	53
After use	48	Breathing system, assembling	80
AIR changeover	37	Breathing system, checking	23
Alarm concept	65	Breathing system, dismantling	82
Alarm information	62	Breathing system, elements	112
Alarm limits, activating	56	Breathing system, fitting	90
Alarm limits, reduction	44	Breathing system, heating	89
Alarm limits, specifying	54	Breathing system, removing	80
Alarm modes, ventilation modes	67	Breathing system, technical data	117
Alarm priority	65	Bronchial aspiration, secretion aspiration	25
Alarm tone, suppress	66	C-Lock	96
Alarms, showing	66	Calibration, flow sensor	29, 30, 55
Alcohols, disinfectant	83	Calibration, flow sensor, manual	30
Aldehyde, disinfectant	83	Calibration, O ₂ sensor	55
Ammonium compounds, quaternary	83	Calibration, O ₂ sensor, manual	29
Anaesthesia ventilation	36	Carboxyhaemoglobin	97
Anaesthetic agent measurement, measuring principle	129	Care	79
Anaesthetic agent vaporiser, Vapor	22	Care intervals	84
Anaesthetic agent, defining	56	Care of the ceiling version	140
Anaesthetic gas scavenging system, checking	22	Care schematic	84
Anaesthetic gas scavenging system, connecting	92	Cause (Messages, Cause and Remedy)	74
Anaesthetic gas scavenging system, removing	80	Caution messages (Messages, cause and remedy)	74
APL valve, pressure limiting valve	32	Caution, definition/display	65
Applying the SpO ₂ sensors	98	Ceiling version, coupling	138
Arrow symbol	14	Ceiling version, disconnecting	139
Artefacts, avoiding, SpO ₂ measurement	97	Ceiling version, operation	137
Audible signals, adjusting	52	Ceiling version, what to do if problems arise	140
Auto WakeUp function	8	Changing parts	47
Automatic calibration, flow	29, 55	Checking against checklist	21
Automatic calibration, O ₂	29, 55	Checking, anaesthetic gas scavenging line	22
AutoSet, ventilation alarms	61	Checking, automatic ventilation function	105
Auxiliary air, anaesthetic gas scavenging system	22	Checking, breathing system	23
Auxiliary electrical equipment, connecting	18	Checking, emergency breathing bag	23
Auxiliary power sockets	18, 20, 116	Checking, linearity of the O ₂ sensor	55, 104
Auxiliary power sockets, auxiliary mains sockets	18, 20, 116	Checking, manual ventilation function	105
		Checking, O ₂ flush	25
		Checking, O ₂ shortage signal	102
		Checking, ORC	25
		Checking, power failure alarm	102
		Checking, readiness for operation	101

Checking, reserve gas cylinders	24	E lements of the breathing system	112
Checking, secretion aspiration	25	Emergency	7
Checking, soda lime	23, 45	Emergency ventilation bag	23
Checking, supply of anaesthetic agent	102	Emergency ventilation, gas failure	37
Checking, Vapor	22	End of operation	48
Checking, water separator	44, 106	Environmental factors, Technical Data	116
Checking, water traps	24, 44	Equipment overview, What's what	108
Cleaning	83	Equipotential bonding connection	5, 94
Cleaning agents	83	Erasing, list	50, 62
CO ₂ alarm on/off	61, 66	Erasing, trend	50, 59
CO ₂ alarm, fixed upper alarm limit	66	Ether	5
CO ₂ measurement, measuring principle	129	Ethylene oxide, sterilisation method	86
Compliance correction, automatic	40	External equipment, connecting	20, 94
Compliance correction, description	127		
Concept, alarms	65	F ailure, electricity/gas	9
Concept, operating	10	Filter, measured gas return line	106
Condensation separator	44	Filter, replacement intervals	106
Conductive rubber parts	5	Fire risk	5
Connecting the ceiling version	138	Flow chart of self-test	124
Connecting the equipment	18	Flow measurement, measuring principle	129
Consumable articles, disposal	85	Flow sensor, calibration	29, 30, 55
Cooling air filter	106	Flow sensor, fitting	93
Cursor frame	14	Flow sensor, removing	81
Cyclopropane	5	Fresh gas outlet, external	27, 38
		Fresh gas setting, notes	128
D ata bits	53	Full trend	59
Data cable	20, 94	Fuses	116
Data screen	57		
Date, setting	54	G as delivery, checking	24
Definitions, "low-flow / minimal flow"	132	Gas delivery, Technical Data	118
Description, S-ORC	135	Gas failure	8
Descriptions	123	Gas failure, emergency breathing	37
DGHM list	83	Gas mixer, gas delivery	24
Dialogue	10, 12	Grey fields	15, 51
Dimensions, stand-alone machine / ceiling version	114	Grey numerals	13
Disconnecting the ceiling version	139	H alogens, disinfectants	83
Disconnecting the ceiling version	139	HF surgery, SpO ₂ measurement	97
Disinfectants	83	HLM Mode	64, 68
Disinfecting	83	Hose changes	42
Dismantling parts	81	Hot-plate, breathing system	80
Display window	12		
Disposal of apparatus	88	I dentification, machine	116
Disposal of consumable articles	85	Intended use	5
Disposal of soda lime	82	Interface, pneumatic	73
DURASENSOR, SpO ₂ measurement	98	Interfaces, configuring, external equipment	53
Dysshaemoglobin	97	Interruption of operation	48

IPPV mode	39	Monitor functions	49
IPPV mode, selecting	39	Monitor, activating	57
K Keys, on monitor	13, 14, 110	Monitor, configuring in standby	51
Keys, on ventilator	11, 109	Monitor, keys	13, 60, 112
Kuhn system, paediatric	43	Monitor, operating elements	110
L anguage, selecting	54	Monitor, operation	57
Leakage test	42, 43	Monitor, structure	14, 110
Leakage test, description	127	Monitor, Technical data	118
Limit settings	40	N ₂ O lock	37
Linearity of the O ₂ sensor, checking	55, 104	Negative pressure, secretion aspirator	25
List entry, defining	53	Neonate mode	52
List screen	62	Notes on setting the fresh gas flow	128
List, erasing	50, 62	O ₂ flush, checking	25
Location of the subsystems	73	O ₂ measurement, measuring principle	129
Location of the valves	73	O ₂ sensor, calibration	55
Lock N ₂ O	37	O ₂ sensor, disposal	88
Low-flow, definition	132	O ₂ sensor, gas sampling, fitting	90
M achine identification	116	O ₂ sensor, inspiratory, dismantling	81
Maintenance intervals	106	O ₂ sensor, inspiratory, fitting	90
Maintenance of the ceiling version	140	O ₂ shortage alarm	37
Malfunctions, system faults	72	O ₂ ... ; see also oxygen...	37
Manual checks	21	Operating concept, general	10
Manual ventilation	37	Operating concept, monitor	13
Master switch	10	Operating concept, ventilator	11
Measuring principle, anaesthetic agent measurement	129	Operating elements, measuring tube bank	111
Measuring principle, CO ₂ measurement	129	Operating elements, monitor	110
Measuring principle, flow measurement	129	Operating elements, ventilator	109
Measuring principle, O ₂ measurement	129	Operating voltage	116
Measuring principle, pressure measurement	132	Operation	35
Measuring principle, SpO ₂ measurement	131	Operational readiness, checking	101
Measuring principle, temperature measurement	132	Operational readiness, checking against checklist	21
Measuring tube bank, operating elements	111	ORC, checking	25
Measuring unit selection	53	ORC, description	135
MEDIBUS interface	20, 94	Overall contents	3
Medical gas supply, Technical Data	117	OXISENSOR, SpO ₂ measurement	95
Messages (Messages, Cause and Remedy)	74	Oxygen ...; see also O ₂ ...	37
Methaemoglobin	97	Oxygen-releasing disinfectants	83
Methods (cleaning, disinfection, sterilisation)	83	P aediatric	42
Microbial filter, breathing system	23	Paediatric hoses	42
Microbial filter, fitting	93	Parameters, defining	53
Minimal flow, definition	132	Parity	53
Minute volumeter	63	Patients, changing	47
Mobile telephones	5	Phenols, disinfectants	83
		Pipeline gas supply (medical gases)	24

Piston pump, assembly	89	Self-test, flow chart	124
Piston pump, dismantling	82	Self-test, interrupting	34
Piston pump, fitting	90	Sensor, selecting, SpO ₂ measurement	94
Piston pump, removal	80	Shortage alarm, O ₂	37
Plethysmogram/SpO ₂	63	SIMV mode	41
Pneumatic interface	73	SIMV mode, selecting	41
Power failure	8	SIMV ventilation, description	133
Power failure, manual ventilation	37	Soda lime container, filling and fitting	91
Power input	116	Soda lime, changing during operation	45, 91
Power supply	116	Soda lime, checking	23, 45
Power supply, connecting	18, 94	SpO ₂ measurement, measuring principle	131
Power supply, connecting	18, 94	SpO ₂ sensor, selecting	94
Preparing for use	17	SpO ₂ sensors, application	98
Pressure measurement, connection	112, 113	SpO ₂ /plethysmogram	63
Pressure measurement, measuring principle	132	Spontaneous breathing	36
Pressure measuring hose, mounting	91	Standard alarm limits, activating	56
Pressure-limiting ventilation	40	Standard screen / monitor	16, 57
Printer connection	20, 94	Standby	34, 48
Printer entry definition	53	Standby screen	50
Protocol interface	20, 94	Steam sterilisation	86
Pulse tone	52, 64	Sterilisation	83
		Sterilisation methods	86
Quick start in an emergency	7	Stop bits	53
		Stripping down machine	80
Rear of machine	108	Subsystems, location of	73
Rear, machine	108	Switching off, end of operation	48
Record, selecting	53	Symbols	142
Reduced alarm limits, monitoring	44	System faults, malfunctions	72
Remedy (Messages, cause and remedy)	71		
Replacement intervals	106	Technical data	117
Reserve gas cylinders, checking	24	Temperature measurement, measuring principle	132
Return arrow, arrow symbol	14	Temperature sensor, breathing gas, removing	81
Rotary control	10, 14	Temperature sensor, fitting	93
		Test result, self-test (leakage, compliance)	31
Sample line, connecting	92	Tidal volumeter	63
Sampling rate CO ₂ /O ₂	53	Time, setting	54
Screen pages, "browsing"	13, 62	Tolerance, measured values for Autoset vent. al.	61
Screen saver	14, 50		
Screen, data	16, 57	Tone	10
Screen, list	62	Tone sequence, EN/Dräger standard	10, 54, 65
Screen, standard	16, 57	Tone, pulse	52, 64
Screen, standby	50	Tone, setting	52, 64
Screen, trend	16, 58	Trend erasing	50, 59
Secretion aspirator	46	Trend page / screen	58
Secretion aspirator, checking	25	Trend page / screen, zoom function	58
Secretion aspirator, installing	93	Trend, full display	59
Selecting the measuring unit	53		
Self-test	26		

U nterruptible power supply	20
Uses	5
V alves, fitting	90
Valves, location of	73
Vapor units	5, 6, 19, 22
Vapor, checking	22
Vapor, interlock	22
Vapor, plug-in adapter	22
Vapor, safety fill	22
Ventilation parameters, adjusting	39
Ventilation, manual	36
Ventilation, mechanical	39, 41
Ventilator messages (Messages, Cause and Remedy)	78
Ventilator, operating controls	109
Ventilator, Technical Data	117
Voltage, current supply	116
Volumeter "curve"	63
Volumeter, starting	63
W arning messages	74
(Messages, cause and remedy)	
Warning, definition/display	65
Water traps, checking	24, 44, 47
Water traps, gas sampling line	44, 47, 92
Weight	116
What's what	107
White fields	51
Z oom function, trend screen	58

These Instructions for Use apply only to
Cato edition
with Serial No.:

If no Serial No. has been filled in by
Dräger these Instructions for Use are
provided for general information only and
are not intended for use with any specific
machine or device.



Directive 93/42/EEC
concerning Medical Devices

Dräger Medizintechnik GmbH

Germany

🏠 Moislinger Allee 53 – 55
D-23542 Lübeck

☎ +49 451 8 82 - 0

📠 26 80 70

FAX +49 451 8 82-20 80

🌐 <http://www.draeger.com>